# AZ Primer

<b>en</b> Instructions for use	<b>de</b> Gebrauchsanleitung	<b>fr</b> Mode d'emploi	<b>nl</b> Gebruiksaanwijzing
<b>es</b> Instrucciones de uso	<b>it</b> Istruzioni per l'uso	<b>ru</b> Инструкция по применению	<b>pl</b> Instrukcja obsługi
<b>ro</b> Instrucțiuni de utilizare	<b>sv</b> Bruksanvisning	<b>el</b> Οδηγίες χρήσης	<b>pt</b> Instruções de uso

<b>Manufacturer</b> SHOFU INC. 11 Kamitakamatsu-cho, Fukuine, Higashiyama-ku, Kyoto 605-0983, Japan www.shofu.com	<b>EC REP</b> SHOFU DENTAL GmbH An der Pönt 70, 40885 Ratingen, Germany
<b>SHOFU DENTAL CORPORATION</b> 1225 Stone Drive, San Marcos, California 92078-4059, USA	<b>SHOFU DENTAL ASIA-PACIFIC PTE.LTD.</b> 10 Science Park Road, #03-12, The Alpha, Science Park II, Singapore 117684

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<b>en</b> READ CAREFULLY BEFORE USE
-------------------------------------

### Primer for bonding alumina or zirconia

**PRODUCT DESCRIPTION**
AZ Primer is a priming agent designed to treat the surfaces of alumina-based or zirconia-based restorations to enhance bond to resin materials. AZ Primer can be used intraorally and extraorally.

**INDICATIONS FOR USE**
Bonding resin to alumina or zirconia restorations

- Cementation of restoration with adhesive resin cement
- Repair of fractured restorations using composite resin

**PRECAUTIONS AND WARNINGS**

- Do not use this product on patients with known allergies to this material,methacrylate monomer and/or acetone.
- Operators with known allergies to this material, methacrylate monomer and/or acetone must not use this product.
- In case of inflammation or other allergic reactions occur on either the patient or operator, immediately discontinue use and seek medical advice.
- Use in well ventilated areas to avoid inhalation of acetone (several appropriate ventilations per hour are recommended). If inhalation causes nausea or illness, take rest and keep warm in fresh air.
- Avoid any source of ignition since this product is flammable.
- Use protective gloves and glasses to avoid sensitization to this product. In case of accidental contact with oral soft tissue or skin, immediately blot with alcohol moistened cotton ball, and rinse with plenty of water. Medical (dental) gloves provide some protection against direct contact with this material; however some monomers may penetrate through the gloves very quickly to reach the skin. Therefore, should this material come in contact with the gloves, immediately remove them and wash hands with plenty of water.
- In case of contact with eyes, immediately flush the eyes with plenty of water and seek medical advice.
- If this product comes in contact with oral soft tissue, the tissue may turn whitish, however, it should resolve in 2 or 3 days. Avoid touching or brushing the area.
- Follow the instructions for use of each material, instrument or equipment to be used in conjunction with this product.
- Do not use this product for any purposes other than specifically outlined in these instructions for use.
- Use this product within the expiration date indicated on the package or container. (Example ☞ YYYY-MM-DD → Year-Month-Date of the expiration date)
- This product is intended for use by dental professionals only.

- DIRECTIONS FOR USE**
- Pre-treatment of the restoration surface
    - Extraoral use
      - Sandblast the adhesive surface of zirconia-based or alumina-based restoration with 50-100 µm aluminum oxide at approx. 0.2-0.3 MPa/2-3 bar. Then, perform ultrasonic cleaning and dry with clean air. Sandblast can be replaced by roughing the surface with a suitable abrasive (eg. diamond bur etc).
    - Intraoral use
      - Place rubber dam and avoid direct contact with oral tissue. Roughen the adhesive surface by using a dental abrasive (eg. diamond bur etc), rinse with water and dry with clean air.
  - Application of AZ Primer
    - Dispense adequate amount of AZ Primer on a dish. Apply to the entire adhesive surface in one layer with a disposable brush. Leave it undisturbed for 10 sec. and dry naturally.
  - Bonding to resin material
    - Brush resin material according to the instructions for use of each product.

- (Notes)
  - Do not mix with any other primers or bonding systems.
  - Tightly close the cap after each use as this product contains volatile components. Use all the dispensed material promptly.
  - Use saliva ejector or suction during the procedure.
  - Avoid contamination of the restoration surface to be bonded. If the surface is contaminated by saliva or blood, thoroughly clean the surface with an alcohol moistened cotton ball. Then, dry and reapply the material.
  - Use all the dispensed material promptly on one patient.
  - Do not reuse disposable brushes to avoid cross-contamination.

**COMPOSITION**
Acetone, phosphonic acid monomer, Thioctic acid monomer

**STORAGE**
Store at room temperature (1-30 °C/34-86 °F). Keep away from direct sunlight and any source of ignition.

**CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.**

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