**Manufacturer**  
SHOFU INC.  
11 Kamitakamatsu-cho, Fukuine,  
Higashiyama-ku, Kyoto 605-0983, Japan  
www.shofu.com

**EC REP**

SHOFU DENTAL GmbH  
An der Pönt 70, 40885 Ratingen, Germany

02.0:2020-09  
300030333

**en**

READ CAREFULLY BEFORE USE

## PRODUCT DESCRIPTION

AcryPoint-E are used for adjusting, finishing and polishing acrylic dentures or other acrylic appliances as well as plastic impression trays. Use AcryPoint-E "Coarse" (dark grey) for adjustment, "Medium" (brown) for finishing and "Fine" (light grey) for polishing.

## PRECAUTIONS AND WARNINGS

- Never exceed the maximum permissible rotation speeds described in Table A.
- Insert the shank completely into the chuck of the handpiece.
- Do not use damaged or defective instruments i.e. corrosion, surface flaw, deformation, dirt and exposure of shank due to repeated use.
- Perform test run to check safety and smooth operation without run-out prior to use.
- Avoid any use at acute angles or with excessive pressure.
- Use local dust extractor and protective equipment while using.
- This product is intended for use by dental professionals only.
- Do not use this product for any purposes other than the intended use.
- Dispose in accordance with relevant local and national regulation.

## STORAGE AND STERILIZATION

- Avoid exposure to moisture, corrosive chemical and its vapor. Keep away from external pressure (physical load) and avoid contamination.
- When necessary, perform autoclave under the condition described in Table B.
- When necessary, sterilize before use.
- When necessary, remove any contaminants with washer disinfectant, then sterilize.
- Follow the instructions for use of washer disinfectant and sterilizer, respectively.
- Do not use chlorine disinfectant liquid.
- For the details about REPROCESSING procedures, visit [www.shofu.de](http://www.shofu.de).

**CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.**

Location of the symbols glossary:  
[https://www.shofu.com/en/symbols\\_glossary/](https://www.shofu.com/en/symbols_glossary/)

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