SHOFU DENTAL CORPORATION
1225 Stone Drive, San Marcos,
California 92078-4059, USA

SHOFU DENTAL ASIA-PACIFIC PTE.LTD.
10 Science Park Road, #03-12,
The Alpha, Science Park II, Singapore 117684

EC REP

SHOFU DENTAL GmbH
An der Pönt 70, 40885 Ratingen, Germany

04.0:2020-09
300030342

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en READ CAREFULLY BEFORE USE

PRODUCT DESCRIPTION

This kit is designed for polishing and superpolishing amalgam and precious alloys. It includes three each Brownie cups and Mini-Points, two each Greenie cups and Mini-Points, one each Supergreenie (yellow band) cup and Mini-Point. Brownie, Greenie and Supergreenie Polishers are autoclavable for reuse.

PRECAUTIONS AND WARNINGS

●Never exceed the maximum permissible rotation speeds described in Table A. ●Insert the shank completely into the chuck of the handpiece. ●Do not use damaged or defective instruments i.e. corrosion, surface flaw, deformation, dirt and exposure of shank due to repeated use. ●Perform test run to check safety and smooth operation without run-out prior to use. ●Avoid any use at acute angles or with excessive pressure. ●Use local dust extractor and protective equipment while using. ●Amalgam should be polished under running water to avoid heat generation. Thoroughly remove amalgam dust. ●This product is intended for use by dental professionals only. ●Do not use this product for any purposes other than the intended use. ●Dispose in accordance with relevant local and national regulation.

STORAGE AND STERILIZATION

●Avoid exposure to moisture, corrosive chemical and its vapor. Keep away from external pressure (physical load) and avoid contamination. ●Perform autoclave under the condition described in Table B. ●When using intraorally, sterilize before each use. ●For reuse after using, remove any contaminants immediately with washer disinfectant, then sterilize. ●Follow the instructions for use of washer disinfectant and sterilizer, respectively. ●Do not use chlorine disinfectant liquid. ●For the details about REPROCESSING procedures, visit www.shofu.de.

CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.

Location of the symbols glossary:
https://www.shofu.com/en/symbols_glossary/

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