Aquasil® Soft Putty – Regular Set

Aquasil® Hard Putty – Fast Set

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# Aquasil® Soft Putty – Regular Set

# Aquasil® Hard Putty – Fast Set

Very high viscosity precision impression materials based on quadrafunctional hydrophilic siloxanes

CAUTION: For dental use only.

USA: Rx only.

Content	Page
1 <b>Product description</b> .....	2
2 <b>Safety notes</b> .....	3
3 <b>Step-by-step instructions</b> .....	4
4 <b>Hygiene</b> .....	6
5 <b>Relevant material properties</b> .....	6
6 <b>Lot number, expiration date and correspondence</b> .....	6

## 1 Product description

**Aquasil® Impression Materials** are suitable for all precision impression techniques where excellent hydrophilic properties, dimensional accuracy, high tear strength, and resistance to permanent deformation are needed.

Aquasil® Impression Materials comply with the requirements of ISO 4823 for dental elastomeric impression materials.

### 1.1 Indications

- Aquasil® Soft Putty is a regular setting very high viscosity type impression material with optimum flexibility for easy removal from undercuts and is the ideal tray material for the **double mix technique**. It may also be used as primary impression material for the putty/wash technique.
- Aquasil® Hard Putty is a fast setting, very high viscosity type impression material with ideal mechanical properties as tray material for the **putty/wash technique**. It may also be used as tray material for the double mix technique for quadrant impressions.

### 1.2 Contraindications

- Do not use Aquasil® Hard Putty in combination with full arch impression trays for the double mix technique. The low flexibility of the set material may interfere with tray removal.

### 1.3 Delivery forms

- Aquasil® Soft Putty and Aquasil® Hard Putty are available in 450 ml cans.

## 1.4 Composition

### Base:

Cristobalite silica crystalline  
Vinylidimethicone  
Polydimethylsiloxane, methyl terminated  
Zeolithe  
Paraffine oil  
Polydimethylsiloxane  
Pigments

### Catalyst:

Cristobalite silica crystalline  
Vinylidimethicone  
Zeolithe  
Paraffine oil  
Platinum complex + Pt-metal  
Calcium carbonate  
Divinyltetramethyldisiloxane  
Pigments

## 2 Safety notes

Be aware of the following general safety notes and the special safety notes in other chapters of these Instructions for Use.

### Safety alert symbol.

- This is the safety alert symbol. It is used to alert you to potential personal injury hazards.
- Obey all safety messages that follow this symbol to avoid possible injury.

### 2.1 Warnings

- **Avoid eye contact** to prevent irritation and possible corneal damage. In case of contact with eyes, rinse with plenty of water.
- **Ingestion:** Do not swallow or take internally. If accidental swallowing occurs, drink lots of water. Seek medical attention if symptoms persist.

### 2.2 Precautions

This product is intended to be used only as specifically outlined in these Instructions for Use.

Any use of this product inconsistent with these Instructions for Use is at the discretion and sole responsibility of the dental practitioner.

- Use protective measures for the dental team and patients such as glasses and rubber dam in accordance with local best practice.
- To prevent the cans from exposure to spatter or spray of body fluids or contaminated hands it is recommended that the cans are handled offside the dental unit with clean/ disinfected gloves.
- Remove unset Aquasil® Impression Material from clothing with a suitable cleaning solvent.
- Interactions:
  - Astringents based on aluminium salts may interfere with the setting reaction of vinyl polysiloxanes. Please consult Instructions for Use of astringent to be used.
  - Aquasil® Impression Materials should not be mixed with, or used in conjunction with polyether materials, condensation cured silicones or polysulfide.
  - "Sulphur-containing" polymeric gloves may interfere with the setting reaction of Aquasil® Impression Materials.
    - Do not wear such gloves when kneading Aquasil® Soft Putty.
    - Do not touch retraction cords. Handle cords with college pliers.
    - Do not touch prepared tooth.

### 2.3 Adverse reactions

- Eye contact: Irritation and possible corneal damage.

### 2.4 Storage conditions

Inadequate storage conditions will shorten the shelf life and may lead to malfunction of the product.

- Keep the product out of sunlight.
- Store at temperatures between 10 °C and 24 °C (55 °F and 77 °F). Use the product at room temperature.
- Do not use after expiration date.

## 3 Step-by-step instructions

### 3.1 Tray preparation

Silfix Adhesive should be used in all techniques with all types of impression trays.

1. Brush a thin layer of Silfix onto tray and allow to dry for 3 minutes.

### 3.2 Preparation of impression area

1. Clean Impression area with air/water spray and remove excess water.

### 3.3 Mixing and filling the tray

Take equal amounts of catalyst and base using the colour coded measuring spoons and knead with fingertips until colour of mix is uniform. Immediately fill prepared tray.

NOTE: Do not interchange measuring spoons and tub lids.

### 3.4 Impression taking

Please take into account the product specific recommendations for the handling of the Aquasil® Impression Materials mentioned below. For the light-viscosity materials, please also see the respective Instructions for Use.

APPLICATION	Aquasil® Soft Putty	Aquasil® Hard Putty	Aquasil® Ultra LV/XLV Regular Set	Aquasil® Ultra LV/XLV Fast Set
Mixing Time	0:30	0:30	n/a	n/a
Total Time for Application <sup>1</sup>	1:30	1:15	2:15	1:15
Minimum Time in Mouth (ISO)	3:00	2:00	2:45	1:45
Setting Time <sup>2</sup>	4:30	3:15	5:00	3:00

<sup>1</sup> Describes the time from the start of mix till the impression has to be seated and corresponds to the Total Work Time (ISO) for hand mixed materials and to the Minimum Work Time (including mixing time) (ADA) for cartridge materials.

<sup>2</sup> Total time from start of mix to complete set.

### 3.4.1 Double mix technique

Beside common steps for the Double Mix Technique, take into account the special recommendations for the handling of Aquasil® Impression Materials mentioned below.

1. Inject light body with cartridge system onto and around tooth preparations. Squeeze dispenser trigger with moderate and even pressure to begin mixing material. Material will stop flowing after trigger is released. It is suggested to syringe completely around preparation at the gingival margins twice, maintaining the syringe tip in the impression material. It is not necessary to blast with air for thinning out the material. Aquasil® Ultra LV and XLV Impression Material's hydrophilic properties improve adaptation to the tooth enabling better "wetting".
2. Seat tray within 1 minute and 30 seconds after start of mix of the Regular Set Materials and 1 minute and 15 seconds after start of mix of the Fast Set Materials.
3. Aquasil® Impression Materials set faster in the mouth than on the bench. Retain impression in position for at least 3 minutes for the Regular Set Materials and 2 minutes for the Fast Set Materials. Prior to removal, make sure the impression is firm, resilient, and non-tacky.

### 3.4.2 Putty/wash technique

Due to the excellent flow characteristics it is recommended to choose Aquasil® Ultra XLV as the light body for the putty/wash technique. Additionally, less material compared to traditional light bodies is used in order to obtain optimal results.

Beside common steps for the putty/wash technique take into account the special recommendations for the handling of Aquasil® Impression Materials mentioned below.

1. For the primary impression seat tray within 1 minute and 30 seconds after start of mixing Aquasil® Soft Putty or 1 minute and 15 seconds after start of mix of Aquasil® Hard Putty. Retain impression in position for a minimum of 3 minutes for Aquasil® Soft Putty and 2 minutes for Aquasil® Hard Putty, respectively.
2. Remove, clean and prepare primary impression for the secondary impression.
3. Clean area to be duplicated with air water spray. Remove spray water.
4. Inject light body with cartridge system onto and around tooth preparations. Squeeze dispenser trigger with moderate and even pressure to begin mixing material. Material will stop flowing after trigger is released. It is suggested to syringe completely around preparation at the gingival margins twice, maintaining the syringe tip in the impression material. It is not necessary to blast with air for thinning out the material. Aquasil® Ultra LV and XLV Impression Material's hydrophilic properties improve adaptation to the tooth enabling better "wetting".
5. Syringe additional light body on remaining occlusal tooth surfaces and/or on prepared primary impression.
6. Seat tray within 2 minutes and 15 seconds after start of injecting Aquasil® Ultra LV/XLV Regular Set and 1 minute and 15 seconds after start of injecting Aquasil® Ultra XLV/LV Fast Set, respectively.
7. Aquasil® Impression Materials set faster in the mouth than on the bench. Retain impression in position for at least 2 minutes and 45 seconds for Aquasil® Ultra LV/XLV Regular Set and 1 minute and 45 seconds for Aquasil® Ultra LV/XLV Fast Set. Prior to removal, make sure the impression is firm, resilient, and non-tacky.

## 4 Hygiene

### 4.1 Cleaning

Rinse impression under cold water and blow dry.

### 4.2 Disinfection

The impression can be disinfected with a standard disinfecting solution or spray.

### 4.3 Tray cleaning

Use standard methods to clean the tray after removal of the impression material. Fix Solvent may be used to help removing any remaining layer of Silfix.

### 4.4 Casting

To avoid surface porosity of models, delay pouring with plaster for 15 minutes. The impression may be poured up to fourteen days later.

### 4.5 Disposal

Dispose of in accordance with local regulations.

## 5 Relevant material properties

	Aquasil® Soft Putty	Aquasil® Hard Putty
Delivery system	Tubs	
Mixing technique	Knead with fingertips until colour is uniform	
Mixing ratio	1 : 1	
Classification ISO 4823	Type 0: Very high consistency-putty	
Linear dimensional change (ISO)	0.05%	
Maximum strain in compression (ISO)	3.5%	2.5%
Recovery from deformation (ISO)	99.0%	
Permanent deformation (ADA)	1.0%	
Detail reproduction	< 50 µm	

## 6 Lot number ( ), expiration date ( ) and correspondence

1. Do not use after expiration date.  
ISO standard is used: "YYYY-MM" or "YYYY-MM-DD".
2. The following numbers should be quoted in all correspondence:
  - Reorder number
  - Lot number
  - Expiration date
3. Any serious incident in relation to the product should be reported to the manufacturer and the competent authority according to local regulations.

© Dentsply Sirona 2020-12-10

[These Instructions for Use are based on Master Version 04]

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Dentsply DeTrey GmbH