astron® CLEARsplint®

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MMA-free and thermoactive resin for the manufacturing of functional therapeutic comfort nightguards and interim restorations

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advantages of astron® CLEARsplint® that speak for themselves!

- extremely break resistant, hence optimal protection of the teeth
- thermoactive flexibility which provides greater, comfortable and tension-free wearing comfort
- hypoallergenic, free of MMA (residual monomer content of MMA=0 %)
- self-adjusting. This superb accuracy permits the closest fit
- always transparent - will not yellow with age, since amine-free

astron® CLEARsplint®

The best solution for indications as easing the mandibular joint (bruxism), for the protection of implants, the adjustment of the occlusion etc.

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Keramik Anmischbecher/ Ceramic mixing bowl

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Store powder and liquid in the refrigerator at 4 - 8 °C. (Practical tips: prior to use put mixing bowl into the refrigerator for 30 minutes to extend processing time - we recommend the use of ceramic - or glass bowls to avoid any chemical bond with the material). Thus working time will be extended of this heat-sensitive material. In order to avoid uncontrollable chemical reactions please use only mixing bowls and -spatulas which do not get in contact with other resins. Processing repairs as usual. Prior to processing repairs, put some CLEARsplint® liquid on area to be repaired and let it dry for some seconds.

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1. prepare and measure model and mark tooth equator – splint can be made above tooth equator
2. wax up splint with a wax plate
3. prepare protection wall and let it cure. Remove it and prepare on the tubers, resp. on both sides of the molar triangle an overflow canal
4. boil out the model and water it
5. isolate the still warm plaster model, dry it and fix with protection wall
6. remove powder and liquid from the refrigerator just prior to mixing and pouring
7. mixing ratio: 10 g powder to 7 g liquid (1 g powder to 0.7 g liquid or a multiple)
Important: respect mixing ratio thoroughly! For one dental restoration made of CLEARsplint® (splint or partial denture) approx. 8 - 10 g powder are required
8. mix it thoroughly for approx. 5 - 10 seconds as quickly as possible and bubble-free (use separate measuring cup and -spatula)
attention: pour resin immediately into protection wall and cure it as quick as possible at 2 bar and 50 - 53 °C (122 - 127 °F) for 45 minutes
9. remove denture mold from pressure pot and place it under cold running tap water for 15 minutes before removing dental restoration from pour mold
10. for removing splint from the plaster model, warm it up at approx. 50 °C (122 °F) and remove it carefully
11. trim and polish as usual
12. immerse dental restoration into handwarm water prior to placing it on the model or in the mouth (flexibilisation)

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1. prepare and measure model and mark tooth equator – splint can be made above tooth equator
2. wax up splint with a wax plate and invest model into flask
3. prepare counter part and after curing heat flask in water bath
4. remove wax, boil out model and counter part, water and isolate them
5. remove powder and liquid from the refrigerator just prior to mixing
6. mixing ratio: 10 g powder to 7 g liquid (1 g powder to 0.7 g liquid or a multiple)
Important: respect mixing ratio thoroughly! For one dental restoration (splint or partial denture) made of CLEARsplint® approx. 8 - 10 g powder are required
7. mix thoroughly for approx. 5 - 10 seconds as quickly as possible and bubble-free (use separate measuring cup and spatula)
8. as soon as mix reaches a doughy state (3 - 5 minutes) pack into regular denture mold using normal press-packing procedure
9. immerse in 50 - 53 °C (122 - 127 °F) warm water and cure for 60 minutes.
10. cool for 15 minutes under cold running tap water before removing from mold
11. trim and polish as usual
12. for removing dental restoration from the plaster model, warm it up at approx. 50 °C (122 °F) and remove it carefully
13. trim and polish as usual
14. immerse dental restoration into handwarm water prior to placing it on the model or in the mouth (flexibilisation)

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Care of the astron® CLEARsplint® nightguard and partial dentures: We recommend astron® DuraClean® – REF 261705
HINT: We recommend to water the finished CLEARsplint® restorations for approx. 3 – 4 hours, then to rinse it under running water with a soft brush and to disinfect it afterwards with an alcohol-free disinfectant.

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KENTZLER-KASCHNER DENTAL GmbH