Cavex Avalloy is a high copper alloy of the HighCopperSingleComposition-type. It consists of micro-fine lath-cut particles with a relatively high copper and low silver content. The bonding characteristics are comparable to a conventional amalgam: sufficient condensation pressure should be applied.

The amalgam made of **Cavex Avalloy** is free from the corrosion-prone and weak gamma-2 phase. Durable restorations with a lasting lustre can be made.

Alloy composition:	Silver (Ag)	: 45.0% (m/m)
	Tin (Sn)	: 30.5% (m/m)
	Copper (Cu)	: 24.0% (m/m)
	Zinc (Zn)	: 0.5% (m/m)

Cavex Avalloy is in full compliance with the latest version of the ISO24234 and ADA No. 1 specifications. The mercury contained in the pre-dosed capsules is in full compliance with ISO24234.

Indication
Stress-bearing fillings in posterior teeth (Class I and II).

Contraindications
• Use of amalgam is not recommended when a physician has deemed it unsuitable for the patient.
• Hypersensitivity (allergic reaction) to amalgam components.
In such cases the use of suitable alternative materials must be considered.

Warnings
• After placement or removal of dental amalgam restorations increased mercury concentration in blood and urine has been observed. According to available scientific knowledge this increase has not been associated with any adverse health effects.
• If placed in close contact with other metal restorations, galvanic effects may occur. In most cases they will be of short lasting duration. If the effect persists, the user should consider placing the amalgam in contact with restorations of other materials.
• There are no proven adverse effects on the fetus associated with the placement or presence of dental amalgam fillings in the mouth. It is sensible however, where clinically feasible, to minimise health interventions during pregnancy and avoid any unnecessary chemical exposure to the fetus. This precaution should be observed with the use of all dental restorations.
• In individual cases, local mucosal reactions (lichenoid) have been observed. Such local reactions may be of an irritative (mechanical, chemical, electrochemical) or allergic nature.
• In case of allergy to components of dental amalgam the use of suitable alternative materials must be considered.
• Unnecessary exposure to mercury vapour or dental amalgam particles during handling, placement or removal of dental amalgam should be avoided. Placement and removal of dental amalgam fillings should be performed with appropriate water spray and vacuum suction.
• Instructions and regulations for storage and disposal of dental amalgam waste must be followed.

Directions for use
Mixing
Cavex Avalloy is available as powder and as pre-dosed capsules. It should be noted, that the working and handling time would be somewhat different, dependent on the type of mixer used: a high energy mixer (such as used for pellets and pre-dosed capsules) gives a faster handling.
The mixed amalgam should be a plastic, coherent mass with a shiny surface.

The recommended mixing proportion for **Cavex Avalloy** is: **10 parts of alloy / 10.3 parts of mercury by weight**

Mixing time in a:		Dentomat/Duomat
High-energy mixer (such as the Silamat)		30 – 35 sec
Cavex Avalloy powder average dosage setting (for each size of amalgam mix) – maximum speed		30 – 35 sec
Cavex Avalloy pre-dosed capsules - I-spill (alloy/mercury 400/420 mg) - II-spill (alloy/mercury 600/630 mg) - III-spill (alloy/mercury 800/840 mg)	In all cases: 7 – 8 sec	


Note:
1. For safety reasons it is best to reclose the empty capsule after taking out the mixed amalgam, before cleaning or disposal.
2. The pre-dosed capsules (Type 1) are self-activating; contact between alloy and mercury occurs automatically when the capsule is subjected to the action of the mixing machine. Length: 32 mm, max. exterior diameter: 14 mm.
3. Storage at temperatures no higher than 25°C is recommended.
4. Dispose of cotton wool and used capsules in accordance with national regulations.

The times indicated are guidelines and will depend on the condition of the mixer.
Insertion and condensation
After mixing, the amalgam should immediately be inserted into the cavity using an amalgam carrier. The filling should be built up in layers under adequate condensation pressure using a plunger appropriate for the size of the cavity. Do not use an ultrasonic condenser. Carefully remove excess mercury.

Finishing and polishing
Approx. 6-10 minutes after mixing, the filling can be finished. Polishing should be done in a subsequent session.

“THIS ALLOY CONTAINS ZINC; THE AMALGAM MADE THEREFROM MAY SHOW EXCESSIVE EXPANSION IF MOISTURE IS INTRODUCED DURING MIXING OR COMPACTING”

Labelling
This information is in accordance with European legislation.

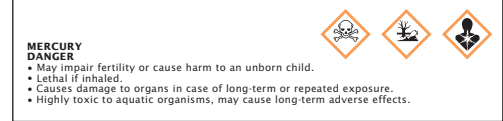RECOMMENDATIONS FOR SAFETY PRECAUTIONS
Dental clinical personnel, when using silver amalgam, should exercise proper procedures to avoid personal contact with mercury. Precautions should be taken to avoid exposure to mercury vapour in the dental environment. Waste amalgam should be collected for disposal in a manner, which will protect both those who handle the waste and the environment.

1. Treat mercury and amalgam with care. BE MERCURY-AWARE.
2. Ideally a high energy mixer should be equipped with a protective cap to cover the capsule whilst mixing.
3. Small droplets of spilled mercury can be collected with the aid of a mercury-collecting device or a small amount of freshly mixed amalgam that will easily absorb liquid mercury. NEVER use a vacuum cleaner!
4. Placement and removal of amalgam fillings must be done under water-cooling and with effective vacuum suction. The use of eye protection and mouth mask is advisable.
5. The wastewater drain of the spittoon should be equipped with an amalgam separator.
Local regulations on type and installation of such a separator and handling/recycling of the amalgam waste should be taken into consideration.
6. Excess mixed amalgam should be kept in a container, designed for the purpose, or at least in a closed container filled with photographic fixer.

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Our technical advice, whether verbal, in writing or by way of trials, is given in good faith but without warranty, and this also applies where proprietary rights of third parties are involved. It does not release you from the obligation to test the products supplied by us as to their suitability for the intended process and use.
The application, use and processing of the products are beyond our control and, therefore, entirely your own responsibility. Should, in spite of this liability be established for any damage, it will be limited to the value of the goods delivered by us and used by you. We will, of course, provide consistent quality of our products within the scope of our General Conditions of Sale and Delivery.

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Our technical advice, whether given orally, in writing or based on experience, is given in good faith but without any guarantee. This also applies when third party property rights are involved. We do not release you from your obligation to test the products we supply for their suitability for the intended treatment and indications.
Application, use and processing of the products is outside our control and, therefore, we are not responsible for any damage, this will be limited to the value of the products delivered to and used by you. Of course, we ensure a consistent quality of our products within the scope of our sales and delivery conditions.

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