# BEAUTIFIL Flow

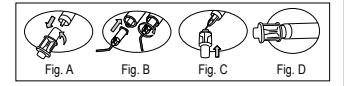
Fig. A Fig. B Fig. C Fig. D

READ CAREFULLY BEFORE USE

## Fluoride Releasing Flowable Restorative Material

BEAUTIFIL Flow is a light-cured radiopaque restorative material. Two different flowabilities, F02 and F10, are available. F02 shows lower flowability, while F10 shows higher flowability.

### SHADES

F02: A1, A2, A3, A3.5, A4, A30, G (Gum shade)

F10: A1, A2, A3, A3.5, A4, A30, A3T (A3 Translucency)

### COMPOSITION

Bis-GMA, TEGDMA, S-PRG filler based on fluoroboroaluminosilicate glass, Polymerization initiator, Pigments and others

### INDICATIONS

- > Anterior restorations
- > Base/Liner
- > Small, non occlusal stress-bearing restorations in posterior teeth

### PRECAUTIONS

- 1. Do not use this product on patients with known allergies to this material and/or methacrylate monomer.
- 2. Operators with known allergies to this material and/or methacrylate monomer must not use this product.
- 3. If any inflammation or other allergic reactions occur on either the patient or operator, immediately discontinue use and seek medical advice.
- 4. Use medical (dental) gloves and protective glasses to avoid sensitization to this product. In case of accidental contact with oral soft tissue or skin, immediately blot with alcohol moistened cotton ball, and rinse with plenty of water. Medical (dental) gloves provide some protection against direct contact with this material; however some monomers can penetrate through the gloves very quickly to reach the skin. Therefore, should this material contact even the gloves, immediately remove them and wash hands with plenty of water.
- 5. In case of contact with eyes, immediately flush the eyes with plenty of water and seek medical advice.
- 6. This product is intended for use by dental professionals only.

### DIRECTIONS FOR USE

- A-1. Cleaning of tooth surface  
Thoroughly clean the tooth surface to remove plaque.
- A-2. Shade selection  
Select appropriate shade using a shade guide while the tooth is still moist.
- A-3. Cavity preparation  
Prepare the cavity following the conventional method.
- A-4. Isolation  
Isolate with a rubber dam.
- A-5. Pulp protection  
In the case where the affected area is close to the pulp, cap the pulp with calcium hydroxide etc.

> Do not use eugenol containing materials since they might inhibit polymerization or adhesion of the material.

**A-6. Conditioning and application of adhesive**

- Apply dentin adhesive following the manufacturer's instructions for use.
- A-7. Setting of a Needle Tip**
- Follow the instructions below to attach the BEAUTIFIL Needle Tip provided.
- 1. Pull off the shield cap while turning it slowly. (Fig.A)
- 2. Attach the needle tip by turning it clockwise until it is securely attached. (Fig.B)
- > Wear clean medical (dental gloves) when replacing the needle tip.

**A-8. Placement**

- Apply necessary amount of BEAUTIFIL Flow directly into the cavity and create the shape desired.
- BEAUTIFIL Flow may also be extruded onto a paper pad to be applied into the cavity with a suitable instrument.
- > When storing these materials in a refrigerator, allow them to reach room temperature before use.
- > The paste dispensed onto the paper pad should be shielded from ambient light by light-shielding cover to avoid unintended polymerization of the material.
- > Needle tips should be used for only one patient and then discarded at the end of the procedure to avoid cross contamination.

**A-9. Light-curing**

- Light-cure using a dental light-curing unit.
- > When the cavity is deep, incremental filling and light-curing not exceeding 2 mm is recommended.
- > When the cavity surface is large, light-cure in segments.

**Dental light-curing unit and light-curing time**

Light-curing unit	Halogen	LED
Light-curing time	30 sec.	10 sec.

**Manufacturer**  
SHOFU INC.  
11 Kamitamanshi-cho, Fukukawa,  
Higashimurayama-ku, Kyoto 603-0983, Japan  
www.shofu.com

**SHOFU DENTAL CORPORATION**  
1225 Stone Drive San Marcos,  
California 92078-4059, USA

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- B-2. Base/Liner**
- Apply necessary amount of BEAUTIFIL Flow directly into the cavity and create the shape desired. BEAUTIFIL Flow may also be extruded onto a paper pad to be applied into the cavity with a suitable instrument. Light-cure the material using a dental light-curing unit (Halogen: 30 sec. or LED: 10 sec.).

- > When storing these materials in a refrigerator, allow them to reach room temperature before use.

- > The paste dispensed onto the paper pad should be shielded from ambient light by light-shielding cover to avoid unintended polymerization of the material.

- > Needle tips should be used for only one patient and then discarded at the end of the procedure to avoid cross contamination.

- A-10. Finishing and polishing**
- After light-curing, contour, finish and polish the restoration following the conventional method.

- B-3. Placement**
- Fill direct composite resin.

- B-4. Finishing and polishing**
- After light-curing, contour, finish and polish the restoration following the conventional method.

## NOTE ON USE

- 1. Follow the instructions for use of each dental material or instrument to be used in conjunction with this product.

- 2. Do not use this product for any purposes other than specifically outlined in the INDICATIONS in these instructions for use.
- 3. Use this product within the expiration date indicated on the package and container.  
(Example <span>YYYY-MM-DD </span> → Year-Month-Date of the expiration date)

## STORAGE

- 1. Store at room temperature (1-30 °C / 34-86 °F). Avoid high temperatures and high humidity. Keep away from direct sunlight and any source of ignition.

- 2. Discard the used Needle Tip and store this product in either way of the following.

- 2-1. Attach new one. Insert the needle into the small hole on the head of the shield cap. (Fig.C)

- 2-2. Replace the shield cap securely. (Fig.D)

## CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.

Location of the symbols glossary:

[https://www.shofu.com/en/symbols\\_glossary/](https://www.shofu.com/en/symbols_glossary/)

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