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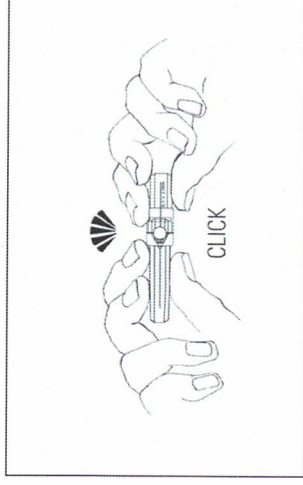
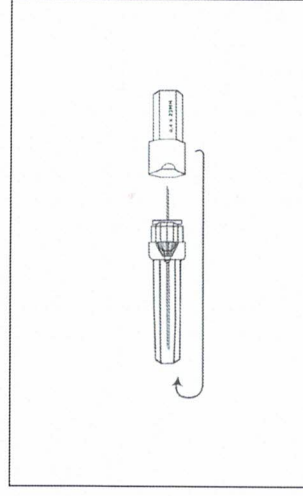
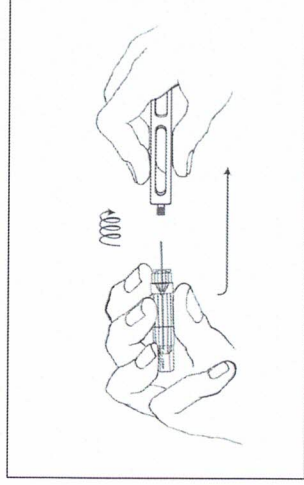
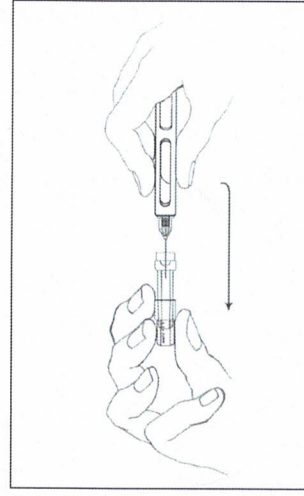
Stand der Information: Januar 2017

CE 0482

CALASEPT-SPEIKO® with x-ray contrast

Composition:

Contains calcium hydroxide and barium sulphate in watery suspension, calcium chloride, potassium chloride, sodium chloride, natrium hydrogen carbonate, purified water.

Fields of application:

- Supply of caries-free, near-pulp dentin
- Direct pulp capping, also following a pulpotomy
- Temporary insertion in root canal
- Caries profunda treatment and/or indirect pulp capping
- Only for dental use

Contra-indications:

- In case of pulp exposure in carious dentin
- In case of inflammatory changes of pulp
- In case of hypersensitivity to ingredients

Side effects/interactions:

None known if used according to intended purpose.

Dosage, type and duration of treatment:

Pulp capping (direct and indirect):

Small amounts of the paste are placed on a mixing block with the aid of the enclosed Calasept cannula (curved) and from there applied with an instrument to the opened/exposed pulp or the near-pulp dentin. Following the application, the past should be compressed with water-absorbing material (cotton pellets or paper tips). Subsequently, it is covered tightly with a lining material.

Temporary insertion in root canal:

Clean root canal with Hypochlorite-SPEIKO® 3% or 5.25% and dry with sterile paper tips. As a temporary filling for root canals, CALASEPT-SPEIKO® can be inserted directly into the prepared canal with the aid of e.g. applicator. The application of the paste into the root canal is also possible with the aid of the enclosed disposable cannula. If necessary, the treatment can be repeated. CALASEPT-SPEIKO® should be removed prior to the definite root filling.

Caries profunda treatment:

Clean and dry the cavity prior to applying CALASEPT-SPEIKO® in a thin layer. If a thicker layer is required, please apply in layers. Briefly dry each layer with a gently air current. Subsequently, place lining and filling.

Notes:

Usage of correct cannulas: In order to prevent curing of the material, the disposable cannula has to be placed into the enclosed rubber stop and the bent cannula closed with the draw wire.

Usage of correct cartridge syringe: We recommend the usage of a non-self-aspiring cartridge syringe. The syringe used for CALASEPT-SPEIKO® has to have a well-centred piston which is not laterally movable. Syringes with aspiration hooks or V-shaped pistons cannot be used.

CALASEPT-SPEIKO® may not be used after the expiration date.

Storage at room temperature.

Trading format:

CALASEPT-SPEIKO® Content: 2 x 2,5 g Art. no. 1005
Accessories: 3 disposable cannulas and 1 bent cannula

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