FR

COMPOSITION

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EN

COMPOSITION

Mint: Glycerol, colouring agents (patent blue V, tartrazine), mint flavour, preservative, excipients
 Raspberry: Glycerol, colouring agent (cochineal red A), raspberry flavour, preservative, excipients
 Orange: Glycerol, colouring agent (yellow orange S), orange flavour, preservative, excipients

PROPERTIES

Mouth washing with CALYPSO:

- prepares the operative field,
- flushes out debris and blood during the procedure,
- makes the patient's mouth feel fresh between two steps of the procedure.

Thanks to the three flavours (mint, orange, raspberry), the dentist can offer patients the mouthwash of their choice.

Sold in a concentrated form, CALYPSO immediately dissolves in water, which makes it very practical to use.

CONTRA INDICATIONS

Hypersensitivity to any component.

INSTRUCTIONS FOR USE

Using the pump, put one dose of CALYPSO into a glass. Fill it with water.

Do not swallow.

Remark: If the spittoon is scaled or worn, the pigments contained in the solution may adhere and tinge it.

WARNINGS AND PRECAUTIONS FOR USE

Green mouthwash contains mint flavour ((S)-p-mentha-1,8-diene).
 May produce an allergic reaction.

PRESENTATION

Pack of 3 x 500-ml bottles (mint, orange, raspberry or assorted flavours) + 3 dosing pumps.

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