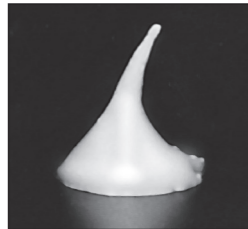
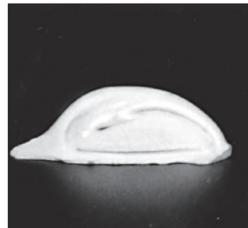
1

2

3

-  **REF** 80312-[Colour-No.]*
-  **REF** 8031300
-  **Set** **REF** 80315-[Colour-No.]*

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Instruction for use for the dentist

Composition:

Powder contains: zinc oxide, magnesium oxide
Liquid contains: polyacrylic acid

Areas of Application:

Cementation of crowns, bridges and inlays,
cavity linings.

Dispensing Instructions:

Cavity lining consistency:
3,6 g powder : 1.0 g liquid
Cementation consistency:
2,9 g powder : 1.0 g liquid
Dispensing and testing climate for testing in accordance with DIN EN ISO 9917:

Mix quantities indicated at 23 ± 1°C and a relative humidity of 50 ± 10%.

The dispensing instructions also apply when mixing the liquid with HOFFMANN'S UNIVERSAL CEMENT.

Processing Recommendations:

A clean, dry glass plate at ca. 20°C (68 °F) and a non-discolouring spatula are recommended for mixing.

Mixing:

Divide the measured quantity of powder into 4 portions (1/2 1/4 1/8 1/8). Beginning with the largest quantity, stroke the powder into the measured liquid within 60 seconds. The mixture should appear thicker as usual for zinc phosphate cement, yet it is just as easy to work.

The cavity lining consistency has been reached when on lifting the spatula, the peak drawn out forms a small hook and does not sink back into the paste (Fig. 1).

The cementation consistency has been reached when the peak formed on lifting the spatula slowly falls back into the paste (Fig. 2 and 3). Discard remaining powder or impure liquids.

Mixing time: 60 seconds

Processing Time from beginning of mixing (at 23 °C, 72 °F):

For cementation: 2 minutes
For cavity linings: 1:40 minute

Setting Time from end of mixing:

For cementation: 3 – 4 minutes
For cavity linings: 2 – 3 minutes

Special Information:

- The mixed cement should never be applied directly onto dentine near to pulp or onto exposed pulp.
- The cement must be applied to the dry tooth surface and must set without contact to moisture.
- Do not expose liquid to sunlight and frost.
- Close containers immediately after removing material.
- Clean instruments and glass plate with water before the cement sets. Zinc poly carboxylate cement can be dissolved using a cement-solvent after setting.

Package Sizes:

Powder: 100 g, Liquid: 40 ml
Set: 35 g powder and 15 ml liquid

HOFFMANN'S CARBOXYLATE CEMENT

- Do not use after the expiration date
- Compliant with DIN EN ISO 9917
- Material safety data sheet (MSDS) available at www.hoffmann-dental.com

*** 15 Colours/Shades:**

- 01 (white), -02 (bluish white), -03 (yellowish white), -04 (light yellow), -05 (yellow), -06 (gold yellow), -07 (gold brown), -08 (pearl grey), -09 (grey), -10 (greenish grey), -11 (bluish grey), -12 (brown), -13 (greyish brown), -14 (light pink), -15 (pink)

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