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User instructions

Dear user,

KaVo hopes that you enjoy your new high-quality product. Following the instructions below will allow you to work smoothly, economically and safely.

Symbols

	See the section Safety / Warning Symbols
	Important information for users and technicians
	CE mark (European Community). A product bearing this mark meets the requirements of the applicable EC directive/regulations.
	Action request

User group

The instructions for use are for dentists and dental assistants, as well as hygiene personnel in the dental practice.

Intended use

KaVo CLEANspray: Cleaning spray for validated internal cleaning of KaVo straight and contra-angle handpieces and KaVo turbine handpieces.

KaVo CLEANspray is free from alcohol (non protein-fixing) and is designed for manual internal cleaning of handpieces/ contra-angle handpieces and turbine handpieces (incl. spray channels). KaVo CLEANspray is a medical device according to applicable national statutory regulations. The suitability of the cleaning was demonstrated based on the ISO/TS 15883-5 standard and the requirements of DGVS – Deutsche Gesellschaft für Sterilgutversorgung – and is confirmed in the form of an expert report (www.kavo.com). This verification is also applicable to instruments and turbine handpieces of earlier series due to the similarity in design.

Do not reuse the solution rinsed out.

KaVo DRYspray: Medically pure drying spray in support of cleaning after cleaning and/or disinfection of internal surfaces of medical or dental hollow-bodied instruments (turbine handpieces, straight and contra-angle handpieces, internally cooled instruments and endoscopes).

KaVo DRYspray is used right after the use of KaVo CLEANSpray for drying of the air, water and gear unit ducts.

Disposal of the products

Residues of the product must be subjected to disposal as non-hazardous waste in compliance with Waste Directive 2008/98/EC and national and regional regulations. Do not mix with other waste.

Disposal of the packaging

The compressed gas packaging must be emptied optimally and can then be recycled.

Reporting of events

Please report all serious events associated with the product to:

- KaVo Dental GmbH
Phone +49 7351 56-1500
Email: service.instrumente@kavokerr.com
- ALPRO MEDICAL GMBH
Email: quality@alpro-medical.de
Phone: +49 7725 9392-0
- and the competent authority of the Member State

Product description

Order dates

REF	Designation	Consisting of
1.007.0573	CLEANSpray- / DRYspray Starterset 2116 P	1 can of KaVo CLEANspray 1 can of KaVo DRYspray 1 adapter set INTRA + MULTIflex
1.007.0579	Pack of KaVo CLEANspray 2110 P	4 cans of KaVo CLEANspray
1.007.0580	Pack of KaVo DRYspray 2117 P	4 cans of KaVo DRYspray
1.007.1776	Spray attachment INTRA	
1.007.1775	Spray attachment for KaVo MULTIflex turbines	

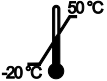

NOTE

Please note that these spray attachments are designed to be reused with additional CLEANspray/DRYspray cans and are not to be disposed with the empty cans.

Transportation and storage conditions

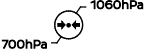Observe the instructions for storage and handling on the can and packaging.

	Content
	Storage and transportation temperature -20 °C–+50 °C / -4 °F–+122 °F
	Humidity 5 %–85 %
	CE mark
	Expiration date YYYY-MM
	Production batch xxxxxx
	Please note the instructions for use
	EEC sign for pre-packed products
	Symbol indicating compliance with Directive 75/324/EEC for aerosol dispensers (inverted epsilon)

	Total capacity (in mL) is the total volume of the open container (up to the brim) of the compressed gas dispenser
	Safety-related information
	Protect product from exposure to sunlight
	Air pressure restrictor
	Store in a dry location
	Material number
	Manufacturer
	Formula Identifier (Unique Formula Identifier)

The products may only be used at room temperature (18 °C–25 °C / 64 °F–77 °F).


Operation

Use KaVo CLEANspray

- ▶ Place the appropriate spray attachment on the can.
- ▶ Place the instrument on the spray attachment.
- ▶ Operate the spray with 3 spray jets during 2 seconds each in the instrument.
- ▶ Remove the instrument from the spray attachment.
- ▶ Lay the instrument down, and let the cleaner work for 1 minute. We recommend an exposure time of 8–10 minutes for first use.

Use KaVo DRYspray

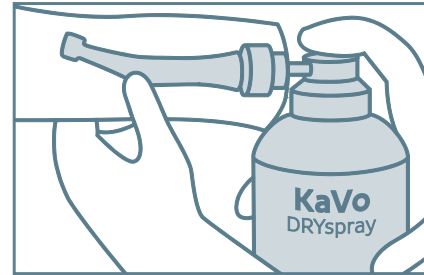- ▶ Place the appropriate spray attachment on the can.
- ▶ Place the instrument on the spray attachment.
- ▶ Apply spray into the instrument for at least 3 seconds.
- ▶ Remove the instrument from the spray attachment.
- ▶ Spray the inside again according to need.

**NOTE**

Keep bottle upright during use.

**NOTE**

Following this internal cleaning, perform the further reprocessing steps according to the applicable recommendations and guidelines.

If the instruments and turbines are disinfected in the thermodisinfectior, they must be treated afterwards with products from the KaVo care system (KaVo Spray/ROTAspray/QUATTROcareplus spray). If the instruments are sterilised in an autoclave, they must be cared for beforehand. Observe the care instructions in the instructions for use of the corresponding instruments and turbines.

KaVo CLEANspray:

Hazard Statements + Precautionary statements:

Warning: H229: Pressurised container: May burst if heated.
P210: Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. P251: Do not pierce or burn, even after use.
P410+P412: Protect from sunlight. Do not expose to temperatures exceeding 50 °C / 122 °F.

KaVo DRYspray:

Hazard Statements + Precautionary statements:

Warning: H229: Pressurised container: May burst if heated.
P210: Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. P251: Do not pierce or burn, even after use.
P410+P412: Protect from sunlight. Do not expose to temperatures exceeding 50 °C / 122 °F.

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