## Compo Blitz Polierer

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## Compo Blitz Polierer

### GB Instructions for use

**Field of application:** Rotating instrument for polishing composite, ceramic and metallic fillings as well as denture-material. To be used on contra angles.

**Use:** For optimum output, adhere to the recommended oscillation speed of 1,000 – 3,000 min<sup>-1</sup> (rpm). Disregarding the maximum speed increases the safety risk by increasing temperatures on the polished surfaces and by raising release of polish agent. Increased temperature may result in pulp damage. Intermittent or dabbing practice method with gentle contact pressure does not require water cooling.

**Material:** Felt with diamond 3-5 µm and peppermint taste on plastic mandrel.  
Felt with diamond 3-5 µm and peppermint taste on steel mandrel.

**Risks:** For single-use only. The products' functionality can no longer be guaranteed if used more than once. Furthermore, a re-use leads to a high risk of infection. Allergic reactions of patients are not known. Continuous use of more than 15 sec. is very likely to warm up the polished surface (pulp damage).

**Disinfection:** The instruments are not sterilisable! Since the Compo Blitz Polishers are for single-use only, we recommend treating them with a disinfectant liquid only before use.

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