Diamond Polish

1.0µm & 0.5µm Polishing Paste

EN

Description & Indications: Ultradent® Diamond Polish is a white, mint-flavored paste available in two grits: 1.0µm and 0.5µm. Both contain diamond particles in a water base. Diamond Polish is used to obtain unsurpassed polish on porcelain or composite restorations. It may be used intraorally or extraorally for achieving high luster.

Procedure:

- For polishing porcelain
1. Contour and finish all surfaces of the restoration with Jiffy® points, cup and discs (Fig. 1).
- Remove Luer Lock cap from syringe. Twist a disposable Mini tip securely onto syringe.
- For optimum control, hold syringe in palm of hand (Fig. 2) rather than in conventional manner with thumb on plunger.
- Apply a small amount of diamond polish with a felt wheel. For final polishing, increase speed of hand piece and use a heavy spray of water while polishing.
- Use a firm air water spray for final rinse.

II. For polishing composite

- Contour and finish all surfaces of the restoration with Jiffy® cups, points, discs, wheels and abrasive strips.
- Apply Diamond Polish with a Jiffy Goat Hair Brush (Fig. 3).
- Rinse and dry surface(s).

Precautions:

- We recommend using Ultradent® Syringe Covers sealed with the Ultradent® Impulse Sealer for a protective barrier to prevent cross-contamination.
- Tips are disposable.

GENERAL PRECAUTIONS

- For professional use only.
- Review instructions, precautions, and SDS before beginning treatment. Use only as directed.
- Keep products out of heat/sunlight.
- Avoid skin exposure to resins.
- Isolate strong chemicals to area of treatment.
- Confirm that patient has no known allergies to treatment materials.
- Test flow of materials from syringe and tip before using intraorally.
- Never force syringe plungers.

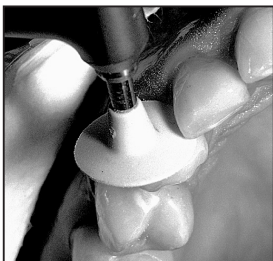
Fig. 1 Use fine stones, diamond burs, Jiffy® cups, points, discs and/or finishing strips to contour and finish the restoration.

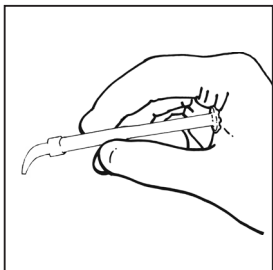
Fig. 2 Hold syringe in palm of hand for optimum control.

Fig. 3 To prevent premature loss of polishing material, use slow revolutions with a low speed handpiece without water.

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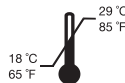
Store at room temperature.

ENGLISH

Provide maximum polish for porcelain or composite. Use with a felt or rag wheel.

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