DE	Gebrauchsanweisung	5
GB	Instructions for use	13
FR	Mode d'emploi	21
ES	Instrucciones de uso.....	29
IT	Istruzioni per l'uso	37
PT	Instruções de uso	45
NL	Gebruiksaanwijzing	53
SE	Bruksanvisning	61
DK	Brugervejledning	69
NO	Bruksanvisning	77
FI	Käyttöohjeet.....	85
GR	Οδηγίες χρήσης.....	93
CZ	Návod k použití.....	101
HU	Használati utasítás.....	109
LV	Lietošanas instrukcija	117
LT	Naudojimo instrukcija	125
PL	Instrukcja obsługi.....	133

A

B

C

D

E

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Stand: 2021-02

Description of applicational/packaging icons

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Instructions for use

Elastomeric, silicone-based precision impression material, addition-cured

The Flexitime range is an optimally matched system and its different types of material can be combined for all impression techniques to suit specific requirements.

Material

Flexitime Easy Putty

Flexitime Dynamix Putty

Flexitime Heavy Tray

Flexitime Dynamix Heavy Tray

Flexitime Monophase

Flexitime Dynamix Monophase

Flexitime Medium Flow

Flexitime Correct Flow

Flexitime Light Flow

Pack

Tub, 1:1, 2 x 300 ml

Cartridge, 5:1, 380 ml

Cartridge, 1:1, 50 ml

Cartridge, 5:1, 380 ml

Cartridge, 1:1, 50 ml

Cartridge, 5:1, 380 ml

Cartridge, 1:1, 50 ml

Cartridge, 1:1, 50 ml

Cartridge, 1:1, 50 ml

Side effects:

This product or one of its components may in particular cases cause hypersensitive reactions. If suspected, information on the ingredients can be obtained from the manufacturer.

Contraindications

The use of this product is contraindicated in case of known or suspected allergies against components of this product.

Notes on use

Not to be used in conjunction with other silicone materials.

Close tubs properly after use. Do not inter-change caps/lids/dosing spoons. Dispense a small amount of material from a 50 ml cartridge before attaching a mixing tip. After use, leave the mixing tip on the cartridge to keep it closed. Immediately before the next use, remove the mixing tip and attach a new one. Mixing Tips and Intraoral Tips are single-use products. To prevent cross-contamination between patients, Mixing Tips and Intraoral Tips must be discarded after each use.

In all addition-curing silicone impression materials, the setting process may be affected by contact with products containing active Sulfur or Nitrogen, or by fresh composite fillings or superstructures. These products include latex gloves, Polysulfide impression materials, retraction solutions, etc. Flexitime contains surfactants for hydrophilation, which in some patients may cause a superficial anesthetic effect.

Surplus material on the impression tray or in the patient's mouth should be avoided as this may result in gagging, swallowing or aspiration. To further reduce these risks during impression taking, the patient should be positioned as upright as possible. Do not leave any residual material in the sulcus or oral cavity, as this can lead to abscess formation.

Flexitime	EN ISO 4823	Product colour		Mixing ratio	Mixing time	Extraoral Working time* (23°C (73°F))	Intraoral Working time (35°C (95°F))	Min. time in mouth	Accessories	
		Base	Catalyst						Mixing Tips	Intraoral Tips
Flexitime Easy Putty	Type 0 putty	violet	light grey	1:1	30	2:30	–	2:30	n.a.	n.a.
Flexitime Dynamix Putty	Type 0 putty	violet	light grey	5:1	auto	2:30**	–	2:30	dynamic yellow 66035999	n.a.
Flexitime Heavy Tray	Type 1 heavy bodied	violet	light grey	1:1	auto	2:30	–	2:30	green 66001989	n.a.
Flexitime Dynamix Heavy Tray	Type 1 heavy bodied	violet	light grey	5:1	auto	2:30**	–	2:30	dynamic yellow 66035999	n.a.
Flexitime Monophase	Type 2 medium bodied	purple	light grey	1:1	auto	2:30	1:00	2:30	green 66001989	white 65687514
Flexitime Dynamix Monophase	Type 2 medium bodied	purple	light grey	5:1	auto	2:30**	1:00	2:30	dynamic yellow 66035999	n.a.
Flexitime Medium Flow	Type 2 medium bodied	light green	light grey	1:1	auto	2:30	1:00	2:30	yellow 66001988	yellow 66000782
Flexitime Correct Flow	Type 3 light bodied	light green	light grey	1:1	auto	2:30	1:00	2:30	yellow 66001988	yellow 66000782
Flexitime Light Flow	Type 3 light bodied	green	light grey	1:1	auto	2:30	1:00	2:30	yellow 66001988	yellow 66000782

* at room temperature of 23°C (73°F), 50% rel. humidity. Higher temperatures shorten, lower temperatures prolong these times.

** measured using the Dynamix® mixer from Kulzer.

Flexitime impression materials should be at room temperature 23°C (73°F) for processing. Visually inspect for damage before first use. Damaged products must not be used.

If there are substantial undercuts or highly divergent teeth in the impression area, appropriate blocking out must be performed if using Flexitime Heavy Tray or Putty.

Optical scanning:

The quality of the scanned data depends on the impression scanner. Follow the manufacturer's recommendations for preparation of teeth and the instructions for the optical scanning procedure.

Model fabrication:

The impression can be cast 30 minutes after removal from the mouth – there are no other time constraints. The impression is stable in long-term storage.

Recommended model material:

All standard model materials can be used.

We recommend:

Class III plaster, MOLDANO®

Class IV plaster, MOLDASTONE®/MOLDASYNT®

Disinfection:

Impressions can be treated with an aqueous sodium hypochlorite solution (5.25%, for up to 10 minutes*) or with other disinfectant solutions suitable for silicones. When doing so, please observe the disinfection/processing instructions issued by the relevant manufacturers.

*Gelson Luis Adabo, DDS, PhD, Elaine Zanarotti, Renata Garcia Fonseca, and Carlos Alberto dos Santos Cruz, DDS, PhD
Effect of disinfectant agents on dimensional stability of elastomeric impression materials. *J Prosthet Dent*, 81, 621-4, 1999

Adhesion:

We recommend using our Universal Adhesive.

Storage conditions:

Do not use Flexitime impression materials if the expiry date has elapsed.

Do not store above 25°C (77°F).

Dynamix System, 380 ml – step by step

Fig. 1
Remove cap.
Insert cartridge
into the mixer

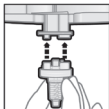
Fig. 2
Attach Dynamix
Mixing Tip

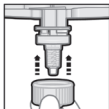
Fig. 3
Position Dynamix
fixation ring

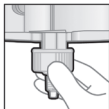
Fig. 4.1
Attach Dynamix
fixation ring.
Slide over
recesses

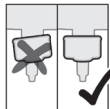
Fig. 4.2

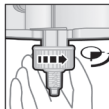
Fig. 5
Lock by giving
fixation ring
a ¼ turn
clockwise

Inserting the Dynamix cartridge:

For information on how to handle the mixer being used please refer to the detailed instructions issued by the relevant manufacturer.

1. Hold the tab on the cap of the outlets and bend it upwards through 90° without applying any excessive force.
2. Take the tab between your thumb and index finger whilst you hold it securely with your other hand and remove the cap from the cartridge completely (Fig. 1).
3. Place one of the enclosed Dynamix Mixing Tips over the outlets of the cartridge (Fig. 2).
4. If it proves difficult to attach the mixing tip, check to make sure that the central internal hexagon of the mixing tip is properly aligned with the hexagon of the drive spindle.
5. When the mixing tip is in the correct position (Fig. 3), push the Dynamix fixation ring over the mixing tip until it stops and turn it clockwise (1/4 turn) until it is securely locked in place (Figs. 4.1, 4.2 and 5).
6. When impression material has been mixed, transfer it to a separate intraoral impression syringe, e.g. Impression Jet (Flexitime Dynamix Monophase only) or a suitable impression tray.
7. When the material has been dispensed, retract the mixer plungers from the cartridge in order to avoid continuous pressure and thus prevent excess material from continuing to emerge from the mixing tip.

Notes about safe use of the Dynamix System

Dynamix cartridges are heavy and can be damaged if they are dropped.

Damaged cartridges must no longer be used.

Only use Dynamix cartridges in mixers with a closed protective flap.

Before using for the first time, adjust the levels of the two material components. Dispense approx. 2–3 cm material and discard this amount before starting to fill the tray. When the cartridge is used in different mixers, 2–3 cm of material has to be discarded after each device change. Each time before you use the material check the outlet channels of the cartridge for blockages.

At the end of a working day or if the mixer is not going to be used for a lengthy period, remove the cartridge from the device and store it in a horizontal position with the outlets closed off by the mixing tip or cap.

After use, leave the mixing tip on the cartridge to keep it closed.

Immediately before the next use, remove the mixing tip and attach a new one.

To do so, detach the Dynamix fixation ring and use it again for the new mixer attached.

Only use Dynamix System components for dispensing and mixing the impression material in Dynamix cartridges.

Check the system components for physical damage on a regular basis and only use components which are intact.

For disposal please follow the instructions on the material safety data sheet and national regulations.

Warnings and safety instructions

Polyvinyl siloxanes are chemically resistant. Material which has not set can soil clothing. **Wear protective gloves according to EN 420, category III, with protection against chemical risks according to EN 374, type C.** Avoid contact with skin and eyes. May be irritating to eyes and skin. On contact with skin wash off with water and soap. If material comes into contact with the eyes, rinse it out with water for several minutes. If symptoms persist, consult an eye specialist.

Do not swallow and ingest. If health problems arise after swallowing impression material, seek medical attention immediately. Intestinal blockage may arise in rare cases.

Only to be used by dentists and for its intended use.

According to the EU Medical Devices Regulation, users / patients are obliged to report serious events with a medical device to the manufacturer and to the competent authority of the country in which they occurred. Please quote batch number and the article number in all correspondence about the product.

Dated: 2021-02

Description of applicational / packaging icons

A = Only use pastes with same LOT No

B = Icon "Use of the special program at the Dynamix® *speed* Mixing Unit possible"

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Kulzer GmbH
Kulzer, LLC