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Deutsch - FFX3XXX-04/21

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FANTESTIC® FILL

CE 0482

Hersteller:

R-dental Dentalerzeugnisse GmbH • Winterhuder Weg 88 • 22085 Hamburg
T 040-30707073-0 • F 040-30707073-73 • I www.r-dental.com

FANTESTIC® FILL

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REF FFT34XX (in Tips, je nach Farbe)

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FANTESTIC® FILL

REF FFS33XX (in syringes, depends on shade)

REF FFT34XX (in tips, depends on shade)

Description

FANTESTIC® FILL is a light-cure, radiopaque microhybrid composite for anterior and posterior restorations.

Characteristics and composition

FANTESTIC® FILL is based on BIS-GMA-resin and inorganic filler particles of a size of 0.05-1.5 µm. The total filler load is 81 wt.-% and 65 vol.-%.

The radiopacity of FANTESTIC® FILL is 200 % Aluminium (radiopacity enamel ≈ 200 % Aluminium, dentin ≈ 100 % Aluminium).

FANTESTIC® FILL-products are available in VITA®-shades.

The material properties of FANTESTIC® FILL meet the requirements of ISO 4049, type 1, class 2, group 1.

Indications

FANTESTIC® FILL is indicated

• for restorations of all cavity classes.

Side effects

Irritations resulting from direct contact with the pulp are possible. Therefore for pulp protection areas close to the pulp must be covered with a thin layer of calcium hydroxide or a bioactive pulp capping material, see section 4.

Incompatibilities

Do not use FANTESTIC® FILL in combination with eugenol containing materials. Eugenol impairs the polymerization of the composite. Do not store FANTESTIC® FILL in proximity of eugenol containing materials. Unpolymerized FANTESTIC® FILL should not come in contact with eugenol containing materials.

Recommended procedures

1. Preparation

Prior to the preparation a cleaning of the tooth with a fluoride-free paste (e. g. fluor of pumice with water) is recommended.

2. Shade matching

Ascertain the tooth shade while teeth are still moist and select the appropriate FANTESTIC® FILL shade (FANTESTIC® FILL-products are available in VITA®-shades).

3. Isolation

A dry area and use of rubber dam (REF KDL31XX, XX = depends on strength) or latex-free dam (REF KDL3500) is recommended.

4. Preparation of the cavity

The cavity will then be prepared in accordance with clinical rules and with minimal tooth reduction.

For pulp protection areas close to the pulp should be protected with a thin layer of calcium hydroxide, Cal de Luxe®, (Ca(OH)₂, "ready-to-use" in a portion blister, REF CDL2503) according to the instructions for use or with a pulp cap material, a bioactive pulp capping material, like ReViCal (REF RVC1002) according to the instructions for use.

Then apply the Total-Etch-Technique (Etch & Rinse) or the Self-Etch-Technique with common adhesives (like FANTESTIC® UniversalBond (REF FUB6000) as a self-etching one-component adhesive) for dentin and enamel according to the instructions for use.

Then FANTESTIC® FILL can be applied directly.

5. Application of the flowable composite FANTESTIC® FLOWFILL according to the instructions for use

FANTESTIC® FLOWFILL (REF FFS32XX, XX depends on the shade) can be used as a lining material for an optimal adaptation of the mouldable composite FANTESTIC® FILL within deep or extended cavities. At small cavities FANTESTIC® FILL can be applied directly.

6. Application of the composite FANTESTIC® FILL

FANTESTIC® FILL should then be applied in thin layers of max. 2 mm into the cavity. The use of a matrix and placing with a suitable instrument is recommended.

7. Light-cure

FANTESTIC® FLOWFILL and FANTESTIC® FILL can be light-cured with commercial polymerization units (halogen or LED) with a wave length of 400-500 nm and with a light intensity > 1000mW/cm².

Cure each layer of lighter shades (e. g. A1, A2, C2) **20 s** and of darker shades (e. g. A3,5, B3) **30 s**. Hold the light emission window as close as possible to the filling material.

8. Finishing and high gloss polishing

After removing the matrix FANTESTIC® FILL can be contoured with finishing diamonds, flexible grinding discs or carbide burs. Polish to high gloss with polishing discs or polishing brushes.

9. Storage

Do not store above 77 °F! Avoid exposure to sunlight. Do not use after expiration date. Close syringe immediately after use to avoid exposure to light. Composites in tips are for single use only.

Precautions to application, product life and/or storage

Avoid contact of the etching gel FANTESTIC® ETCH, calcium hydroxide Cal de Luxe® and the pulp capping material ReViCal with soft tissues, skin or eyes. If there is an accidental contact, flush immediately with copious amounts of water.

Pay attention to the Material Safety Data Sheets (MSDS >> www.r-dental.com).

The ambient light of the dental lamp may start polymerization of the composite.

Do not adjust the viscosity of the composite restorative material by using any resin.

The contact of FANTESTIC® FILL with skin should be avoided, especially by anyone having known resin allergies against methacrylates.

Commercial medical gloves do not protect against methacrylates.

Shelf life see label. Do not use after expiry date.

*VITA is not a registered trademark of R-dental Dentalerzeugnisse GmbH.

FANTESTIC and Cal de Luxe are registered trademarks of R-dental Dentalerzeugnisse GmbH.

Keep out of the reach of children!

Caution: For dental use only.

Warranty

R-dental Dentalerzeugnisse GmbH warrants this product will be free from defects in material and manufacture. R-dental Dentalerzeugnisse GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. The user is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and R-dental Dentalerzeugnisse GmbH's sole obligation shall be repair or replacement of the product.

Limitation of Liability

Except where prohibited by law, R-dental Dentalerzeugnisse GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

© 2021 R-dental Dentalerzeugnisse GmbH. All rights reserved.