
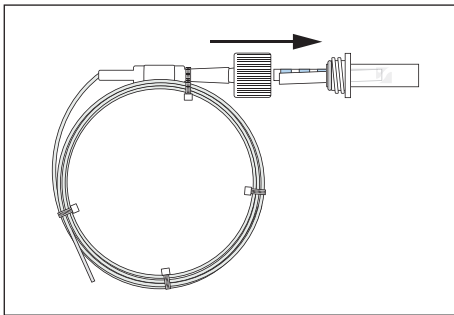
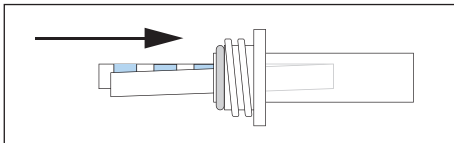
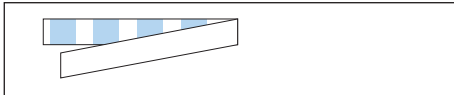
Hygotest

REF 6050-600-03 (134 °C - 3,5 min / 121 °C - 15 min)

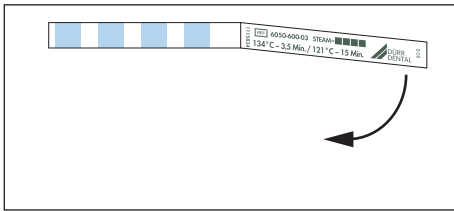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EN Operating Instructions


i Observe the operating instructions. Incorrect use can impact on the indicator colour change and may lead to incorrect results.

- › Open a Helix tube with a 3/4 twist.
- › Fold the indicator strip along the line so that the indicator is on the inside. The different lengths of the strip make it easier to take out the strip after use.
- › Place the indicator strips with the folded side first into the Helix tube.
- › Screw the sealing cap onto the Helix tube.
- › Place the Helix test body with inserted indicator strip in the sterilisation chamber and start the program.

For batch monitoring the Helix test body that has been prepared here can also be enclosed with the object(s) being sterilised.





Result:

After successful sterilisation the indicator will change colour from light to dark.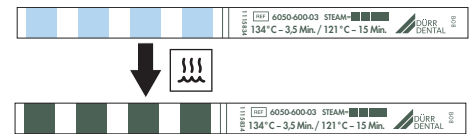

The text and company logo on the indicator strip show the reference colour. Minor colour deviations may occur.

The indicator has a self-adhesive strip on the rear so that it can easily be archived with other important data once the backing strip has been peeled off.

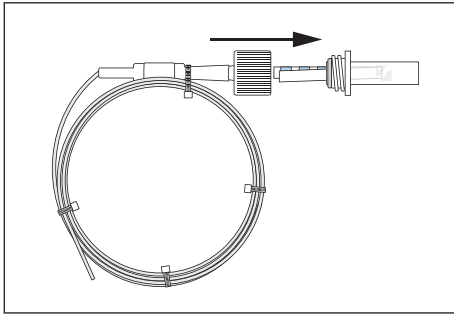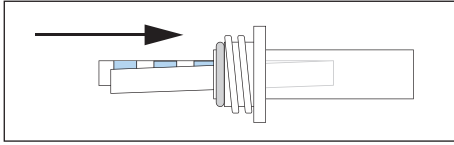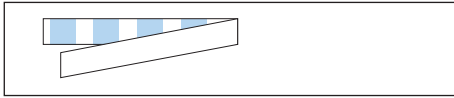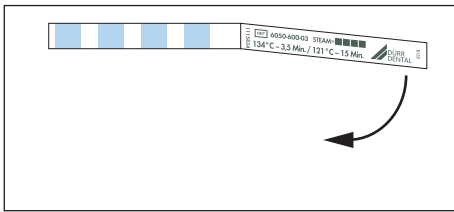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