INSTRUMENTS

GREEN&CLEAN IK

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METASYS

GREEN&CLEAN IK

DE Gefahr

EN Danger

FR Danger

IT Pericolo

ES Peligro

PT Perigo

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SV Fara

FI Vaara

NO Faretype

DA Farlig

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EN GREEN&CLEAN IK

GREEN&CLEAN IK is a ready-to-use disinfectant for the decontamination of spray water and spray air channels of handpieces, contra-angles and turbines. Fixed adaptors guarantee security and user friendliness.

Spectrum of efficacy/action times: limited virucidal (e.g. HBV/HCV/HIV, Herpes-simplex-Virus Type I): 15 seconds; bactericide, fungicide (C. albicans): 2 minutes; tuberculocide: 2 minutes.

Instructions for use GREEN&CLEAN IK (spray can): Rinse transmission instrument with water after the application on the patient. **1** Attach the handpiece, contra-angle or turbine to the adapter and **2** completely flush the spray water and spray air channels with GREEN&CLEAN IK (spray time: approx. 2 sec). GREEN&CLEAN IK is highly compatible with the materials used in the transmission instruments.

Security advice: The can is pressurized! Only use spray head when transmission instrument is attached correctly. Apply over a sink.

Note: **3** + **4** To lubricate the O-rings, place the GREEN&CLEAN IK adapter in the palm of your hand (wear gloves), spray briefly and rub liquid onto the surface of the adapter. The use of GREEN&CLEAN IK does not replace but only precedes the necessary maintenance with oil.

DANGER: Extremely flammable liquid and vapour. Causes serious eye damage. May cause drowsiness or dizziness. If medical advice is needed, have product container or label at hand. Keep out of reach of children. Wear protective gloves/protective clothing/eye protection/face protection. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Immediately call a POISON CENTER or doctor.

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Spectre/temps d'action: virucide sur virus enveloppés (pex HBV/HCV/HIV/Herpes-simplex type I): 15 secondes; bactéricide, fongicide (C. albicans): 2 minutes; tuberculocide: 2 minutes.

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**METASYS Medizintechnik GmbH**