
## Glossary

	Konformität mit MDR (EU) 2017/745 / Conformity with MDR (EU) 2017/745 / Marquage CE de conformité / Marchamo CE de conformidad / Marcatura di conformità CE
	Temperaturbegrenzung / Temperature limitation / Limite de température / Limitación de temperatura / Limitazione di temperatura
	Verwendbar bis / Expiry date / Date d'expiration / Fecha de caducidad / Data di scadenza
	Hersteller / Manufacturer / Fabricant / Fabricante / Fabbicante
	Artikelnummer / Catalogue number / Numéro de catalogue / Número de artículo / Numero d'articolo
	Chargennummer / Batch code / Numéro de lot / Número de lote / número de lote
	Medizinprodukt / Medical device / Dispositif médical / Producto sanitario / Dispositivo medico
	Gebrauchsanweisung beachten / Consult instructions for use / Consultez le mode d'emploi / Observar las indicaciones de uso / Osservare le istruzioni per l'uso
	Verwendung nur durch Fachpersonal / Use only by professionals / Utilisation par du personnel spécialisé uniquement / Sólo podrá ser utilizado por personal especializado / Utilizzo solo da parte di professionisti

37440/3720 - Drewe Design Rev. 131120-35961

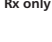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MegaBase® adhesive for dental prostheses.

## Indications for use

Adhesive primer for adhesion between denture relining material and PMMA acrylic resins.

## Patient target group

Patients for whom dental treatment is necessary.

## Intended users

Dentists, dental technicians

## Cleaning and disinfection

The denture base with relining can be disinfected (immersion disinfection).

## Instructions for use

The adhesive is applied uniformly thinly and comprehensively to all smoothed, clean areas of the prosthesis where MegaBase® is to adhere, twice at an interval of about 30 sec. The drying time is about 1:30 min. Only continue working once the adhesive is completely dry.

## Warnings

Do not use MegaBase® adhesive with condensation-curing silicones, polyether or polysulfide materials. Do not use MegaBase® adhesive after the expiry date. For dental use only. Should the contents of the bottle be spilled, it is recommended that the room be ventilated and the spilled liquid absorbed with a universal binding agent.

## Keep away from children!

## Contraindication

MegaBase® adhesive must not be used in case of allergy to any of the ingredients. Not desired side effects of this medical device are not to be expected in case of proper processing and application. However, immune reactions (e. g. allergies) or irritations principally cannot be excluded. Should you get to know non-desired side effects – also in doubtful cases – we appreciate in any case to be informed with a detailed description of the attendant circumstances and symptoms. We will follow each hint.

## Serious incidents

All serious incidents relating to the product have to be reported to the manufacturer and to the competent authority of the member state where the user and/or patient is resident.

## Disposal

Dispose of the contents / container according to local / regional / national / international regulations and according to the safety data sheet.

## Storage conditions

Store in a dry place at room temperature; do not expose to sunlight and avoid exposure to extreme temperature fluctuations. Close the bottle immediately after use.

**Contains:** ethyl acetate, methyl methacrylate

## Potential risks

Observe the safety data sheet in the use of MegaBase® adhesive.

### Hazard statements

H225 Highly flammable liquid and vapour. H319 Causes serious eye irritation. H317 May cause an allergic skin reaction. H336 May cause drowsiness or dizziness.

### Precautionary statements

P210 Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. P241 Use explosion-proof electrical / ventilating / lighting / equipment. P303+P361+P353IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water / shower P305+P351+P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. P405 Store locked up. P501 Dispose of contents / container in accordance with local / regional / national / international regulations.

This product was developed for dental use and has to be applied according to the instructions. In the case of damages resulting from ulterior or improper use, the manufacturer assumes no liability. Furthermore the user is bound to check the product according to suitability and possible use in self dependence before using it subject to its purpose, especially in case the purpose is not mentioned in the instructions.

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