MEMOREG® 2/MEMOSIL® 2

DE	Gebrauchsanweisung	3
GB	Instructions for use	7
FR	Mode d'emploi	11
ES	Instrucciones de uso.....	15
IT	Istruzioni per l'uso	19
PT	Instruções de uso	23
NL	Gebruiksaanwijzing	27
SE	Bruksanvisning	31
DK	Brugervejledning	35
NO	Bruksanvisning	39
FI	Käyttöohjeet	43
GR	Οδηγίες χρήσης	47
PL	Instrukcja obsługi	51
RU	Инструкции по применению.....	55

KULZER
MITSUBI CHEMICALS GROUP

KULZER
MITSUBI CHEMICALS GROUP

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Instructions for use

MEMOREG® 2

Polyvinyl siloxane for direct application
very high hardness

Indications

- Standard interocclusal records
- location material for intra-oral pin tracing template

Contraindications

Memoreg 2/Memosil 2 should not be used for conventional impressions since its high final hardness makes it very difficult to remove from the mouth. Undesired tooth extractions may result. The use of this product is contraindicated in case of known or suspected allergies against components of this product.

Side effects

This product or one of its components may in particular cases cause hypersensitive reactions. If suspected, information on the ingredients can be obtained from the manufacturer.

MEMOSIL® 2

Polyvinyl siloxane for direct application
transparent

Indications

- Front, occlusal record
- Transfer matrix for brackets (indirect bonding)
- Fixing of x-ray balls when planning the implants

	MEMOREG® 2	MEMOSIL® 2
Working time*1	30 seconds	30 seconds
Minimum time in mouth*2	60 seconds	3 minutes
Shore A hardness	90	72
Product colours	pink	transparent

*1 Total working time (including mixing) at 23°C (73°F); shortened to 20s at oral temperature (35°C (95°F))

*2 Minimum time that the bite registration material must be in a closed mouth.

The data given are furnished to the best of our knowledge and are based on internal test results.

Instructions for use

Visually inspect for damage before first use. Damaged products must not be used. Mixing tips and bite registration tips are for single use only. To avoid cross contamination Bite Registration Tips and Mixing Tips must be disposed of immediately after initial use.

Dispense a small amount of material from a 50 ml cartridge before attaching a mixing tip. Attach mixing tip and fix by rotating clockwise. Dosing and mixing is done automatically on application.

When bite registration materials based on polyvinyl siloxane are touched with latex gloves, retraction solutions or freshly inserted composite fillings, it can change the curing behaviour.

Memoreg 2/Memosil 2 should be used at room temperature (23°C (73°F)). Lower temperatures, for example when the materials are stored in a refrigerator can extend the working time, and higher temperatures can shorten it.

Hardened polyvinyl siloxanes are chemically resistant. Clothes can be permanently stained if the contact material is not completely cured.

Memoreg 2/Memosil 2 are applied with the green mixing tip (1:1). It can also be combined with the Bite Registration Tip.

Disinfection

Memoreg 2/Memosil 2 can be treated with an aqueous sodium hypochlorite solution (5.25% for up to 10 minutes*). The treatment does not affect the surface or dimensions.

When using other disinfectant solutions suitable for silicones, follow the disinfection and processing instructions of the manufacturer.

* Gelson Luis Adabo, DDS, PhD, Elaine Zanarotti, Renata Garcia Fonseca, and Carlos Alberto dos Santos Cruz, DDS, PhD
Effect of disinfectant agents on dimensional stability of elastomeric impression materials. J Prosthet Dent, 81, 621-4, 1999

Storage conditions

Do not store above 25°C (77°F)!

Do not use the material after the expiration date.

Please quote batch number and the article number in all correspondence about the product.

Expiration date  and batch number : See note on packaging.

Warning and safety instructions

Avoid contact with skin and eyes. May be irritating to eyes and skin. Wear protective gloves according to EN 420, category III, with protection against chemical risks according to EN 374, type C. On contact with skin wash off with water and soap.

If the material contacts your eyes, rinse with copious amounts of water for several minutes. If symptoms persist, consult an eye specialist.

Do not swallow or ingest. If health problems arise after swallowing impression material, seek medical attention immediately.

Intestinal blockage may arise in rare cases.

Do not leave any residual material in the sulcus or oral cavity, as this can lead to abscess formation. Surplus material on the bite fork or in the patient's mouth should be avoided as this may result in gagging, swallowing or aspiration. To further reduce these risks during impression taking, the patient should be positioned as upright as possible.

According to the EU Medical Devices Regulation, users/ patients are obliged to report serious events with a medical device to the manufacturer and to the competent authority of the country in which they occurred. For disposal please follow the instructions on the material safety data sheet and national regulations.

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Kulzer GmbH
Kulzer, LLC