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METAL-POWDER Dry Blue

**Hersteller**

R-dental Dentalerzeugnisse GmbH • Winterhuder Weg 88 • 22085 Hamburg
Telefon 0 40 - 30707073-0 • Fax 040 - 30707073-73 • www.r-dental.com

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METAL-POWDER Dry Blue**Order-No.: MPS0475****Description and indication**

METAL-POWDER Dry Blue in a spray can is a pigment suspension for universal, intraoral or extraoral applications.

After a short pressure onto a single-use atomizing nozzle a spray fog will be induced, which leads quickly and efficiently to an optimal matting.

METAL-POWDER Dry Blue covers the tooth perfectly, so that a perfect optical representation of the different morphological surfaces of dentin and enamel will be achieved. The matting leads to detailed edge solutions, so that finest morphological details can be visualized with highest precision.

Recommended procedures**1. Preparing the spray can**

First the single-use atomizing nozzle will be locked cautiously onto the spray can. The atomizing nozzle can be turned 360° and the single-use cannula can be bent to achieve an optimal angle for spraying. Do not bend the cannula square.

Then the spray can is shaken several times vertically so that the ball in the socket will homogenize the pigment suspension.

2. Preparing the surfaces

The surfaces which will be matted should be clean and dry.

3. Isolation

A dry area and use of rubber dam (Order-No.: KDL31XX, XX = depends on strength) or latex-free dam (Order-No.: KDL3500) is recommended.

4. Matting

The spray can is held vertically and the atomizing nozzle will be cautiously pressed until a soft suspension extrude which is drawn uniformly over the surface of matting (start from mesial or distal over preparation). The distance between the atomizing nozzle and surface should be about 4 cm.

Attention: Short sprays are contraindicated, since suspension selectively leaking too much and leads to a too intense layer.

5. Optically scan

After matting the surface can be scanned with a corresponding camera (CAD/CAM) according to the manufacturer's instructions.

6. Cleaning the surface

The applied pigment suspension can be removed from teeth with water/air blowers. Intraoral cleaning can be supported with a prophylaxis brush.

Special storage and/or handling conditions

The spray can is under pressure and may burst if heated.

Shelf life see label. Do not use after expiry date.

Do not store above 25 °C and protect from temperatures > 50 °C and sunlight. Do not open or burn the spray can after use. Keep out of the reach of children!

Pay attention to the material safety data sheets (MSDS >> r-dental.com).

Caution: For dental use only. US Federal Law restricts this device to sale by or on the order of a dentist or licensed professional, to use or to order the use of this device.

The described product must be used according to the instructions for use and is determined for dental use only. There is no liability for deflections resulting from inappropriate use or not intended purposes. The user is responsible for testing the suitability of the product for any purposes not explicitly stated in the instructions for use.

If there are damages to this product within the warranty period, R-dental is obliged to replace or to repair this product. There is no further claim. Outside of the liability by law there is no liability of R-dental for any loss or damages arising from this product, independently whether these damages are direct, indirect, special, incidental or consequential.

© 2017 R-dental Dentalerzeugnisse GmbH. All rights reserved.

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R-dental Dentalerzeugnisse GmbH • Winterhuder Weg 88 • 22085 Hamburg • DE