# Mirafloss® Big

Gebrauchsanleitung • Instructions for use  
Istruzioni per uso

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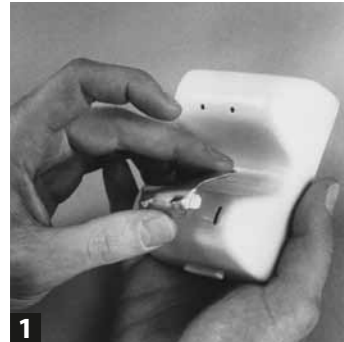
### ATTACH CUTTER SHIELD

- Peel protective covering off the cutter shield and snap cutter shield into vertical slots on housing.

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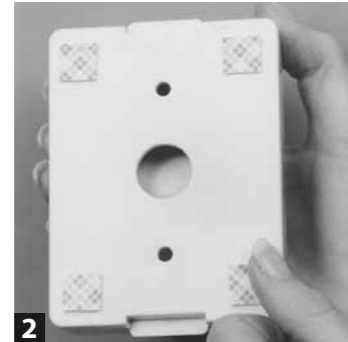
### ATTACH ADHESIVE PATCHES TOMOUNTING PLATE

- Leave covering on adhesive patches as shown.

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### DETACH MOUNTING PLATE FROM HOUSING

- Insert finger in large hole in mounting plate.
- Pull mounting plate away from housing.

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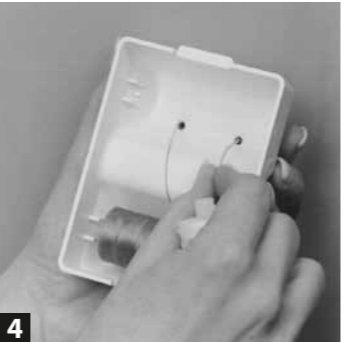
### LOAD FLOSS SPOOLS

- Thread first floss spool through either round hole in upper cup and snap this spool into lower cup.
- Thread second spool through other round hole in upper cup and snap this spool into upper cup.

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### MOUNTING THE BIG BOX

- Snap housing and mounting plate back together.
- Hold the box up to the chosen location and mark it in pencil.
- It can be mounted on a wall, on a countertop, on a piece of equipment, or on any clean smooth surface.
- Peel covering off adhesive patches and press the box onto the chosen location.

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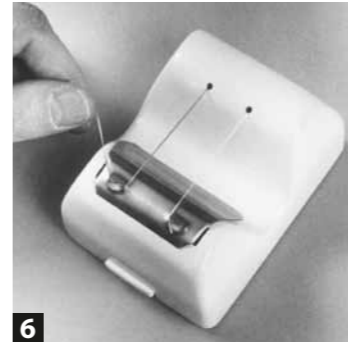
### HYGIENIC DISPENSING OF FLOSS

- Grasp the leading end of floss.
- Lay the floss in the middle of the cutter shield.
- Sever as shown, by swinging floss outward and upward.

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### DETACHING CUTTER SHIELD FROM HOUSING

- Pry top of cutter shield away from housing while pressing thumb against bottom of cutter shield.

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### STERILIZATION

- The housing, mounting plate and cutter shield are compatible with all approved sterilizing solutions.
- The cutter shield may also be detached from the housing and autoclaved.

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Hager & Werken GmbH & Co. KG