Optosil® Xantopren®

- DE Gebrauchsanweisung
- GB Instructions for use
- FR Mode d'emploi
- ES Instrucciones de uso
- IT Istruzioni per l'uso
- PT Instruções de uso
- NL Gebruiksaanwijzing
- SE Bruksanvisning
- DK Brugervejledning
- NO Bruksanvisning
- FI Käyttöohjeet
- GR Οδηγίες χρήσης
- CZ Návod k použití
- HU Használati utasítás
- LV Lietošanas instrukcija
- LT Naudojimo instrukcija
- PL Instrukcja obsługi

Manufacturer: Kulzer GmbH
Leipziger Straße 2
63450 Hanau (Germany)
Made in Romania
Importado e Distribuído no Brasil por
Kulzer South América Ltda.
CNPJ 48.708.010/0001-02
Rua Censo Sbrighi, 27 – cj. 42
São Paulo – SP – CEP 05036-010
sac@kulzer-dental.com
Resp. Técnica: Dra. Regiane Marton – CRO 70.705
Nº ANVISA- vicia embalagem

Pastý
Pâteux
Pastoso
Pastoso
Pastosus
Trögflytande
Pasta-agliig
Pastaligende
Pastamainen
παστέας
Pastovité
Paszta/zseró anyag
Pastos pavido
W postaci pasty

680656477/14

Technische Informationen

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Technical Information

Silicon based precision condensation curing impression material.

Optosil®/Xantopren®	EN ISO 4823	Colours	Dosage**		Mixing-time	Working-time†	Time in mouth		
			BASE					Activator universal Plus	
			Liquid	Paste				s	min : s
Optosil® Comfort®	Typ 0	yellow	1 spoon (10,7/9,5 ml)	6-8 drops (0,14 g/0,13 ml)	6-8 graduations (0,32 g/0,29 ml)	30	1:00	4:00	
Optosil® P Plus	Typ 0	yellow	1 spoon (14,4/9,5 ml)	4-6 drops (0,09 g/0,09 ml)	4-6 graduations (0,21 g/0,19 ml)	30	1:00	4:00	
Xantopren® H green	Typ 1	green	6 graduations (11,0/8,8 ml)	12 drops (0,27 g/0,26 ml)	12 graduations (0,65 g/0,59 ml)	30	1:00	4:00	
Xantopren® M mucosa	Typ 2	orange	12 graduations (9,0/7,6 ml)	12 drops (0,27 g/0,26 ml)	12 graduations (0,65 g/0,59 ml)	30	1:30	4:00	
Xantopren® L blue	Typ 3	blue	12 graduations* (8,6/7,6 ml)	12 drops (0,27 g/0,26 ml)	12 graduations (0,65 g/0,59 ml)	30	1:30	4:00	
Xantopren® VL Plus	Typ 3	blue	12 graduations* (8,5/7,6 ml)	12 drops (0,27 g/0,26 ml)	12 graduations (0,65 g/0,59 ml)	30	1:30	4:00	

* At room temperature of 23°C (73°F), 50% rel. humidity.
Higher temperatures shorten, lower temperatures prolong these times.
** 1 graduation = 1,3 cm (applies to mixing pad available)
** Bottom ring on mixing dish is equivalent to 7,6 ml
The stated dosage should be observed.

† Typ 0 = putty consistency
Typ 1 = heavy bodied consistency
Typ 2 = medium bodied consistency
Typ 3 = light bodied consistency

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Stand: 2019-07

User Advice

The Optosil®/Xantopren® assortment is a complete system. Components can be individually combined for all impression techniques. For best results usage with other silicon materials and Activators is not recommended. Visually inspect for damage before first use. Damaged products must not be used. Tubes and Tubs must be carefully sealed after use.

Contraindications: The use of this product is contraindicated in case of known or suspected allergies against components of this product. Optosil Comfort contains peppermint oil.

Side effects: This product or one of its components may in particular cases cause hypersensitive reactions. If suspected, information on the ingredients can be obtained from the manufacturer. Surplus material on the impression tray or in the patient's mouth should be avoided as this may result in gagging, swallowing or aspiration. To further reduce these risks during impression taking, the patient should be positioned as upright as possible.

Pouring the model: Pouring can take place 30 minutes after the impression is removed from the mouth. Impressions remain dimensional stable up to 7 days.

Recommended model material: All commercially available modeling stones can

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