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R-SI-LINE® Lab-PUTTY

Hersteller

R-dental Dentalerzeugnisse GmbH • Winterhuder Weg 88 • 22085 Hamburg
T 040 - 30707073-0 • F 040 - 30707073-73 • I www.r-dental.com

R-SI-LINE® Lab-PUTTY

REF LPU1016

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REF LPU1016

Description and indication

R-SI-LINE® Lab-PUTTY is used as an universal precision impression material for extraoral lab applications, for examples overcasts.

Characteristics

R-SI-LINE® Lab-PUTTY is a kneadable, condensation curing silicone based impression material with high final hardness. It is easy to mix and is not sticky.

R-SI-LINE® Lab-PUTTY meets the requirements of DIN EN ISO 4823 Type 0 (very high consistency - putty).

Application**Dosage**

Take one spoon and remove the excess material.

For every spoon diameter take 3 strands of catalyst paste R-SI-LINE® C-HARDENER (REF KPH1008).

Knead both materials for about 30 s until the colour is homogeneous.

The working time is about 1 min 30 s. The hardening time is about 8 min at room temperature (start mixing - hardening).

Times mentioned for 74 °F. Higher temperatures shortens, lower temperatures prolongs these times.

Special storage and/or handling conditions

Shelf life see label. Do not use after expiry date.

Avoid exposure to sunlight. Do not store above 25 °C (78 °F)! Keep out of the reach of children!

Incompatibilities

Until now no side effects are known in the application of C-silicones. Discontinue using the products if allergic reactions are observed.

Technical data

Mixing time:	about 30 s
Working time incl. mixing time:	< 1 min 30 s
Time (start mixing - hardening) at RT:	≥ 8 min
Dimensional change after 4 hours:	< 0,2 %
Dimensional change after 24 hours:	< 0,6 %
Shore A-hardness:	about 85

R-SI-LINE is a registered trademark of R-dental Dentalerzeugnisse GmbH.

Caution: For dental use only.

Warranty

R-dental Dentalerzeugnisse GmbH warrants this product will be free from defects in material and manufacture. R-dental Dentalerzeugnisse GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. The user is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusively remedy and R-dental Dentalerzeugnisse GmbH's sole obligation shall be repair or replacement of the product.

Limitation of Liability

Except where prohibited by law, R-dental Dentalerzeugnisse GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

© 2019 R-dental Dentalerzeugnisse GmbH. All rights reserved.

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R-dental Dentalerzeugnisse GmbH • Winterhuder Weg 88 • 22085 Hamburg • DE