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R-SI-LINE® PUTTY MATIC

**Hersteller:**R-dental Dentalerzeugnisse GmbH • Winterhuder Weg 88 • 22085 Hamburg
T 040-30707073-0 • F 040-30707073-73 • I www.r-dental.com**PMK1013 (R-SI-LINE® PUTTY MATIC)****Beschreibung und Indikation**

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REF PMK1013 (R-SI-LINE® PUTTY MATIC)

Description and indication

R-SI-LINE® PUTTY MATIC is an automatically mixed addition curing impression material on vinylpolysiloxane base with a mixing ratio of 5:1 (base:catalyst). The 5:1 cartridge can be used with commercial mixing devices according to the instructions for use of the mixing device manufacturer.

R-SI-LINE® PUTTY MATIC is used as a preliminary impression material for the putty-wash-technique (two steps) in conjunction with R-SI-LINE®-correction materials (for example with - LIGHT, - UltraLIGHT) or is used generally as a base impression material for the sandwich-technique (one step) in conjunction with R-SI-LINE®-correction impression materials (e. g. with - MEDIUM SH).

Characteristics

R-SI-LINE® PUTTY MATIC is preferred used for impressions in conjunction with the super hydrophilic R-SI-LINE®-correction impression materials (super hydro). It shows good setting properties and an optimized detail accuracy.

R-SI-LINE® PUTTY MATIC meets the requirements of ISO 4823, type 0 (putty consistency)

Recommended procedures

1. Preparation of the tray

It may be used perforated or non-perforated metal or plastic trays. In order to improve the adhesion of R-SI-LINE® PUTTY MATIC to the tray, one of the R-SI-LINE®-adhesives should be applied as a thin even film.

2. Insertion and preparation of the 5:1 cartridge

Before first use hold the cartridge with the sealed outlet openings facing upwards and take the handle of the sealing cap and remove the sealing upwards and dispose it. Then insert the R-SI-LINE® PUTTY MATIC cartridge according to the instructions for use of the mixing device manufacturer. Before screwing on a new mixing cannula the cartridge should be tested if both pastes will be extruded equally.

Then the R-SI-LINE® MIXING CANNULA will be placed over the openings of the cartridge. After the mixing cannula is correctly seated and the central hexagonal socket at the lower end of the cannula is properly aligned with the hexagonal drift shaft, then the yellow bayonet ring will be placed over the mixing cannula and pushed down completely and turned clockwise with a 1/4 turn clockwise to lock the mixing cannula securely in place.

Attention: Check whether the hexagonal drift shaft does engage when starting. The spiral within the mixing cannula must rotate when starting.

Attention: If the used mixing device offers different mixing speed levels, always the *lowest* mixing speed must be used!

Do not use the first 3 cm of the mixed impression material.

3 a. Putty-wash-technique (two steps)

Before application a small amount of material (always 1-2 cm, except first 3 cm) should be pressed out and should be disposed immediately.

The automatically mixed impression material is then loaded into the prepared tray. Insert the loaded tray into the mouth within 1 min 30 s after start of mixing. Maintain the position for PUTTY MATIC at least 3 min in the mouth. R-SI-LINE® PUTTY MATIC could be used directly after taking out of the mouth with R-SI-LINE®-correction impression materials (e. g. with - LIGHT SH, - UltraLIGHT SH) according to the instructions for use.

3 b. Sandwich-technique (one step)

Before application a small amount of material (always 1-2 cm, except first 3 cm) should be pressed out and should be disposed immediately.

The automatically mixed impression material is loaded directly into the prepared tray. Insert the loaded tray into the mouth within 1 min 30 s after start of mixing together with the R-SI-LINE correction impression material (e. g. with - MEDIUM SH) (working time). Apply the parallel mixed correction impression material according to the instructions for use directly onto the top of the base material in the tray and on the prepared areas into the mouth.

Maintain this two-phase impression with normal set components at least 3 min in the mouth (hardening time).

Times mentioned for 74 °F (23 °C). Higher temperatures shortens, lower temperatures prolongs these times.

4. Casting and disinfection

The impression should be stored at normal room temperature (not above 78 °F) and can be poured 30 min after removal from the mouth. There are no other time limitations. No separating materials are needable.

The impressions can be copper - and silver plated and disinfected with standard disinfection solutions.

Special storage and/or handling conditions

Before first use hold the cartridge with the sealed outlet openings facing upwards and take the handle of the sealing cap and remove the sealing upwards and dispose it. Closing the cartridge by turning on the original cap between use could cause cross contamination of base - and catalyst paste and thus premature setting.

When screwing on a new mixing cannula the cartridge should be tested if both pastes will be extruded equally.

Before application a small amount of material (always 1-2 cm, except first 3 cm) should be pressed out and should be disposed immediately.

The filled MIXING CANNULA functions as a seal until the next application. Do not re-use the original cartridge cap.

To remove the used mixing cannula turn the bayonet ring counter clockwise with a 1/4 turn and draw it off.

Pay attention to the material safety data sheets (MSDS >> www.r-dental.com).

Expiry date see label. Do not use after expiry date.

Do not store above 78 °F (25 °C)! Avoid exposure to sunlight. Keep out of the reach of children!

Do not use R-SI-LINE® PUTTY MATIC with standard condensation reaction type silicone impression materials.

Do not use Latex gloves, these may interfere with the setting reaction of the A-silicone impression material.

Technical data

Mixing time:	automix
Working time incl. mixing time:	< 1 min 30 s
Recommended time in mouth:	≥ 3 min
Time (Start mixing - withdrawal of mouth):	≥ 4 min 30 s
Dimensional change after 24 hours:	< 0.1 %
Elastic recovery:	> 98.8 %
Strain in compression:	about 2.6 %
Shore A-hardness:	about 67

R-SI-LINE is a registered trademark of R-dental Dentalerzeugnisse GmbH.

Caution: For dental use only. US Federal Law restricts this device to sale by or on the order of a dentist or licensed professional, to use or to order the use of this device.

Warranty

R-dental Dentalerzeugnisse GmbH warrants this product will be free from defects in material and manufacture. R-dental Dentalerzeugnisse GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. The user is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and R-dental Dentalerzeugnisse GmbH's sole obligation shall be repair or replacement of the product.

Limitation of Liability

Except where prohibited by law, R-dental Dentalerzeugnisse GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

© 2020 R-dental Dentalerzeugnisse GmbH. All rights reserved.