PRODUCT DESCRIPTION

SoftCut-E Polishers are used for polishing fragile porcelain veneers, inlays and onlays. Use PA (green band) for polishing after rough finishing (for example with Dura-Green Stones) Use PB (blue band) for polishing after fine finishing (for example with Dura-White Stones). Adjust polishing performance by varying the pressure: the lighter the pressure, the higher the polish. Start with normal to light pressure to adjust polishing performance.

PRECAUTIONS AND WARNINGS

- Never exceed the maximum permissible rotation speeds described in Table A.
- Mount an unmounted wheel onto the mandrel securely.
- Insert the shank completely into the chuck of the handpiece.
- Do not use damaged or defective instruments i.e. corrosion, surface flaw, deformation, dirt and exposure of shank due to repeated use.
- Perform test run to check safety and smooth operation without run-out prior to use.
- Avoid any use at acute angles or with excessive pressure.
- Always use an unmounted wheel in clockwise motion
- Use local dust extractor and protective equipment while using.
- This product is intended for use by dental professionals only.
- Do not use this product for any purposes other than the intended use.
- Dispose in accordance with relevant local and national regulation.

STORAGE

Avoid exposure to moisture, corrosive chemical and its vapor. Keep away from external pressure (physical load) and avoid contamination.

CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.

Location of the symbols glossary:
https://www.shofu.com/en/symbols_glossary/

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