Spectrum[®] TPH[®]3

Universal Microhybrid Composite Restorative

Composite micro-hybride de restauration universel

Composito microibrido universale

Universelles Microhybrid-Komposit-Füllungsmaterial

Composite restaurador micro-híbrido universal

Material de restauro compósito híbrido universal

Universeel microhybride-composiet-vullingsmateriaal

Universal mikrohybridkomposit

Universal mikrohybrid komposit fyldningsmateriale

Universal mikrohybridkomposit

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CAUTION: For dental use only.

USA: Rx only.

Content	Page
1 Product description	2
2 Safety notes	3
3 Step-by-step instructions	5
4 Hygiene	7
5 Lot number, expiration date and correspondence	8

1 Product description

Spectrum® TPH®3 restorative is a visible-light activated, radiopaque sub-micron hybrid composite for anterior and posterior restorations. The composite is pre-dosed in Compules® Tips or delivered in traditional syringes and is available in a selection of precise VITA®¹ shades.

1.1 Indications

- Restorations of all cavity classes in anterior and posterior teeth.

1.2 Contraindications

- Use with patients who have a history of severe allergic reaction to methacrylate resins or any of the other components.

1.3 Delivery forms

- Predosed Compules® Tips
- Syringes

1.4 Composition

- Silanated barium-aluminium-borosilicate glass
- Urethane modified Bis-GMA resin
- Fumed silica
- UV stabilizer
- Ethyl-4(dimethylamino)benzoate
- Camphorquinone
- Butylated hydroxy toluene (BHT)
- Fluorescent agent
- Iron oxide pigments
- Titanium dioxide

¹ Not a registered trademark of Dentsply Sirona, Inc.

1.5 Compatible adhesives

The material is to be used following application of a suitable dentin/enamel adhesive and is chemically compatible with conventional (meth)acrylate-based dentin/enamel adhesives including Dentsply Sirona adhesives designed for use with visible light cured composite restoratives (see complete instructions for use of selected adhesive).

2 Safety notes

Be aware of the following general safety notes and the special safety notes in other chapters of these Instructions for Use.

Safety alert symbol.

- This is the safety alert symbol. It is used to alert you to potential personal injury hazards.
- Obey all safety messages that follow this symbol to avoid possible injury.

2.1 Warnings

The material contains methacrylates and polymerizable monomers which may be irritating to skin, eyes and oral mucosa and may cause sensitization by skin contact and allergic contact dermatitis in susceptible persons.

- **Avoid eye contact** to prevent irritation and possible corneal damage. In case of contact with eyes rinse with plenty of water and seek medical attention.
- **Avoid skin contact** to prevent irritation and possible allergic response. In case of contact, red-dish rashes may be seen on the skin. If contact with skin occurs, remove material with cotton and alcohol and wash thoroughly with soap and water. In case of skin sensitization or rash, discontinue use and seek medical attention.
- **Avoid contact with oral soft tissues/mucosa** to prevent inflammation. If accidental contact occurs, remove material from the tissues. Flush mucosa with plenty of water and expectorate/evacuate the water. If inflammation of mucosa persists, seek medical attention.

2.2 Precautions

This product is intended to be used only as specifically outlined in these Instructions for Use.

Any use of this product inconsistent with these Instructions for Use is at the discretion and sole responsibility of the dental practitioner.

- If refrigerated, allow material to reach room temperature prior to use.
- Contact with saliva, blood and sulcus fluid during application may cause failure of the restoration. Use adequate isolation such as rubber dam.
- Use protective measures for the dental team and patients such as glasses and rubber dam in accordance with local best practice.
- Devices marked "single use" on the labeling are intended for single use only. Discard after use. Do not reuse in other patients in order to prevent cross-contamination.
- The syringes cannot be reprocessed. To prevent syringes from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the syringes are handled offside the dental unit with clean/disinfected gloves. Discard syringes if contaminated.
- As additional precautionary measure, syringes may be protected from gross debris but not from all contamination by applying a protective barrier.
- Use of Compules® Tips with the Dentsply Sirona Compules® Tips Gun is recommended.
- The Compules® Tips Gun is only intended to be used with Dentsply Sirona Compules® or Compula® Tips.
- For further information please refer to the Dentsply Sirona Compules® Tips Gun Instructions for Use.

- The shade guide and the individual shade guide tabs cannot be reprocessed. To prevent the shade guide and the individual shade guide tabs from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the shade guide and the individual shade guide tabs are handled with clean/disinfected gloves.
- The material should extrude easily. **DO NOT USE EXCESSIVE FORCE.** Excessive pressure may cause the Compules® Tips to rupture or to eject from the Compules® Tips Gun.
- Tightly close syringes immediately after use.
- Interactions:
 - Do not use eugenol- or hydrogen peroxide-containing materials in conjunction with this product because they may interfere with hardening and cause softening of the polymeric components of the material.
 - Spectrum® TPH®3 restorative is a light cured material. Therefore, it should be protected from ambient light. Proceed immediately once the material has been placed.
 - If mineral-impregnated (e.g., ferric compounds) retraction cords and/or hemostatic solutions are used in conjunction with adhesive procedures, marginal seal may be adversely affected, allowing microleakage, subsurface staining and/or restoration failure. If gingival retraction is necessary, use of plain, non-impregnated cord is recommended.

2.3 Adverse reactions

- Eye contact: Irritation and possible corneal damage.
- Skin contact: Irritation or possible allergic response. Reddish rashes may be seen on the skin.
- Contact with mucous membranes: Inflammation (see Warnings).

2.4 Storage conditions

- Inadequate storage conditions may shorten the shelf life and may lead to malfunction of the product.
- Store at temperatures between 2 °C and 28 °C (35 °F and 82 °F).
 - Keep out of direct sunlight and protect from moisture.
 - Do not freeze.
 - Do not use after expiration date.
 - Store in a dry environment (< 80% relative humidity).

3 Step-by-step instruction

3.1 Shade selection

Shade selection should be made prior to the restorative procedure while the teeth are hydrated. Remove extraneous plaque or surface stain if any with prophylaxis paste. Use the Spectrum® TPH^{®3} shade guide provided with samples of original Spectrum® TPH^{®3} restorative material. The color coding dot on the shade guide matches the colored label on the syringe.

Alternatively, a VITAPAN^{®2} classical shade guide may be used. The Spectrum® TPH^{®3} shade corresponds to the central part of the respective VITA® tooth.

Avoid contamination of the shade guide.

To prevent the shade guide from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the shade guide is handled offside the dental unit with clean/disinfected gloves. For shade selection, remove individual tabs from shade guide holder.

3.2 Cavity preparation

1. Clean the tooth surface with a fluoride free prophylaxis paste (e.g., Nupro® prophylaxis paste).
2. Prepare cavity (unless not necessary, e.g., cervical lesion).
3. Use adequate isolation such as rubber dam.
4. Rinse surface with water spray and carefully remove rinsing water. Do not desiccate the tooth structure.

3.2.1 Matrix placement and wedging (Class II restorations)

1. Place a matrix (e.g., AutoMatrix® or Palodent® matrix system) and wedge. Burnishing of the matrix band will improve contact and contour. Pre-wedging or BiTine® ring placement is recommended.

3.3 Pulp protection, tooth conditioning/dentin pre-treatment, adhesive application

Refer to adhesive manufacturer's Instructions for Use for pulp protection, tooth conditioning and/or adhesive application. Once the surfaces have been properly treated, they must be kept uncontaminated. Proceed immediately to placement of the material.

3.4 Application

Apply material in increments immediately after the application of the adhesive. The material is designed to be cured in increments up to a 3 mm depth/thickness.

Using Compules® Tips

Excessive force.

Injury.

1. Apply slow and steady pressure on the Compules® Tips Gun.
2. Do not use excessive force. Compules® Tip rupture or ejection from Compules® Tips Gun may result.

² Not a registered trademark of Dentsply Sirona, Inc.

1. Insert Compules® Tip into the notched opening of the Compules® Tips Gun. Be certain that the collar on the Compules® Tip is inserted first.
2. Remove the colored cap from the Compules® Tip. The Compules® Tip may be rotated 360° to gain the proper angle of entrance into the cavity or to the mixing pad.
DO NOT USE EXCESSIVE FORCE.
Indirect use: Dispense the necessary amount of material onto a mixing pad and protect against light. Apply material into the cavity with a plastic instrument.
- or -
Direct intraoral use: dispense the material directly into the cavity preparation using a slow, steady pressure.
3. To remove the used Compules® Tip, be sure that the Compules® Tips Gun plunger is pulled back completely by allowing the handle to open to its widest position. Apply a downward motion to the front end of the Compules® Tip and remove.

Using syringes

Avoid contamination of the syringe.

To prevent the syringe from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the syringe is handled offside the dental unit with clean/disinfected gloves.

1. Remove the cap.
2. Turn the handle of the syringe slowly in a clockwise direction and dispense the necessary amount of the material onto a mixing pad.
3. Point the front tip of the syringe upwards and turn the handle anti-clockwise to prevent oozing of the material.
4. Immediately close the syringe with the cap.
5. Protect the restorative material on the mixing pad against light.

3.5 Light curing

Insufficient curing.

Inadequate polymerization.

1. Check compatibility of curing light.
2. Check curing cycle.
3. Check minimum irradiance.
4. Cure each area of each increment for the recommended curing time.
5. Check distance to surface to be cured.

1. Light cure each increment with a suitable curing light designed to cure materials containing camphorquinone (CQ) initiator, i.e. peak spectral output 440-480nm. Minimum light output must be at least 550 mW/cm². Refer to curing light manufacturer's Instructions for Use for compatibility and curing recommendations.
2. Cure each layer according to the table below.

Curing Light Output \geq 550 mW/cm²

Shades	Increment/layer	Curing time
A1; A2; A3; A3,5; B1; Incisal-B1; B2; C2; C3; D3	3 mm	20 sec
Opaque-A2; Opaque-A3,5; A4; B3; C4	2 mm	20 sec

3.6 Finishing and polishing

1. Contour the restoration using finishing burs or diamonds.
2. Use Enhance® finishing devices for additional finishing.
3. Prior to polishing, check that surface is smooth and free of defects.
4. For achieving a very high luster of the restoration, we recommend Enhance® and Enhance® PoGo®, finishing and polishing instruments or Prisma® Gloss™ Regular and Extrafine polishing paste.

For contouring, finishing, and/or polishing, follow the manufacturer's Instructions for Use.

4 Hygiene

Cross-contamination.

Infection.

- Do not reuse single use products. Dispose of in accordance with local regulations.
- Syringes cannot be reprocessed. Dispose of contaminated syringes in accordance with local regulations.
- Reprocess reuseable products according to the instructions.

4.1 Compules® Tips Gun

For reprocessing instructions please refer to the Instructions for Use of the Compules® Tips Gun, which is available on our webpage at www.dentsplysirona.com/IFU. If requested, we will send you a free printed copy of the Instructions for Use in the language you require within 7 days. In the USA, call 1-800-532-2855. Outside North America, use the order form provided on www.dentsplysirona.com/IFU for this purpose.

4.2 Compules® Tips – after intraoral use and/or if contaminated

Cross-contamination.

Infection.

- Do not reuse Compules® Tips. Dispose of Compules® Tips in accordance with local regulations.

4.3 Syringes – cross-contamination

Cross-contamination.

Infection.

- The syringes cannot be reprocessed.
- To prevent the syringes from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the syringes are handled with clean/disinfected gloves. Do not reuse syringes if contaminated.
- Dispose of contaminated syringes in accordance with local regulations.

To prevent syringes from exposure to spatter or spray of body fluids or contaminated hands, or oral tissues, use of a protective barrier is recommended. The use of protective barriers is an additional precautionary measure against gross debris but not against all contamination.

Incidental contact of the syringe with water, soap or a water-based hospital-level disinfection solution will not damage syringe body. Do not allow any solution contact with contained material. Discard composite material that has been in contact with any fluid or non-sterile instrument.

Repeated liquid contact may damage label. Dry the syringe with a lint-free single use cloth.

NOTE: Vigorous wiping can destroy the label. Wipe syringe gently.

4.4 Shade guide and individual shade guide tabs – cross-contamination

Cross-contamination.

Infection.

- The shade guide and the individual shade guide tabs cannot be reprocessed.
- To prevent the shade guide and the individual shade guide tabs from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the shade guide and the individual shade guide tabs are handled with clean/disinfected gloves. Do not reuse shade guide and the individual shade guide tabs if contaminated.
- Dispose of contaminated shade guide and the individual shade guide tabs in accordance with local regulations.

4.5 Disposal

Dispose of in accordance with local regulations.

5 Lot number (), expiration date () and correspondence

1. Do not use after expiration date.
ISO standard is used: "YYYY-MM" or "YYYY-MM-DD".
2. The following numbers should be quoted in all correspondence:
 - Reorder number
 - Lot number
 - Expiration date
3. Any serious incident in relation to the product should be reported to the manufacturer and the competent authority according to local regulations.

© Dentsply Sirona 2019-06-18

[These Instructions for Use are based on Master Version 08]

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Dentsply DeTrey GmbH