AESCULAP®

en **Instructions for use/Technical description**
STERILIT I Drip Lubricator JG598

USA Note for U.S. users

This Instructions for Use is NOT intended for United States users. Please discard. The Instructions for Use for United States users can be obtained by visiting our website at www.aesculapusaifus.com. If you wish to obtain a paper copy of the Instructions for Use, you may request one by contacting your local Aesculap representative or Aesculap's customer service at 1-800-282-9000. A paper copy will be provided to you upon request at no additional cost.

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1. About this document

Note

General risk factors associated with surgical procedures are not described in these instructions for use.

1.1 Scope

These instructions for use apply to the product below:

Art. no.	Designation
JG598	STERILIT I Drip Feed Oiler

1.2 Safety messages

Safety messages make clear the dangers to patient, user and/or product that could arise during the use of the product. Safety messages are labeled as follows:

⚠ WARNING

Indicates a possible threat of danger. If not avoided, minor or moderate injury may result.

2. Clinical use

2.1 Product description

Oil for the maintenance of surgical instruments prior to sterilization. Suitable for steam sterilization methods validated according to EN ISO 17665.

Properties

- Oil film permeable to water vapor
- Oil film not affecting the effectiveness of sterilization
- Lubrication and anticorrosion effect
- Silicone-free, no staining or encrustations on instrument surfaces

2.2 Areas of use and limitations of use

2.2.1 Intended use

STERILIT is used within the reprocessing process to lubricate joints and movable parts in order to reduce friction and prolong product life. It is used on the cleaned medical devices prior to steam sterilization validated according to EN ISO 17665.

2.2.2 Indications

Note

The manufacturer is not responsible for any use of the product against the specified indications and/or the described applications.

For indications, see Intended use.

2.2.3 Contraindications

No known contraindications.

2.3 Safety information

2.3.1 Clinical user

General safety information

To prevent damage caused by improper setup or operation, and to not compromise the manufacturer warranty and liability:

- ▶ Use the product only according to these instructions for use.
- ▶ Follow the safety and maintenance instructions.
- ▶ Ensure that the product and its accessories are operated and used only by persons with the requisite training, knowledge and experience.
- ▶ Store any new or unused products in a dry, clean, and safe place.
- ▶ Prior to use, check that the product is in good working order.
- ▶ Keep the instructions for use accessible for the user.

Note

The user is obligated to report all severe events in connection with the product to the manufacturer and the responsible authorities of the state in which the user is located.

Notes on surgical procedures

The medical professional will make decisions on concrete applicability based on the warranted properties and technical data.

2.3.2 Product specific safety information

Risk of injury and/or malfunction!

- ▶ Check product for loose, bent, broken, cracked, worn or missing parts before every application.

Safe handling

- ▶ Prior to the application of STERILIT, medical products must be cleaned and disinfected according to their respective instructions for use.
- ▶ Only apply prior to sterilization.
- ▶ Store in a dry place, avoid direct sun exposure/UV radiation
- ▶ Keep out of reach of children.
- ▶ Observe the Safety Data Sheet, see eifu.bbraun.com

Emergency telephone number

+49 (0) 6132-84463 GBK GmbH

- 24/7
- 190 languages
- medical advice in case of poisoning
- Advice in the event of an accident or other incidents

Unique Formula Identifier (UFI-Code)

UFI: JERA-E6J8-170F-GMFN

2.3.3 Sterility

The product is supplied non-sterile and shall be used under non-sterile conditions.

- ▶ Inspect the new product after removing its transport packaging and prior to first use to ensure it is in good working order.

2.4 Application

- ▶ Before first use, cut off the tip on the apex of the cap, see Fig. A.
- ▶ Observe the instructions for use of the corresponding instrument.
- ▶ Shake well prior to use
- ▶ Apply the oil directly onto the joint area.
- ▶ Open and close the instrument several times so that the oil is evenly spread.
- ▶ Remove excess oil using a lint-free cloth.

3. Service addresses

Aesculap Technischer Service

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78532 Tuttlingen / Germany

Phone: +49 7461 95-1601

Fax: +49 7461 16-2887

E-Mail: ats@aesculap.de

Other service addresses can be obtained from the address indicated above.

4. Disposal

- ▶ Adhere to national regulations when disposing of or recycling the product, its components and its packaging.

5. Symbols on product and packaging

	Manufacturer's order number
50 ml	Nominal filling quantity
	Conformity mark for pre-packaging ordinance
	Use by: see bottom
	Emergency telephone number
	Observe the instructions for use

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