#### PRODUCT DESCRIPTION

Super-Snap is an easy to use, color co-ordinated, 4-step disposable disk system with metal-free center, designed for contouring, finishing and polishing of all types of direct aesthetic resins (microfilled, micro-hybrid, nano and nano-hybrid). Dedicated Super-Snap metal and plastic mandrels are available.

#### INDICATIONS FOR USE

Contouring, finishing and polishing of all types of direct aesthetic resin materials

#### PRECAUTIONS AND WARNINGS

- Do not use damaged, defective or worn metal mandrels, i.e. corrosion, surface degradation due to repeated use.
- Use protective mask and eyewear when operating handpieces.
- Avoid contacting soft tissues to avoid injury during use.
- Super-Snap Disks and plastic mandrels are designed for single use only. Discard the Super-Snap Disks and plastic mandrel after use in accordance with the disposal protocol of your healthcare facility.
- Do not use this product for any purposes other than specifically outlined in these instructions for use.
- The product is intended for use by dental professionals only.

#### DIRECTIONS FOR USE

- Select proper abrasive for each case, Super-Snap Black, Super-Snap Violet to remove excess restorative material and contour to the desired shape. When using Super-Snap Disks, mount the disk securely on the dedicated Super-Snap mandrel.
- Insert the shank completely into the chuck of the handpiece (micro motor), following the manufacturer’s instruction. Perform test run to check safety and smooth operation without run-out prior to use.
- Apply uniform, steady pressure for finishing and feather light pressure for polishing. Unidirectional movement of the disks prevents formation of scratches and a visible step at the restoration-tooth junction.
  - Follow the recommended finishing and polishing sequence with the color coded disks for the best results. Reducing or skipping the recommended steps may compromise the quality of the final polish.

- Use saliva ejector or suction to eliminate moisture and debris during the procedure. Use protective mask and eyewear while using the product.
- Avoid exceeding maximum permissible speed.

Product name	Maximum permissible rotation speed	Water spray
Super-Snap Disks with metal mandrel	15,000 min <sup>-1</sup>	No
Super-Snap Disks with plastic mandrel	5,000 min <sup>-1</sup>	No

- Clean the debris from restoration surface after each finishing/ polishing step.
- After using dedicated metal mandrel, remove any contaminants immediately, disinfect and sterilize. Dispose the Super-Snap Disks and plastic mandrel after each use.

#### STORAGE AND STERILIZATION

Super-Snap Disks and mandrels should be stored away from direct sunlight, moisture and corrosive chemicals/vapor. Sterilize Super-Snap metal mandrels before use.

For reuse after using, remove any contaminants immediately with washer disinfecter, then sterilize.

Product name	Cycle Type	Temperature	Holding time	Dry time
Super-Snap metal mandrel	Pre-vacuum 3 times	134 °C (273 °F)	3’00’’	10’00’’

Follow manufacturer’s instructions for use of washer disinfecter and sterilizer, respectively. Chloride disinfectant and dry heat should be avoided

**CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.**

For the details about REPROCESSING procedures, visit **www.shofu.de**.

**Location of the symbols glossary:**  
**https://www.shofu.com/en/symbols\_glossary/**

en	READ CAREFULLY BEFORE USE
<b>de</b>	VOR GEBRAUCH SORGFÄLTIG DURCHLESEN
<b>fr</b>	A LIRE ATTENTIVEMENT AVANT USAGE
<b>nl</b>	LEES ZORGVULDIG VOOR GEBRUIK
<b>pt</b>	LEIA COM ATENÇÃO ANTES DE USAR
<b>ru</b>	ВНИМАТЕЛЬНО ПРОЧЕШЬ ПЕРЕД ПРИМЕНЕНИЕМ
<b>sv</b>	LÄS OMSÄKTNADEN FÖR ANVÄNDNINGEN
<b>zh</b>	使用前請細心閱讀說明書

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