making work easy

www.renfert.com

de

en

fr

it

es

pt

tr

ru

pl

da

uk

zh

ja

ko

SYMPRO

Made in Germany

21-7025 Rev 02 30042021

de

en

fr

it

es

pt

tr

ru

pl

nl

da

uk

sv

no

fi

hu

cs

sk

sl

hr

bg

ro

lt

lv

et

el

cy

ga

is

mt

zh

ja

ko

Inhalt

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Contents

en

1	Introduction	2
1.1	Symbols	2
1.2	EU Conformity	2
2	Safety	3
2.1	Intended Use	3
2.2	Improper Use	3
2.3	Ambient Conditions for Safe Operation	3
2.4	Ambient Conditions for Storage and Transport	3
2.5	Hazard and Warning Information	3
2.5.1	General Information	3
2.5.2	Specific Information	4
2.6	Information about electromagnetic compatibility (EMC)	4
2.7	Authorized Persons	4
2.8	Disclaimer	4
3	Product Description	5
3.1	Components and Functional Elements	5
3.2	Scope of Delivery	5
3.3	Accessories	5
4	Setting Up	6
4.1	Unpacking	6
4.2	Setup	6
4.3	Electrical Connection	6
4.4	Hygienic Preparation	6
4.5	Application Table	8
4.6	Application parameters Cleaning agent	9
5	Operation	9
5.1	Preparing the object	9
5.2	Filling the cleaning vessel	10
5.3	Select the cleaning time and intensity	10
5.4	Starting, pausing, and stopping cleaning	10
5.5	Remove the cleaning object	11
5.6	Optional intensive cleaning with help:ex discolor f	11
5.7	Disposal of cleaning fluid	11
6	Cleaning / Maintenance	11
6.1	Rinse cleaning bowls, lids incl. seals and cleaning pins	12
6.2	Disinfection	12
6.3	Cleaning the appliance and the retainer magnet	12
6.4	Storage	12
6.5	Adding cleaning pins	13
6.6	Replacing cleaning pins	13
6.7	Cleaning beaker	13
6.8	Changing a fuse	13
6.9	Spare Parts	13
7	Troubleshooting	14
8	Technical Data	14
9	Guarantee	15
10	Disposal instructions	15
10.1	Disposing of the Unit	15
10.1.1	Disposal instructions for countries in the EU	15

1 Introduction

We are pleased with your decision to purchase the SYMPRO denture cleaning unit.

This appliance is to be used exclusively for the professional cleaning of removable dentures as well as orthodontic appliances and splints.

This appliance sets new standards of functionality, performance, and ergonomics.

Please read the following operating instructions carefully and observe the included safety and hygiene instructions to ensure of long and problem-free function.

1.1 Symbols

In the instructions for use and on the unit itself you will find these symbols with the following meanings:

Danger

This indicates a direct risk of injury. Consult accompanying documents!

Electrical current

This indicates a risk of hazard due to an electrical current.

Attention

Disregarding this warning may result in damage to equipment.

Note

This provides the operator with useful information to improve and ease use.

Follow the instructions for use!

The operating instructions can be found in electronic format on our website.

Visit www.renfert.com/p915. Enter the following item number: 67001000.

Please observe the operating time.

After no more than 15 minutes of operation, you must pause for at least 30 minutes.

Permitted humidity range for transport and storage temperature. Refer to Chapter 2.4.

The EU regulations applicable to the product are adhered to.

The device is subject to the EU directive 2002/96/EG (WEEE directive).

Manufacturer

Date of manufacture

Medical device

To be used by specialist staff.

► List, particular attention should be paid

- List
- List

⇒ Instructions / appropriate action / input / operational sequence:

You will be asked to carry out the action in a specified order.

- ◆ Result of an action / reaction of the device / reaction of the program:

The unit or program reacts as a result of your actions or when a specific incident occurs.

Other symbols are explained as they occur.

1.2 EU Conformity

Renfert GmbH hereby declares that this product:

SYMPRO

Item number: 6700 1000

complies with the following European regulations:

Regulation (EU) 2017/745

2 Safety

In case of complaints and events in connection with the medical device, please inform Renfert immediately. Renfert will investigate the matter and report serious incidents to the authorities in the Member States concerned.

en

2.1 Intended Use

This appliance is to be used exclusively for the professional cleaning of removable dentures as well as orthodontic appliances and splints.

The appliance is designed for non-sterile use only.

2.2 Improper Use

Only spare parts and accessories supplied or authorized by Renfert GmbH may be used with this product. If other spare parts or accessories are used, this could have a detrimental effect on the safety of the device, increase the risk of serious injury and lead to damage to the environment or the device itself. In particular, only use cleaning agents distributed by Renfert GmbH. If any other cleaning agents are used, the results of the cleaning cannot be guaranteed and this may damage parts of the unit.

2.3 Ambient Conditions for Safe Operation

The device may only be operated:

- Indoors,
- Up to an altitude of 2,000 m above sea level,
- At an ambient temperature of between 5 - 40 °C [41 - 104 °F] *),
- At a maximum relative humidity of 80 % at 31 °C [87.8 °F], dropping to a linear of up to 50 % relative humidity at 40 °C [104 °F] *),
- With mains power where the voltage fluctuations do not exceed 10 % of the nominal value,
- Under contamination level 2 conditions,
- Under over-voltage category II conditions.

*) Between 5 - 30 °C [41 - 86 °F] the device can be operated at a relative humidity of up to 80 %. At temperatures between 31 - 40 °C [87.8 - 104 °F] the humidity must decrease proportionally in order to ensure operational readiness (e.g. at 35 °C [95 °F] = 65 % humidity, at 40 °C [104 °F] = 50 % humidity). The device may not be operated at temperatures above 40 °C [104 °F].

2.4 Ambient Conditions for Storage and Transport

For storage and transport the following specifications to ambient conditions apply:

- Ambient temperature - 20 – + 60 °C [- 4 – + 140 °F].
- Maximum relative humidity 80 %.

2.5 Hazard and Warning Information

2.5.1 General Information

- ▶ If the device is not used in compliance with the supplied instructions, the safety of the device can no longer be guaranteed.
- ▶ Only to be used indoors. The device is only intended for dry operation and should not be used or stored outdoors or in wet conditions.
- ▶ The device may only be operated using a mains cable with the country-specific plug system. Any necessary alterations must be carried out by a qualified electrician.
- ▶ The device may only be operated if the information on the identification plate conforms to the specifications of your local mains power supply.
- ▶ The device may only be plugged into outlets which are connected to the protective conductor system.
- ▶ The mains plug must be easily accessible.
- ▶ Disconnect the device from the mains before carrying out work on the electrical parts.
- ▶ Pull out the mains plug to disconnect the equipment from the mains supply.
- ▶ Check connection cables (such as power supply cords), tubes and housing (i.e. the key-pad) regularly for damage (i.e. kinks, cracks and porosity) or signs of ageing. Devices with damaged connection cables, tubes or housing parts or other defects must not be operated!
- ▶ Defective devices must be put out of service immediately. Remove the mains plug and ensure the device is not used. Send the device for repair!
- ▶ Only operate the device under supervision.

- ▶ **Risk of injury!**
Risk of injury if the components used are not approved.
Always use original Renfert components.
- ▶ Please observe the national accident prevention regulations!
- ▶ It is the responsibility of the operator that national regulations during operation and regarding a repeated safety inspection of electrical equipment are complied with. For Germany these are the regulation 3 by DGUV (German Statutory Accident Insurance) in relation with VDE 0701-0702 (Association for Electrical & Electronic Technology).
- ▶ Information on REACH and SVHC is available on our website www.renfert.com, in the Support area.

2.5.2 Specific Information

- ▶ Cleaning pins contain a slight amount of nickel (see product data sheet).
- ▶ Rinse the object under running water after cleaning and disinfect.
- ▶ Check the object after cleaning and remove any adhering cleaning pins.
- ▶ Contact with cleaning fluids or prolonged inhalation of their vapours may cause irritation. Use appropriate personal protective gear (protective goggles, gloves). Please note the information contained within the Material Safety Data Sheet.
- ▶ Please do not let the cleaning beaker filled with cleaning fluid in the unit for a long period of time (e.g. overnight).
- ▶ Damage or discolouration may occur should the cleaning liquid come into contact with other objects or laboratory furniture.
This is valid particularly for surfaces out of PVC, linoleum, marble, natural stone, artificial stone, stainless steel and wooden surfaces.
In case of contamination, immediate treat with a copious amount of water.
- ▶ Increased cleaning times in the unit may cause the cleaning liquid to heat up. Use tweezers only to remove the cleaning object.
- ▶ The appliance contains strong magnets!
People with cardiac pacemakers should keep 50 cm from the appliance.
Do not place data storage mediums, credit cards or watches in the vicinity.
- ▶ The retaining magnet (Fig. 1, Pos. 5) contains lead

2.6 Information about electromagnetic compatibility (EMC)

Information about electromagnetic compatibility (EMC) can be found in a separate section at the end of these instructions.

2.7 Authorized Persons

Operation and maintenance of the device may only be performed by qualified personnel.

Any repairs not specifically described in these operating instructions may only be carried out by a qualified electrician.

2.8 Disclaimer

Renfert GmbH is not liable for claims for compensation or claims under guarantee if:

- ▶ the object to be cleaned will be damaged by incorrect treatment.
- ▶ the product is used for purposes other than those stated in the operating instructions.
- ▶ the product is modified in any way – apart from modifications described in the operating instructions.
- ▶ the product has not been repaired by a specialist firm or original Renfert replacement parts have not been used.
- ▶ there is continued use of the product despite obvious safety defects or damage.
- ▶ the product has been subjected to mechanical knocks or has been dropped.

3 Product Description

3.1 Components and Functional Elements

- | | |
|--|---|
| 1 SYMPRO Denture cleaning unit | 8 Cleaning intensity selection button |
| 2 Disinfectable cup (white)
help:ex plaque f/p
for use in the SYMPRO | 9 + / - buttons, change cleaning time |
| 3 Plastic cup with lid (gray), help:ex discolor f
for use outside of the SYMPRO | 10 Start / pause button |
| 4 Tweezers | 11 Stop button |
| 5 Retaining magnet | 12 Mains switch |
| 6 Cleaning time indicator | 13 Mains connection |
| 7 Cleaning intensity | 14 Fuse |
| | 15 Mains cable |
| | 16 SYMPRO min-cup with adapter ring
(optional accessory) |

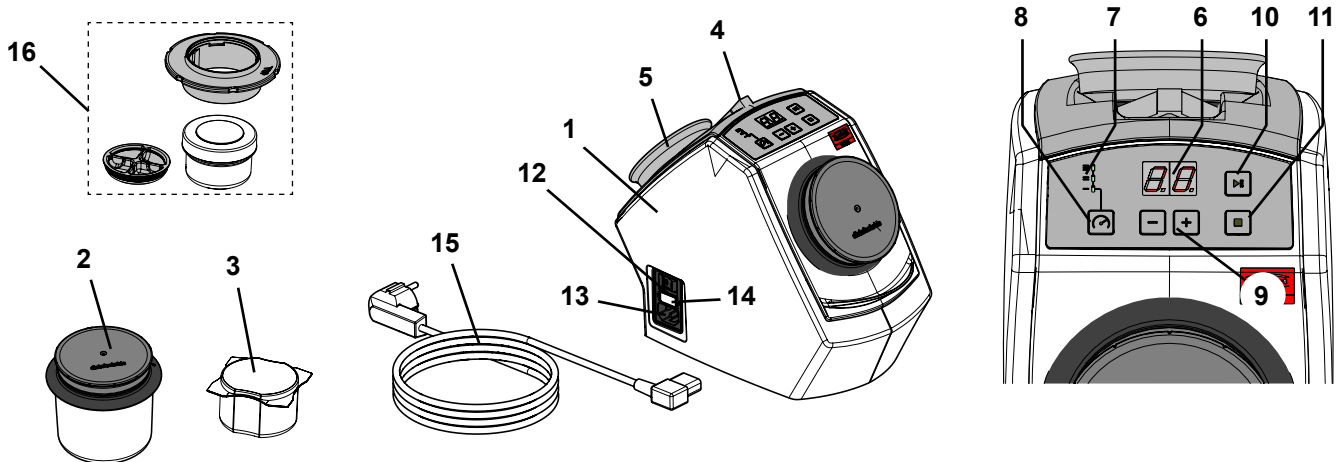
Fig. 1

3.2 Scope of Delivery

- 1 SYMPRO Denture cleaning unit
- 1 Disinfectable cup (white) help:ex plaque f/p for use in the SYMPRO
- 1 Plastic cup with lid (gray), help:ex discolor f for use outside of the SYMPRO
- 1 Cleaning pins (container, 75 g)
- 1 Tweezers
- 1 Retaining Magnet
- 2 Power cord (design depends on the unit's item number)
- 1 Instruction manual
- 1 Guidelines for operation and hygienic preparation

3.3 Accessories

- 6500 0450 Disinfectable cup (white) help:ex plaque f/p for use in the SYMPRO
- 6500 0460 SYMPRO disinfectable beaker set
- 1850 0007 Plastic cup with lid (gray), help:ex discolor f for use outside of the SYMPRO
- 6500 0400 SYMPRO mini-cup
- 6500 0410 SYMPRO mini-cup set
- 6500 0550 Cleaning pins 75 g
- 6700 0100 help:ex plaque f (cleaning agent – liquid 4 x 1 L)
- 6700 0000 help:ex plaque p (cleaning agent – powder 20 packets)
- 6700 0200 help:ex discolor f (cleaning agent – liquid 1 L)
- 6600 0000 temp:ex Solvent for temporary cement 250 ml

4 Setting Up

4.1 Unpacking

- ⇒ Remove the device and all the accessories from the delivery package.
- ⇒ Check the delivery for completeness (refer to Sec. 3.2).

4.2 Setup

- ⇒ Place the device on an acid-resistant and non-slip work surface; if necessary use a suitable support, in a comfortable working height.
- ⇒ Place the tweezers supplied with the unit and the retaining magnets in the holder on the back of the appliance (Fig. 1).

Danger of injury through falling equipment!

Remove the retaining magnets from the holder when transporting the appliance.

4.3 Electrical Connection

Ensure the mains supply and voltage marked on the nameplate is identical.

- ⇒ Choose a power cable (15, Fig. 1) with the proper plug for your mains supply.
- ⇒ Connect the power cable (13, Fig. 1) to the power supply.
- ⇒ Insert the power plug into the building installed wall socket.
- ⇒ Turn on the appliance at the mains switch (12, Fig. 1).

4.4 Hygienic Preparation







Please note:



Before using the SYMPRO system for the first time, prepare the components hygienically according to the following instructions.

- ⇒ Fill the cleaning beaker (2, Fig. 1) with 75 g cleaning pins (corresponds to one packaging unit).
- ⇒ Fill the SYMPRO mini cup (accessory) (16, Fig. 1) with ~ 5 g cleaning pins

Observe personal protection as understood under national practice, labor and hygiene regulations prior to handling disinfectant, the cleaning object and/or any of the system components of the cleaning unit.

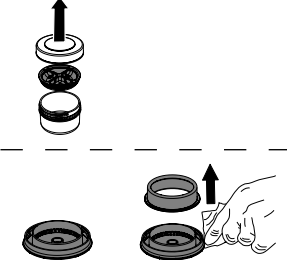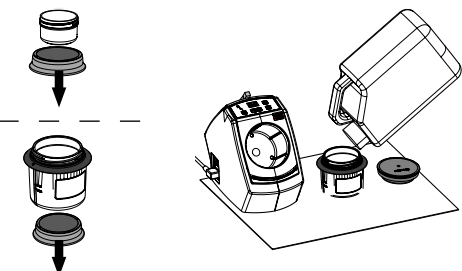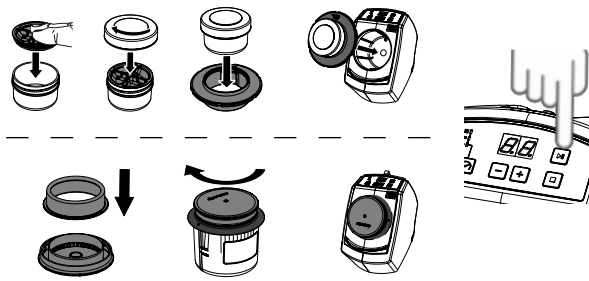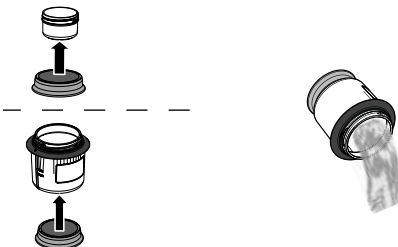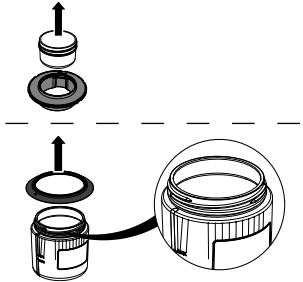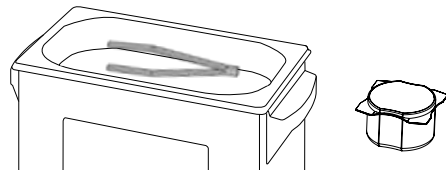		Manual disinfection:
D1		Disinfection process – lid & seal Open the cleaning beaker. Remove the silicone seal from the lid. Thoroughly rinse both objects (cover and seal) under running water. Wipe the entire lid (outer and inner surfaces including the thread) and the silicone seal (outer and inner surface) with a wipe disinfectant (*). (*) Recommendation: CaviWipes, Metrex
D2		Fill the cleaning beaker with disinfection liquid Remove the cleaning beaker magnets (not applicable for first time use). Fill the cleaning bowl (incl. cleaning pins) with an instrument disinfection fluid for heavy organic contamination (*). For fill quantity, see Application Table. (*) Recommendation: Sporicidin, Contec / Metricide OPA plus, Metrex
D3		Disinfection process – Cleaning the pins & inside the beaker Replace the disinfected silicone seal back into the lid or in the case of the SYMPRO mini-cup, back into the cup. On the control panel, set a runtime of at least 10 minutes for higher intensity (*) and start the disinfection process. (*) These values apply to the disinfection fluids recommended in D2 and in the Application Table. Other settings may be required for other types of liquid or concentrations.
D4		Disposal of disinfection liquid After the end of the program, remove and open the cleaning beaker, keep the cleaning pins with the help of the magnet, on the bottom of the cleaning beaker, pour out the disinfectant and dispose of according to the manufacturer's instructions.
D5		Disinfection process – Cleaning the outside of the beaker and & sound protection ring Remove magnets. Remove the silicone sound-proof ring / adaptor ring from the cleaning bowl. Wipe the surface of the cleaning bowl (incl. thread and sound-proof ring / adaptor ring) with a standard wipe disinfectant (*). (*) Recommendation: CaviWipes, Metrex
D6		Disinfection process – tweezers / Plastic cup with lid (gray), help:ex discolor f After use, place the tweezers and vessel into an instrument bath with commercially available highlevel instrument disinfection liquid (*) for a high degree of organic soling and immerse according to the manufacturer's instructions. (*) Recommendation: Sporicidin, Contec / Metricide OPA plus, Metrex

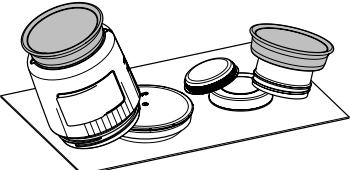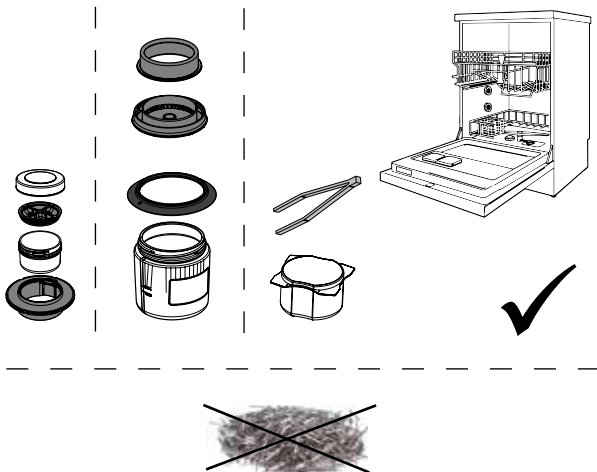		Manual disinfection:
D7		Next cleaning process / Storage The cleaning beaker and cleaning pins are now ready for use again. If the next use is not carried out immediately, the cleaning pins must be held on the underside of the cleaning beaker with the help of the magnet. Then, place the cleaning beaker upside down on its lid so that it can drip dry. It is advisable to place a paper towel underneath to absorb the moisture.
		Machine disinfection:
D8		Machine thermodisinfection of the cleaning beakers (incl. lid WITHOUT pins): The cleaning beakers, lids, silicone seals, sound-proof ring / adaptor ring, and tweezers can be thermally cleaned and disinfected in a certified cleaning and disinfecting device, according to the "Hygiene requirements for the treatment of medical Devices" (*). For this purpose, all listed components must be placed in the cleaning and disinfecting unit in a dismantled state. i <i>Take care when placing light parts appropriately (e.g. in the instrument basket). The cleaning pins must only be disinfected manually! Observe points D3 to D5 and store the disinfected cleaning pins in a suitable container (for example cleaning cups, spare parts) only for the duration of the cleaning process. Finally, for longer periods of storage, see point D7.</i> (*) Recommendation of the Commission for Hospital Hygiene and Infection Prevention (KRINKO) at the Robert Koch Institute (RKI) and the Federal Institute for Drugs and Medical Devices (BfArM).

4.5 Application Table

(For detailed description see from chapter 5)

Filling	65000450 SYMPRO disinfectable beaker (2, Fig. 1)	65000410 SYMPRO mini-cup (16, Fig. 1)
Amount of cleaning pins	75 g (1 pck.)	~ 5 g
Fill level min.	~ 200 ml (MIN marking)	10 ml
Fill level max.	MAX marking – do not exceed	Fill bowl to max. 50 %
Max. no. objects	2 complete sets of dentures or 1 misc. restoration	4 single crowns or up to 4-unit bridge

4.6 Application parameters Cleaning agent

Recommended Application Parameters Contamination / Cleaning liquids			SYMPRO disinfectable beaker SYMPRO mini-cup set	Recommended application time (Depending on the degree of soiling)
			Intensity 	
Disinfection procedure These values apply to the disinfecting liquids listed. For other products or different concentrations, other settings may have to be used.	Bacteria and viruses	(*) Sporidicin (USA) Metricide OPA plus (USA) BIB forte eco 4 % (EU)	 High intensity	10 min.
	Bacteria and viruses	(*) Cavi Wipes (USA) Minuten Wipes (EU)	n / a	Use wipe disinfection according to manufacturer's instructions
Cleaning objects with finely detailed areas and delicate functional elements. (We recommend covering especially sensitive areas)	Stubborn stains such as plaque, tartar, discoloration (tea, coffee, red wine etc.) and denture adhesive	help:ex plaque f **	 Low intensity	10 - 15 min. We recommend checking after about 5 min.
		help:ex plaque p **	 n / a	
	Tough nicotine / tar stains and discoloration	help:ex discolor f	Insert into Plastic cup with lid (gray), help:ex discolor f	10 - 15 min. We recommend checking after about 5 min.
Cleaning objects made partially of stainless steel and orthodontic restorations. (We recommend covering especially sensitive areas).	Stubborn stains such as plaque, tartar, discoloration (tea, coffee, red wine etc.) and denture adhesive	help:ex plaque f **	 Medium intensity	10 - 12 min. We recommend checking after about 5 min.
		help:ex plaque p **	 n / a	
	Tough nicotine / tar stains and discoloration	help:ex discolor f	Insert into Plastic cup with lid (gray), help:ex discolor f	10 - 15 min. We recommend checking after about 5 min.
Cleaning objects made from NEM, total dentures, model cast frameworks and splints. (We recommend covering especially sensitive areas)	Stubborn stains such as plaque, tartar, discoloration (tea, coffee, red wine etc.) and denture adhesive	help:ex plaque f **	 High intensity	10 - 30 min. We recommend checking after about 5 min.
		help:ex plaque p **	 n / a	
	Tough nicotine / tar stains and discoloration	help:ex discolor f	Insert into Plastic cup with lid (gray), help:ex discolor f	10 - 15 min. We recommend checking after about 5 min.
Temporarily cemented restorations	Dental temporary cement zinc - oxide based (with / without eugenol)	temp:ex	 High intensity	5 - 10 min. depending on the degree of soiling

(*) Recommendation: Sporidicin, Contec or Metricide OPA plus, Metrex

(**) help:ex plaque f – ready-to-use cleaning liquid
Help:ex plaque p – cleaning powder and neutralizer

5 Operation

5.1 Preparing the object

Inspect the object for unstable areas or damage before cleaning it in the appliance. These areas may be affected by chemical and mechanical stress during cleaning.

Delicate areas and function elements (such as thin crown margin, electroplated parts, friction surfaces of external telescopes) may be damaged.
As a rule, these will have to be covered with an appropriate wax / silicone layer.

i **High-gloss areas may become dulled through cleaning. In these cases, we recommend a post-polishment. If necessary, cover the high-gloss areas with an appropriate wax / silicone layer before cleaning.**

If there are any denture adhesive cream deposits on the objects to be cleaned, they must be removed with a dry cleaning cloth prior to the cleaning process.

5.2 Filling the cleaning vessel

For use in the SYMPRO denture cleaning unit.

⇒ Select the cleaning cup and cleaning agent suitable for the cleaning task (see table 4.5 and 4.6).

⇒ Fill the cleaning bowl with:

- the cleaning pins.
- objects to be cleaned
- Cleaning agents (Please refer to the instructions for use in the help:ex operating manuals and safety data sheets)

⇒ Close the cup completely (ensure that the lid / seal is correctly in place) and insert it into the unit.

Never operate the unit with the cleaning beaker open!

Only use the cleaning fluid once!

The object to be cleaned could cause a bacterial contamination of the cleaning fluid. Therefore, the cleaning fluid must be replaced for each new cleaning process.

5.3 Select the cleaning time and intensity

⇒ Adjust the cleaning time with the + / - buttons (9, Fig. 1) (range 0 to 30 minutes).

⇒ Use the selection button (8, Fig. 1) to select / set the intensity.

Refer to Table in Chapter 4.6 for the recommended intensity and cleaning time.

The cleaning time in the unit depends on the cleaning object, the set cleaning intensity and the type and degree of soiling. We recommend checking in between, especially for sensitive cleaning objects and if you do not have much experience in working with the unit.

In the case of particularly stubborn residues, an extended running time (+ max. 10 min.) and short, manual cleaning with a hard toothbrush under running water can help to further improve the cleaning results.

Never open the cleaning beaker in the device!

5.4 Starting, pausing, and stopping cleaning

⇒ Start cleaning with the Start / Pause button (10).

♦ The appliance stops automatically when the set cleaning time has elapsed.

♦ The remaining time is displayed in the indicator during cleaning.

• Cleaning can be stopped and restarted by pressing the Start / Pause button.

• Once cleaning is stopped, the remaining cleaning time can be adjusted.

• The intensity can be modified at any time during the cleaning process.

• Cleaning can be stopped at any time before the cleaning time has elapsed by pressing the Stop button (11).

♦ After the cleaning time has elapsed the indicator shows „00“.

⇒ Stop cleaning altogether by pressing the Stop button (11).

♦ The display shows the previously set cleaning time.

To improve the cleaning performance, the sense of rotation will change every 30 seconds.

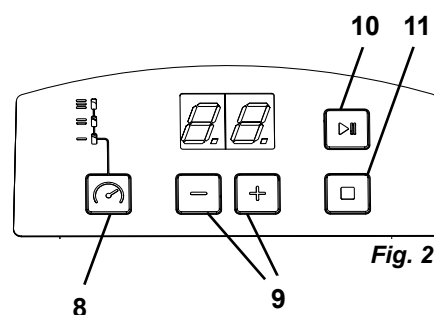
Fig. 2

5.5 Remove the cleaning object

After cleaning:

⇒ Remove the cleaning vessel and open the lid while holding over a washbasin or sink unit.

Never open the cleaning beaker in the device!

⇒ Place the holding magnets on the outside of the beaker so that the cleaning pins collect on the inner wall of the beaker.

⇒ The object can now be easily removed using the supplied tweezers.

Carefully rinse the cleaned object with water for at least 5 seconds and disinfect it according to the guidelines defined in national hygiene regulations before returning it to the patient.

Check the object carefully for any cleaning needle residues before handing back to the patient or inserting into the patient's mouth.

Use the cleaning fluid only once!

Bacterial contamination can occur when the object is placed into the cleaning liquid. Therefore, the cleaning liquid must be replaced for each cleaning operation.

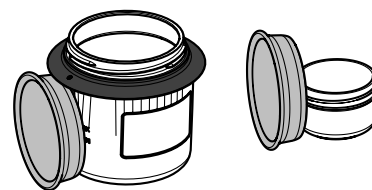
Fig. 3

5.6 Optional intensive cleaning with help:ex discolor f

If tough residue, such as nicotine / tar stains, still remain after the cleaning process with help:ex plaque (f or p), we recommend subsequent cleaning with the special cleaner help:ex discolor f. To do this, completely cover the cleaning object in the **plastic cup with lid (gray), help:ex discolor f** (Fig. 4). After an immersion time of 5 - 10 mins (without the use of cleaning pins and the cleaning device!) the cleaned object can be taken out using plastic tweezers. Remove any residue by brushing under warm running tap water with a toothbrush or denture cleaning brush.

Fig. 4

Carefully rinse the cleaned object with water for at least 5 seconds and disinfect it according to the guidelines defined in national hygiene regulations before returning it to the patient.

Use the cleaning fluid only once!

Bacterial contamination can occur when the object is placed into the cleaning liquid. Therefore, the cleaning liquid must be replaced for each cleaning operation.

5.7 Disposal of cleaning fluid

After cleaning in the SYMPRO denture cleaning unit, **first**, with the aid of a magnet, hold the cleaning pins on the bottom of the cleaning cup (Fig. 5) and then carefully pour the cleaning liquid into a storage tank (Fig. 6). We recommend empty help ex containers 1000 ml made of polypropylene (PP), for example..

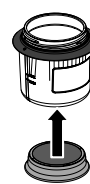
Fig. 5

Fig. 6

Hold the cleaning pins back by holding the magnet against the cleaning beaker.

Dispose of according to national disposal regulations.

More information can be found in the cleaning liquid's operating manual and safety data sheet.

Since these are corrosive cleaning fluids, please wear personal protective equipment!

Remove residues of cleaning liquids on working surfaces immediately with a damp cloth and rub dry with a dry cloth.

6 Cleaning / Maintenance

This device does not contain any parts that require maintenance. Opening the device other than described in the instructions is not permitted!

The operator and user are responsible for the correct cleaning and disinfection of all system components (Cleaning device, cleaning bowl incl. lid and seals, sound-proof ring, adaptor ring, cleaning pins, Nicoclean cleaning bowls, tweezers) after each cleaning process and for compliance with the national hygiene regulations.

6.1 Rinse cleaning bowls, lids incl. seals and cleaning pins

Fill the SYMPRO cleaning beaker up to the marked "MAX" fill level with clean water. Close the lid and clean the inside of the beaker by shaking repeatedly and intensely. When using the SYMPRO mini-cup, fill the cup to the brim under running water.

Use the magnet to fix the cleaning pins to the base of the SYMPRO beaker and pour out the water. This operation must be repeated at least 3 times with fresh water.

Then rinse the lid under running water. Check visually whether the deposits have been removed.

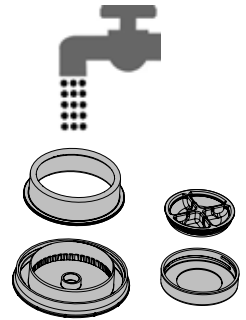
Fig. 7

6.2 Disinfection

For the manual or machine preparation of the system components see chapter „4.4 Hygienic Preparation“

- Manual: D1 - D7
- Machine (in the thermodisinfector): D8

The device is designed for use in non-sterile areas.

The following system components are thermostable up to 121°C:

- Cleaning cup (white) incl. lid,
- Seal and sound-proof ring,
- Cleaning pins.

(*) Recommendation:

Germany: Minute Wipes, ALPRO Medical GmbH
USA: CaviWipes, Metrex

6.3 Cleaning the appliance and the retainer magnet

Never clean the device under running water or immerse under water.

Do not steam clean.

DO NOT use solvent-containing (e.g. acetone), aggressive or abrasive cleaners.

To clean the appliance and the retainer magnet wipe only with a damp cloth.

Wipe the drip collector (Fig. 8) at least once a day.

6.4 Storage

If the device is not used again immediately, the cleaning pins must be held on the underside of the cleaning beaker with the help of the magnet. Then, place the cleaning beaker upside down on its lid so that it can drip dry.

It is advisable to place a paper towel underneath to absorb the moisture.

If the device is not used for a longer period of time (e.g. vacation), it is recommended to carry out hygienic preparation of the system components before use (see chapter 4.4).

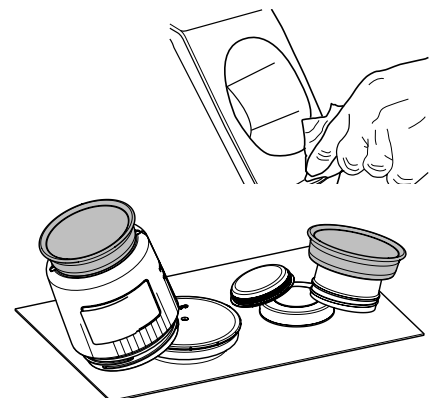
Fig. 9

6.5 Adding cleaning pins

Cleaning the pins or removing the denture from the cleaning beaker can result in a loss of cleaning pins. This reduces the cleaning performance, which is why it is advisable to check the quantity at regular intervals.

To restore the intended quantity of 75 g of cleaning pins, place the dry cleaning beaker, including dry pins, on some scales and add new pins to the total weight of 145 g (gross weight) as required.

6.6 Replacing cleaning pins

Replace the cleaning pins when the cleaning and polishing effect starts to subside.

To do this:

- ⇒ Place the holding magnet on the underside of the beaker and gently drain the cleaning fluid. For disposal instructions (see chapter „5.7 Disposal of cleaning fluid“).
- ⇒ Rinse the cleaning beaker and pins to be disposed of with water (see chapter 6.1) disinfect (see chapter 4.4).
- ⇒ Remove the retaining magnet and tip out the old pins.
- ⇒ The cleaned and disinfected cleaning pins can be disposed of in the regular waste.
- ⇒ Rinse out the cleaning vessel.
- ⇒ Fill in new cleaning pins.

When using the SYMPRO mini-cup, ~ 5 g of pins can be conveniently dosed by filling a quarter of the cup seal.

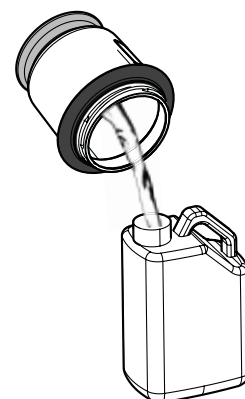
Fig. 10

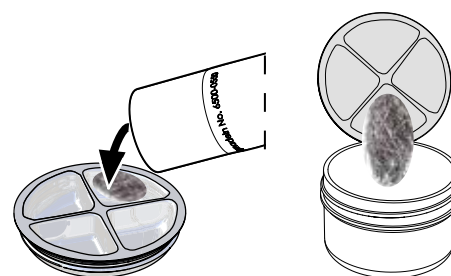
Fig. 11

6.7 Cleaning beaker

The SYMPRO cleaning cup, lid, incl. the silicone seals, adapter ring, sound-proof ring and gray cleaning cup are temperature resistant up to 121 °C.

Cleaning cup, lid, seal and cleaning pins are subject to wear. For this reason, check integrity of these parts before use. In order to grant a safe reprocessing procedure, change cleaning vessel, cover and joint presenting visible signs of wear (e.g. with heavy discoloration, leakage, scratches and perforation caused by cleaning pins).

6.8 Changing a fuse

Before replacing a fuse, disconnect the appliance from the mains.

Never use a fuse of larger impedance.

The fuse is found next to the mains connection (14, Fig. 1).

- ⇒ Disconnect from mains supply.
- ⇒ Unlatch the fuse holder on both sides and remove.
- ⇒ Remove faulty fuse and replace with a new one.
- ⇒ Replace the fuse holder by sliding back into the socket until both sides lock into place.

6.9 Spare Parts

You can find components subject to wear and the spare parts on the spare part list in the internet at www.renfert.com/p918.

Enter the following item number: 67001000

The components excluded from the warranty (such as consumables or parts subject to wear and tear) are marked on the spare part list.

Serial number and date of manufacturing are shown on the type plate of the unit.

7 Troubleshooting

Trouble	Possible Cause	Corrective Action
No display.	• Appliance not connected to mains. • Appliance not turned on. • No power from the socket. • Faulty fuse.	• Check correct fit of power cable in the appliance socket and wall socket. • Turn on appliance at the mains switch (12, Fig. 1). • Check power socket, for example, with another appliance. • Replace the fuse (see chapt. 6.9). If the fuse blows repeatedly, have the appliance inspected by a service technician.
Cannot change cleaning time.	• Appliance still running.	• Interrupt the cleaning process with the Start / Pause button, and then adjust cleaning time.
Cannot save cleaning time.	• Cleaning time was entered during an interrupted cleaning process.	• Stop the cleaning process with the Stop button, and then enter new cleaning time.
Insufficient cleaning performance.	• Cleaning time too short. • Active ingredients of the cleaning fluid spent. • Insufficient number of cleaning pins. • Cleaning pins spent. • Incorrect cleaning fluid used.	• See chapt. 5.4. • Prepare fresh cleaning fluid after each application. • Fill up the cleaning pins (see chapt. 6.6). • Replace the cleaning pins (see chapt. 6.7). • See chapt. 4.5, Application Table.
Cleaning pins have corroded.	• Cleaning pins have been insufficiently rinsed. • Cleaning pins have been stored incorrectly. • Cleaning pins in use for longer period of time.	• Rinsing and storage see chapt. 6.1. • Removing the corrosion: Clean pins with 200 ml help:ex plaque (f or p) without the object for 5 minutes at a high intensity • The pins are so-called consumable goods. If it is not possible to regenerate the pins, or the pins' performance deteriorates, see chapt. 6.7.

8 Technical Data

Working Voltage:	230 V
Permissible mains voltage:	100 - 240 V
Mains frequency:	50 / 60 Hz
Power consumption:	90 VA
Operating mode:	Not for permanent operation! After two mixing processes in succession, ensure that there is a pause of 7 minutes.
Mains power fuse:	T 1.6 AL / 250 V
Cleaning intensity:	Low / medium / high
Noise level:	< 70 dB(A)
Dimensions (width x height x depth):	150 x 240 x 280 mm [5.9 x 9.5 x 11.0 inch]
Weight (without vessel) approx.:	3 kg [6.6 lbs]

9 Guarantee

Used according to manufacturer's instructions, all parts of the SYMPRO carry a **3-year guarantee**.

The original sales invoice of the specialist depot is required for a claim under guarantee.

Parts which are subject to natural wear and tear (wear parts) and consumables are excluded from the guarantee. These parts are marked in the spare part list.

The guarantee is rendered void in the case of incorrect use, nonadherence to the operating, cleaning, servicing or connection instructions, repairs carried out by the owner or repairs that are not carried out by a specialist firm, use of replacement parts from another manufacturer or unusual or unapproved uses not specified in the user guidelines.

Successful claims under guarantee do not extend the guarantee period.

Full operational safety can only be guaranteed if RENFERT system components (cleaning appliance, pins, vessels, fluids) are used exclusively.

en

10 Disposal instructions

10.1 Disposing of the Unit

The appliance should be disposed of by a specialist firm. The specialist firm should be informed of any hazardous residue in the appliance.

Crushing hazard!

The unit contains strong magnets! Wear protective gloves for disassembly.

Risk of splitter!

The magnets are brittle. Wear protective glasses for disassembly

10.1.1 Disposal instructions for countries in the EU

To conserve and protect the environment, prevent environmental pollution and improve the recycling of raw materials, the European Commission adopted a directive that requires the manufacturer to accept the return of electrical and electronic units for proper disposal or recycling.

Within the European Union appliances with this symbol should not therefore be disposed of in unsorted domestic waste.

For more information regarding proper disposal please apply at your local authorities.

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2 Information about electromagnetic compatibility (EMC)

2.1 General information

- ▶ The product/equipment was tested in accordance with DIN EN 60601-1-2.
- ▶ The product/equipment has no essential performance characteristics as per the definition in IEC 60601-1-2.
- ▶ The product / equipment satisfies the noise emission and immunity requirements to operate in a domestic healthcare environment.
- ▶ The product / equipment satisfies Class B according to CISPR11:
The products / equipment can be used in areas that are directly connected to the public mains supply, and supply buildings that are also used for residential purposes.
- ▶ When using the product / equipment in a SPECIAL ENVIRONMENT, its function may not be guaranteed nor can it be ensured that it satisfies the ESSENTIAL PERFORMANCE CHARACTERISTICS.
- ▶ SPECIAL ENVIRONMENTS may include, for example:
 - Military areas (e.g. a submarine, close to radar equipment or weapons control systems)
 - Areas of heavy industry
 - Medical treatment areas with high power medical electrical (ME) systems (e.g., RF surgical equipment, magnetic resonance imaging equipment).
- ▶ Operating in an PROFESSIONAL HEALTH CARE FACILITY ENVIRONMENT is permitted provided an adequate distance is kept to the high power ME systems.
- ▶ The product / equipment must not be placed and operated directly next to other electrical equipment. If this is unavoidable, the function of the product/equipment and the other electrical equipment should be checked to rule out any malfunctioning caused by electromagnetic interference.
- ▶ The use of accessories other than those provided for in the operating instructions, however they may have an adverse effect on the EMC properties.
- ▶ Stacking products/equipment is not permitted.
- ▶ Portable and mobile communication devices (e.g. cell phones, radio equipment) can affect the functioning of the product/equipment.
Maintain a distance of at least 30 cm (12 inches) between the product/equipment and mobile communication devices.

2.2 Interference Emission and Immunity Levels

2.2.1 Maintaining the Interference Emission Level

Interference Measurement	Compliance
Conducted interference (interference voltage) CISPR11	ClassB Limit values are complied with.
Radiated interference variable (noise field intensity) CISPR11	Class B Limit values are complied with.
Distortion due to harmonics IEC 61000-3-2	Class A Limit values are complied with.
Voltage fluctuations and flicker IEC 61000-3-3	Satisfied

2.2.2 Maintaining the Interference Immunity Level

Interference Immunity Test	Standard / Test Level	Match Level
Static electricity discharge	IEC 61000-4-2 Air discharge 8 kV maximum Contact discharge: 4 kV maximum	8 kV 4 kV
High-frequency electromagnetic fields 80 MHz - 2.7 GHz	IEC 61000-4-3 10 V/m	10 V/m
High-frequency electromagnetic fields (near field) 380 MHz - 5.8 GHz	IEC 61000-4-3 28 V/m maximum	No interference
Electrical fast transients (burst)	IEC 61000-4-4 2 kV	2 kV
Surge voltages (surge)	IEC 61000-4-5 2 kV	2 kV
Conducted disturbances 150 kHz - 80 MHz	IEC 61000-4-6 10 V	10 V
Magnetic fields with energy frequency 50 Hz	IEC 61000-4-8 30 A/m	30 A/m
Mains voltage dips and short interruptions	IEC 61000-4-11 Test voltage / exposure time 70 % / 0.5s 0 % / 0.02s 0 % / 0.01s 0 % / 5s	The response to the mains voltage dips and short interruptions is device specific. The behavior of the product / equipment in these scenarios is specified in the operating instructions for the product.

2.3 Unknown EMC Environment

Avoid operation in the vicinity of stationary transmitters such as base stations of radio telephones, land mobile radios, amateur radio stations, AM and FM broadcasting and television transmitters.

When operating in this kind of environment, pay attention to the behavior of the product / equipment.

Consider changing the location if there are any unusual functional limitations or reduced performance characteristics.

The field strength measured at the site should not exceed the above referenced compliance levels.

Hochaktuell und ausführlich auf ...
Up to date and in detail at ...
Actualisé et détaillé sous ...
Aggiornato e dettagliato su ...
La máxima actualidad y detalle en ...
Актуально и подробно на ...

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Renfert GmbH • Untere Gießwiesen 2 • 78247 Hilzingen/Germany