Harvard TEMP Cem

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Instructions for Use EN

Non-eugenol temporary cement

Harvard TEMP Cem is a non-eugenol temporary crown and bridge cement suitable for cementing temporary crowns or bridges as well as for trial cementing of permanent restorations. Harvard TEMP Cem shows excellent adhesion to temporary crown and bridge materials, but can be easily removed from prepared teeth. Harvard TEMP Cem is delivered in Minimix syringes, so it is very easy and comfortable to use.

Due to its non-eugenol formulation it does not influence the setting of resin-based filling and luting materials. The temporary crown and bridge cement is based on zinc oxide and organic acids. Especially in cases of allergies to eugenol it is strongly recommended to use this eugenol-free type of cement.

Contraindications

If the patient is known to be allergic to any of the ingredients.

Side effects

In singular cases, Harvard TEMP Cem may cause a sensitizing reaction in patients with a hypersensitivity to any of the ingredients. In these cases, the material should not be used.

Application

1. Preparing the minimix syringe

Remove the cap of the minimix syringe and throw it away (**do not use it again!**). Attach a 1:1 mixing tip. Make sure that the guidance of the minimix syringe is aligned with that of the mixing tip and turn the tip 90° clockwise until it locks in position. The material is now ready for application.

Note:

Discard the first 2-3 mm of the extruded material. This must be done for each new mix.

Leave the used mixing tip on the minimix syringe. It serves as a cap.

2. Application

Apply Harvard TEMP Cem directly into the temporary crown or bridge under slight pressure.

Apply a thin layer to the inside of the temporary crown or bridge and place the temporary restoration within **1:30 minutes** (working time at 23 °C/74 °F) cautiously but firmly over the prepared tooth structures.

Note:

Do not dry stumps, best moisten them with some water. A humid milieu (humid stumps) additionally facilitates to remove the temporary crown or bridge later on.

3. Precautions

- Avoid contact of Harvard TEMP Cem with skin and eyes.
- In case of contact with skin and eyes rinse immediately with plenty of water and consult a physician if necessary.
- If swallowed seek medical advice.

Storage information

Store at 8 – 25 °C (47 - 77 °F).
Do not use after expiry date.

Warranty

Harvard Dental International GmbH warrants this product will be free from defects in material and manufacture. Harvard Dental International GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and Harvard Dental International GmbH's sole obligation shall be repair or replacement of the Harvard Dental International GmbH product.

Limitation of liability

Except where prohibited by law, Harvard Dental International GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

For dental use only!

Store product out of reach of children!
Harvard TEMP Cem meets the requirements of ISO 3107 and the ADA/ANSI Specification No. 30.
Keep the instructions for use for the duration of the application.

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