Indications

The VacuSoft universal aspirator tip is used for large volume suction of liquids and solids during dental treatment, such as water, aerosols, saliva, blood, filling material. The cannula is also used to keep the cheek and tongue away during dental treatment, in order to allow as unobstructed a view of the treatment field as possible.

Clinical benefit

The wide and flat tip makes the VacuSoft universal aspirator tip an excellent device for keeping the patient's cheek and tongue away and ensuring a clear view of the treatment field.

The tip is available in various degrees of hardness. This means that various clinical requirements and personal preferences of the user can be met during aspiration.

The following tips are available:

- **sensitive comfort** (blue tip) – for gentle aspiration of particularly (pressure-) sensitive patients
- **comfort** (green tip) – for gentle aspiration in un-complicated treatment cases
- **solid comfort** (black tip) – for aspiration where more resistance is needed for retraction

The funnel-shaped internal construction of the tip prevents gravity-induced backflow of the aspirated media (so-called reflux) into the patient's mouth.

The cannula tube is designed for connection to aspiration systems with a 16 mm tip connector according to EN ISO 7494-2.


Application

The VacuSoft universal aspirator tip consists of two parts:

- Cannula tube (VacuSoft Base) – reprocessable
- Tip (VacuSoft Tip) – for single use. Non-reprocessable

Both components must be connected together as follows

1. The cannula tube must be reprocessed before the first application and subsequently after each application according to the ISO 17664 processing instructions - see separate reprocessing instructions. **Note:** The cannula tube must always be reprocessed without the tip.
2. Select the tip according to the clinical situation (sensitive comfort, comfort or solid comfort).
3. Attach the tip to the end of the cannula tube on which there is a recessed grip on the left and right (Fig. 1).
4. Push the tip into the cannula tube until the tip and tube are flush. The tip must be fitted into the triangular notch in the cannula tube (Fig. 2). The grooves in the shaft of the tip ensure a secure fit in the cannula tube.
5. Connect the assembled VacuSoft to the aspiration system.

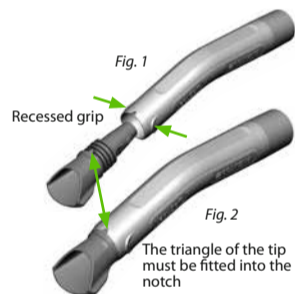Reprocessing

The cannula tube must be reprocessed before the first application and subsequently after each application according to the ISO 17664 processing instructions - see separate reprocessing instructions. **Note:** The cannula tube must always be reprocessed without the tip.

Note

All serious incidents occurring in conjunction with the product must be reported to the manufacturer and to the competent authority of the member state where the user and/or patient is registered.

ORDER INFORMATION

Starter set REF 60023549

Content: 2 tubes, 5 of each tip: sensitive comfort, comfort, solid comfort

Base Refill REF 60023550

Content: 10 pcs.

Tips Refill

| | | |
|--------------------------|----------|----------|
| sensitive comfort (blue) | 50 pcs. | 60023551 |
| | 200 pcs. | 60023556 |
| comfort (green) | 50 pcs. | 60023552 |
| | 200 pcs. | 60023557 |
| solid comfort (black) | 50 pcs. | 60023554 |
| | 200 pcs. | 60023558 |

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