# Wax Spacer

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- (en) Instructions for use
- (fr) Notice d'utilisation
- (it) Istruzioni per l'uso
- (es) Instrucciones de uso
- (pt) Instruções de utilização
- (pl) Instrukcja użycia
- (nl) Gebruiksaanwijzing
- (el) Οδηγίες χρήσης

# Wax Spacer

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Stand der Information 2020-04

# Wax Spacer

Wax bite with defined bite height (1 mm, 2 mm, 3 mm)  
to allow reliable bite registration for occlusal splints,  
functional orthodontic (FO) devices and in sleep medicine.

**(en) Instructions for use, please read carefully!**

## Intended use

The Wax Spacer allows fast, reliable and reproducible functional bite registration for the fabrication of occlusal splints, FO devices and in sleep medicine. A bite-resistant insert worked into a wax wall ensures a constant interocclusal space during occlusion.

## Indication

- Dental functional or construction bites

## Contraindication

- Hypersensitivity to one of the components

## Composition

- The Wax Spacer consists of paraffins, microcrystalline waxes and a polyethylene wax

## Classification according to MDR (EU) 2017/745

- Class I medical device for temporary contact with intact and/or injured mucous membranes.

## Patient target groups

- Adolescents and adults

## Symbols

 Article number

Upper temperature limit

 Batch

Observe Instructions for Use (IfU)

Use by

Reference that the product is a medical device

Protect from the sun

Manufacturer

## Processing

It is not necessary to become accustomed to new work processes. Dip the Wax Spacer into a 45 °C warm water bath for 20 - 25 sec before use.

### Step 1

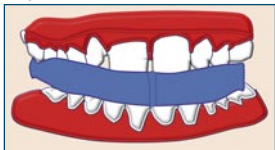
- Place the Wax Spacer in the mouth

### Step 2


- Position the Wax Spacer in the required region, then adapt slightly to the upper or lower jaw.
- Slowly move the lower jaw into the required position until the teeth touch the wax.

### Step 3

- Then continue to bite slowly until there is noticeable resistance or a slight cracking sound.
- The target position is now reached.

### Step 4

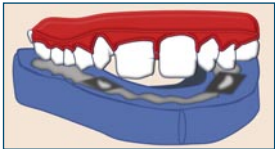- The Wax Spacer can be removed

The positioning of the bite-resistant insert can influence the occlusal height - depending on the region (3-5, 5-6, 6-7).

### Quantity specifications / labelling

Dark blue 1 mm

Medium blue 2 mm

Light blue 3 mm

## **Storage**

- Avoid direct sunlight
- Do not store above 25 °C

## **Notes**

- Do not use the material after the expiration date.
- Record the LOT number with every process that requires identification of the material.
- The Wax Spacer allows the appropriate bite height to be determined, so the use of a facebow, necessary for bite elevation in the articulator set for the jaw joint, is not necessary.

## **Disposal**

- The processed Wax Spacer must be disposed of with the residual waste.

---

For detailed information please see the applicable Material Safety Data Sheet, available upon request or as download from [www.merz-dental.de](http://www.merz-dental.de).

---

**The product properties are based on compliance with and observation of these instructions for use.**

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Sallinger GmbH**
**Merz Dental GmbH**