**Manufacturer:**  
Kulzer GmbH  
Leipziger Strasse 2  
63450 Hanau (Germany)  
Made in Romania

Importado e Distribuido no Brasil por  
Kulzer South America Ltda.  
CNPJ 48.708.010/0001-02  
Rua Centro Sthigh, 27 – cj. 42  
São Paulo – SP – CEP 05036-010  
sac@kulzer-dental.com  
Resp. Técnica: Dra. Regiane Marton – CRO 70.705  
NF AVISA: vide embalagem

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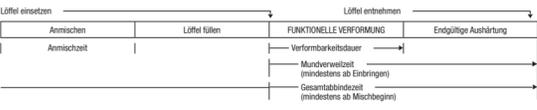
## Xantopren® function

**Special silicone impression material for functional impression taking.**

**Indications for use:**  
– Closed mouth functional impression with Xantopren function Activators No. 1 Plus and 2 Plus  
– Open mouth functional impression with Activator universal Plus

**Product colour:** light yellow

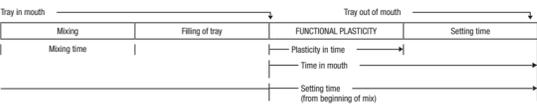
**Precautions for handling:** Visually inspect for damage before first use. Damaged products must not be used. Close carefully after use. Please use only the accessories included with the package (spoon, dropper). Since all silicone impression materials will stain clothing or surfaces once cured, contact should be avoided.

**Procedure:**  
Mixing Ratios: 1 spoon of Xantopren function (9.5 ml/14.4 g) with  
A: 4 drops Xantopren function Activator No. 1 Plus (rot) (0.08 ml/0.09 g) and 4 drops Xantopren function Activator No. 2 Plus (light yellow) (0.11 ml/0.10 g) or  
B: 6 drops Xantopren function Activator No. 1 Plus (rot) (0.13 ml/0.13 g) and 6 drops Xantopren function Activator No. 2 Plus (light yellow) (0.17 ml/0.16 g) or  
C: 3 drops Activator universal Plus Liquid (0.07 ml/0.07 g) or  
D: 2 graduations Activator universal Plus Paste (0.10 ml/0.11 g)

**Mixing:** Flatten the Xantopren function into a thin disc on the mixing pad. Using the spatula, carve shallow grooves into putty in a criss-cross pattern.

Add Xantopren function Activator No. 1 Plus and No. 2 Plus according to sections A or B. Alternatively Activator universal Plus Liquid according to section C or Activator universal Plus Paste according to section D. Spatulate thoroughly for 30 to 60 seconds.

**Impression Taking:** Duration of plasticity phase in the mouth before setting (see diagram) is dependent on the Activator and the mixing ratios.

The data given are furnished to the best of our knowledge and are based on internal test results.

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Instructions for use <b>(GB)</b>	
<b>Note for all activators:</b> Always use the specified mixing ratios. Excess or insufficient activator will result in a reduction of the elastomeric properties of the impression material. Higher ambient temperatures shorten, lower ambient temperatures prolong working and setting times.	
<b>Storage of the impression:</b>	No special storage conditions are required. The impression is stable for very long periods.
<b>Pouring the model:</b>	Pouring can take place between min. 15 minutes and max. 24 hours after the impression is removed from the mouth, no other time limitations.
<b>Recommended model material:</b>	Class III gypsum, e.g. MOLDANO® Class IV gypsum, e.g. MOLDASTONE®/MOLDASYN™
<b>Electro plating:</b>	Impressions can be copper or silver plated using conventional plating baths.
<b>Disinfection:</b>	Impressions should generally be treated by immersion for 10 minutes* in 5.25% Sodium Hypochlorite solution. Other silicon specific disinfectant solutions should be used as directed by the products manufacturer. *(DGZMK 3/93 V 1.0)
<b>Adhesion:</b>	We always recommend the use of our Universal Adhesive
<b>Storage conditions:</b>	Do not store above 25°C (77°F)
<b>Warnings for the activators:</b>	Flammable. Inhalation can cause health problems. Eyes, respiratory organs and skin can be irritated. In an emergency, rise off immediately with copious amounts of water and consult an eye specialist. Avoid skin contact. <b>Wear protective gloves/protective clothing/eye protection/face protection (EN 374, category III, e.g. Dermatri from KCL, Germany).</b> Do not swallow and ingest. If health problems arise after swallowing impression material, seek medical attention immediately. Internal blockage may arise in rare cases. Hypersensitive persons may experience an allergic reaction to the product. Do not use in pregnant or nursing patients or in children under 6 years of age. For use by dental personnel only.
<b>Xantopren function Activator 1 Plus and 2 Plus Activator universal Plus Liquid and Paste</b>	

Stand: 2016-07

Please note the following times:		Xantopren function Activator 1 Plus and 2 Plus		Activator universal Plus	
	A	B	LIQUID	PASTE	
MIXING RATIO	60 s	60 s	30 s	30 s	
Mixing time	3–5 min	2–3 min	> 1 min	> 1 min	
Plasticity time	8 min 30 s	7 min 30 s	5 min	5 min	
Time in mouth	10 min	9 min	6 min	6 min	

Times stated are related to room temperature of 23°C (73°F), 50% rel. humidity.

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