Uso previsto

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Dental impression materials

Indications for use

Zerosil® soft dental impression material is intended to be placed on a preformed impression tray and to reproduce the structure of a patients teeth and gums.

Indication

Dental Impression Material for the reproduction of the patient's oral situation.

- temporary restoration
- situation impression
- opposing jaw impression
- impression for the production of models for the thermoforming technique

Target Patient Group

Patients who need dental care.

Intended users

Dentist

Description of material

Zerosil® soft is an addition-vulcanising impression silicone based on vinylpolysiloxane (mixing ratio: A:B = 1:1 and A:B = 5:1) especially for situation impressions. Short setting time in the mouth, easy removable out of the mouth, unlimited shelf life of the impressions and a high detail reproduction let Zerosil® soft be an advantageous alternative to alginates.

Impression tray: Preparation and adhesives

All standard (in metal or hard plastics) and / or individual impression trays are applicable corresponding to the impression technique.

To improve the adhesion of the impression material to the metal or plastic trays, a tray adhesive for A-silicones should be applied.

Processing 380 ml cartridge

Carefully remove the 380 ml cartridge from the package. **Note:** The cartridges are heavy and dropping may result in damage to the cartridge rendering it unusable.

380 ml cartridge loading

- Firmly hold the full 380 ml cartridge with the closed dispensing nozzles up. Grasp the shackle of the cap in the dispensing nozzles and bend in an upward direction of 90°. Do not place undue stress on the cap.
- Remove the cap from the 380 ml cartridge completely and insert the cartridge into the automatic mixing / dosing machine according to the directions supplied by the manufacturer.

Impression taking

The area where the impression is taken has to be cleaned and dried carefully.

Position the prepared tray inside the mouth within the working time, applying slight pressure. Fix the impression without pressure until the material has set completely.

Please find processing times in "technical data" overview.

Disinfecting

The cured impression can be disinfected with disinfectants for dental silicones usually used in dental practices and dental laboratories.

Casting

After disinfecting dry the impression and store it at normal room temperature (max. 25 °C). The impression should be poured 30 min. at the earliest after removal from the mouth. In the following the impression can be poured out with usual dental plasters class 2–5 and with model resins (e. g. Modralit® 3K); normally an insulation is not necessary.

Important advice

- Zerosil® products are not to be used in combination with condensation-vulcanising impression materials.
- When using 380 ml cartridges, the mixing tip should remain on the cartridge as sealing.
- The setting times can be delayed by cool storage or shortened by warmer storage.
- The polymerisation, i. e. the setting of addition-vulcanising material can be interrupted by latex gloves, cremes, cleaning agent, resins etc. Therefore, we recommend to wear standard gloves made of polyethylene.
- Mixed material is chemically persistent. Spots onto clothing cannot be removed.

Keep away from children!

Only for dental use!

Contraindication

In case of allergies to any of the ingredients do not use Zerosil® soft. Not desired side effects of this medical device are not to be expected in case of proper processing and application. However, immune reactions (e. g. allergies) or irritations principally cannot be excluded. Should you get to know non-desired side effects – also in doubtful cases – we appreciate in any case to be informed with a detailed description of the attendant circumstances and symptoms. We will follow each hint.

Serious incidents

All serious incidents occurring in relation to the product shall be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is settled.

Single-use advice

After impression taking, cured silicone cannot be used for a further impression due to its physical physical features.

Disposal

Disposal of contents / container in accordance with local / regional / national / international regulations and safety data sheet

Storage conditions

At room temperature, i. e. 18 °C–28 °C (64 °F–82 °F).

Lot number / Expiry date

The lot number and the expiry date are indicated on the external packaging as well as on each double cartridge or tube. In case of claims please always indicate the lot number of the product. Do not use the product after expiration.

This product was developed for dental use and has to be applied according to the instructions. In the case of damages resulting from ulterior or improper use, the manufacturer assumes no liability. Furthermore the user is bound to check the product according to suitability and possible use in self dependence before using it subject to its purpose, especially in case the purpose is not mentioned in the instructions.

Processing with tubes

Mixing: When using tubes two equal-sized lines of components A and B have to be dosed on the mixing plate, then mix with a spatula for 45 seconds until a homogeneous colour is arises. The A and B components of a packaging must only be used in this combination. Tubes should be closed well after use.

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