# CELALUX® 3

DIRECTIONS FOR USE  
GEBRAUCHSANWEISUNG  
MODE D'EMPLOI  
INSTRUCCIONES DE USO

## CONTENT

DIRECTIONS FOR USE.....	3
GEBRAUCHSANWEISUNG .....	11
MODE D'EMPLOI.....	19
INSTRUCCIONES DE USO.....	27

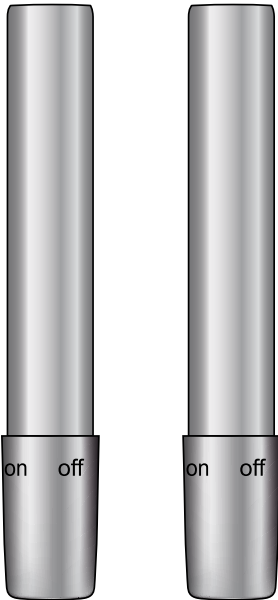
1 Handpiece


2 Light guide


3 Battery pack

4 Rubber shield

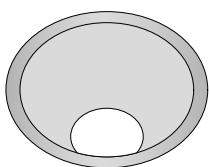
5 Recharging station

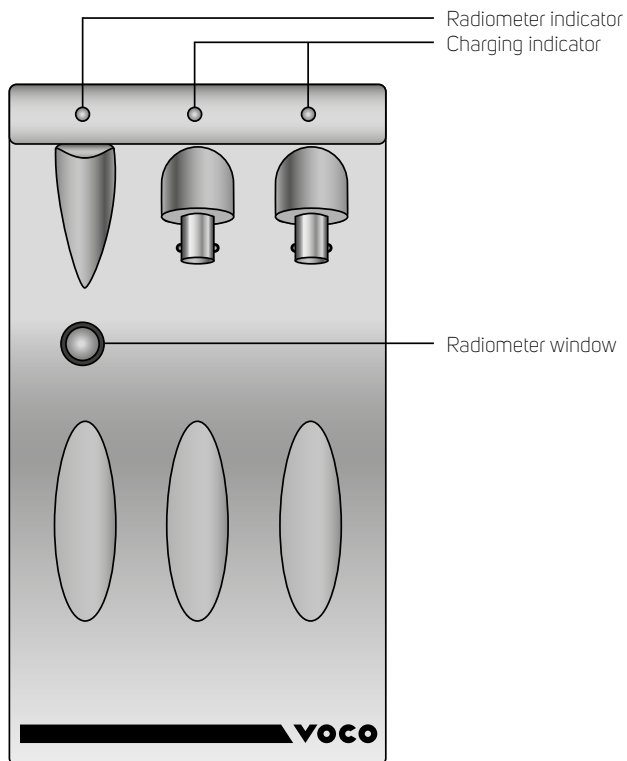Radiometer:


- Green = OK
- Red = low output

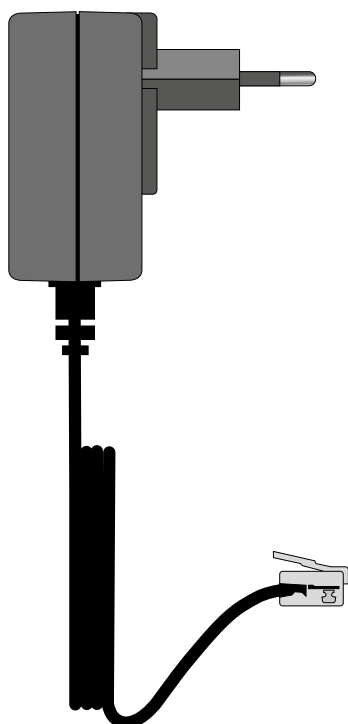6 Power cable


7 Handpiece:

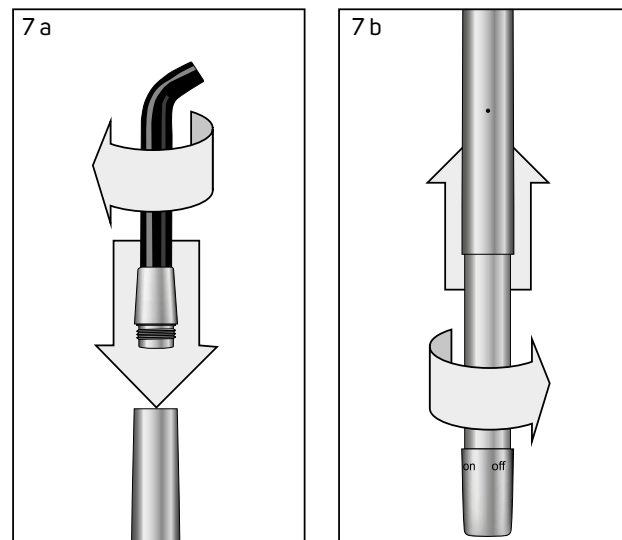- a Light guide screw-on
- b Battery insertion; "Rotate while inserting"

8 Recharging Station:

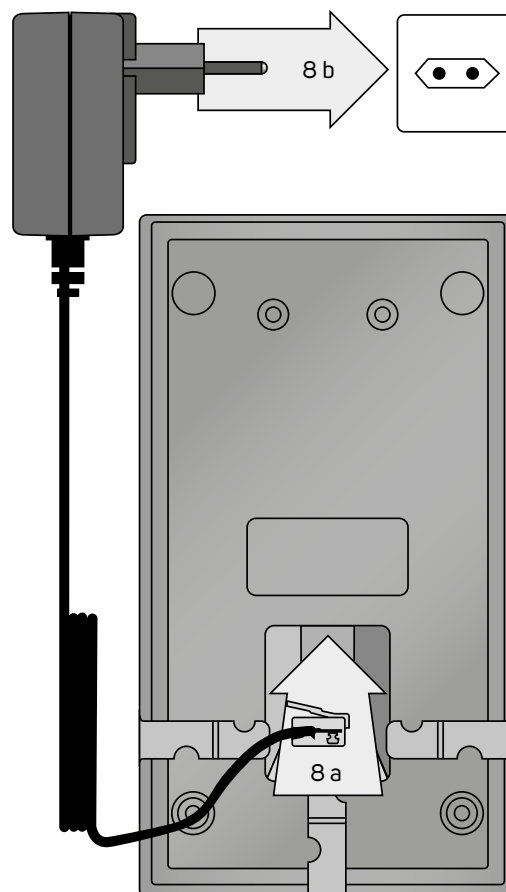
- a Power cable insertion to RC Station
- b Power cable to wall socket

## Celalux 3 Curing Light • Operation at a Glance


### 9 Handpiece:

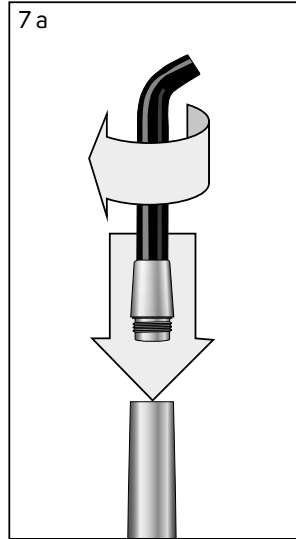- Adjusting light angle
- Position light guide tip close to material for optimal results
- Check light guide attachment,
- Check & clean both ends of light guide for debris or other obstructions,
- Check Handpiece window

### 10 Recharging Station:

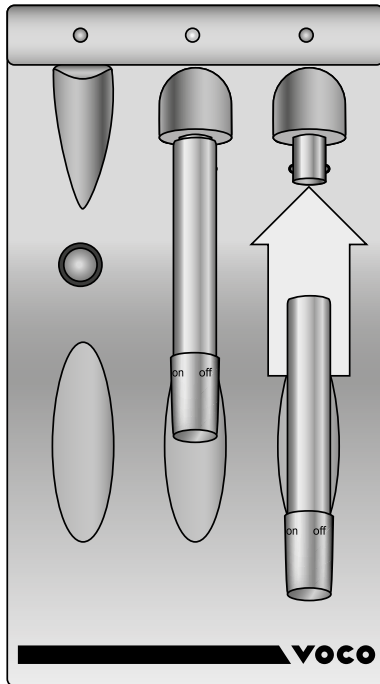- Battery on charging mount (2-steps)
- Do not replace on recharging station before cleaning & disinfection
- Use radiometer regularly to assure correct light output
- Check battery charge indicators

### Recharging:

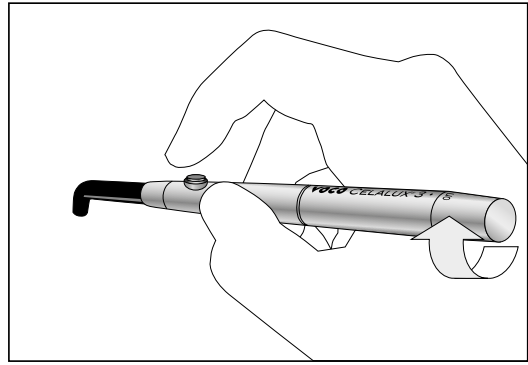
- Constant light = charging
- Light off = fully charged battery; ready to use

### Radiometer:


- Green = OK
- Red = low output

## Celalux 3 Curing Light • Operation at a Glance


### 11 Activation of Celalux 3: switch on handpiece


### 12 Light is activated for 20 seconds: depress switch on handpiece

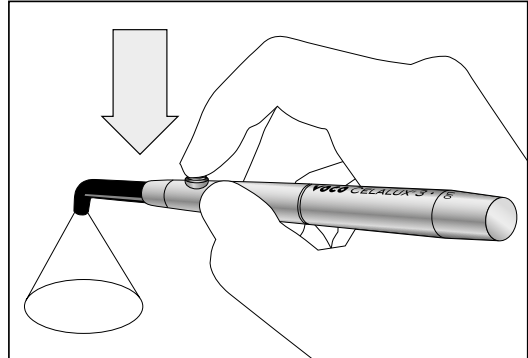
### 13 Handpiece will vibrate and flash after 10 seconds
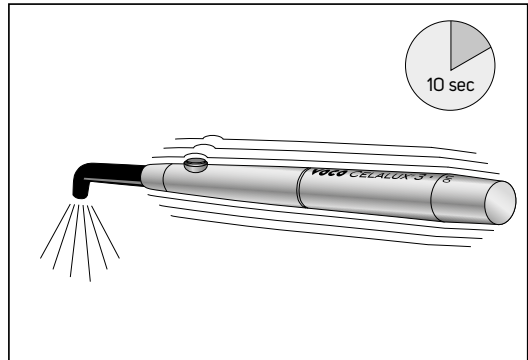
### 14 Light off at once: depress switch on handpiece
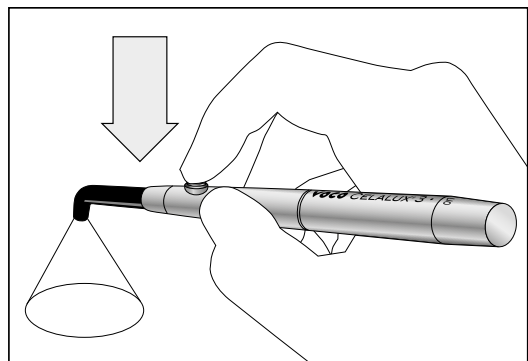
### 15 Light is off
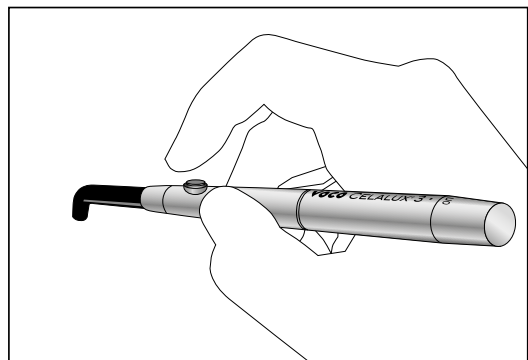
# CELALUX® 3

## Curing Light

**Caution:** For dental use only.  
**USA:** Rx only  
**Made in Switzerland**

Content	Page
<b>1. Product Description &amp; Intended Use</b> .....	<b>6</b>
1.1. Indications for use.....	6
1.2. Contraindications.....	6
<b>2. Safety Notices</b> .....	<b>6</b>
2.1. General warnings.....	6
2.2. Handpiece.....	6
2.3. Charging Station.....	6
2.4. Precautions.....	6
<b>3. Operation Instructions</b> .....	<b>7</b>
3.1. Owner's Obligations.....	7
3.2. Set-up.....	7
3.3. Charging.....	7
3.4. Operation.....	7
<b>4. Dental Material Recommendations</b> .....	<b>7</b>
<b>5. Hygiene</b> .....	<b>7</b>
5.1. Before Patient Treatment.....	7
5.2. During Patient Treatment.....	7
5.3. After Patient Treatment.....	7
<b>6. General Care &amp; Maintenance</b> .....	<b>8</b>
6.1. Battery Care.....	8
6.2. Repair Service.....	8
<b>7. Trouble-Shooting</b> .....	<b>8</b>
<b>8. Compliance</b> .....	<b>8</b>
8.1. Declaration of Conformity.....	8
8.2. EMC Tables.....	8
8.3. Regulatory Certifications.....	8
<b>9. Technical Data</b> .....	<b>8</b>
<b>10. Warranty</b> .....	<b>8</b>
<b>11. Re-Order Information</b> .....	<b>8</b>
<b>12. Storage &amp; Transport Information</b> .....	<b>8</b>
<b>13. Explanation of Symbols</b> .....	<b>9</b>
<b>14. Contact Information</b> .....	<b>9</b>
<b>Annex 1</b> .....	<b>9</b>

### 1. Product Description

The **Celalux 3** curing light is a cordless, pen-shaped, high energy LED device intended for the polymerization of light curing dental materials which are activated by wavelengths in the 450 nm to 480 nm light range. It is exclusively designed for use by trained dental professionals in the dental operatory or dental laboratory. The **Celalux 3** is characterized by:

- High output of blue light
- Light weight, small compact size, and ergonomic design
- Cordless operation with easy to exchange battery packs
- Exceptionally fast recharging of battery packs
- Detachable fiber-optic light guide(s) designed for optimal intra-oral access
- Effective curing field for polymerization of light curing materials
- Convenient and easy to use recharging station with extra battery packs and built-in radiometer

The **Celalux 3** has been shipped from the manufacturer in a safe and technically sound condition. The following directions for use must be adhered to in order: to maintain product performance; to ensure risk-free operation; and to prevent injury to patients and dental professionals.

#### 1.1. Indications for Use

The **Celalux 3** curing light is indicated for polymerization of dental materials which cure with blue light (450 nm to 480 nm).

#### 1.2. Contraindications

The **Celalux 3** curing light is not for use with patients who:

- A. are prone to photobiological reactions, including patients with solar urticaria or erythropoietic protoporphyria; or
- B. are currently undergoing treatment with photosensitizing pharmaceuticals.

Anyone (users, patients, etc.) with questions, or concerns or a history of health issues concerning their eyes (ex. Retinal disease, retinopathy, cataracts etc.), should consult their eye specialist before operating or prior to receiving treatment with the **Celalux 3** curing light.

## 2. Safety Notices

The **Celalux 3** is only intended for use as described in this manual. Any and all other uses constitute improper use and are the sole responsibility of the user and owner.

**Safety alert symbol**  
 In this manual, this symbol alerts you to information regarding possible safety and/or personal injury hazards. Obey all safety messages that follow this symbol.

### 2.1. General Warnings

- Never modify the **Celalux 3** curing light or any **Celalux 3** accessories. Modification may compromise safety and effectiveness.
- Maintain adequate infection control measures when using the **Celalux 3** in the oral cavity, and execute the appropriate hygiene plan after usage with patients.
- Do not use damaged equipment. Routinely check the light guide for scratches and surface damage.
- Only authorized technicians can repair the **Celalux 3** curing light, battery packs and recharging station.
- Avoid placing the **Celalux 3** or the recharging station on top, under, or close to other equipment. A safe distance from other electrical appliance should be maintained.
- Use of accessories not mentioned in this manual may result in an unsafe conditions and/or reduced product performance. For this reason, use only factory authorized accessories.
- Do not charge or use the device near flammable or combustible substances.

### 2.2. Handpiece

- Make sure that the exit-port of the light guide is not covered by debris or items that may block the light emission.
- Do not use the **Celalux 3** light guide as a tissue retractor or as an instrument to manipulate the tongue and/or cheeks.
- Make certain the **Celalux 3** eye-protector is installed securely and inspect it routinely. Replace as needed.
- Always clean & disinfect the **Celalux 3** handpiece before placing it on/in the recharging station.
- To change the direction of the light, rotate the handpiece as a whole, similar to rolling a pen in your fingers or rotate the light guide independent of the handpiece.
- If not in use, always switch the battery pack into the "Off position" to avoid unintentional activation.

### 2.3. Recharging Station

- Make certain there is a match: incoming power supply and the product electrical requirement. Do not use the recharging station with voltages different than stated in this manual or labeled on the equipment.
- Voltage is present in the recharging station and battery packs: do not open! Operate only in dry conditions. Never use when wet.
- Installation and operate only on flat surfaces.
- Unplug the power connector from main electrical supply (ex. wall socket) before disconnecting the power cable connector from the recharging station.
- Only touch the recharging station with clean/disinfection gloves to avoid contamination by body fluids.
- Do not use the power connector for other devices than **Celalux 3**.

### 2.4. Precautions

- People who have undergone treatment for cataracts, have a history of photobiological reactions, or who are under the influence of photosensitizing medication (ex. 8-methoxypsoralen or dimethylchloro – tetracycline), may be sensitive to light. Such patients should not be treated with the **Celalux 3**.
- Patients as well as dental professionals should always use appropriate safety precautions. Use the **Celalux 3** curing light with great care and carefully comply with all safety precautions, including wearing suitable light filtering safety glasses or goggles.
- Only use the **Celalux 3** after it has completed an adequate hygiene procedure.
- Do not use mobile phones or other hand-held electronic devices at the same time as the **Celalux 3**.
- Avoid directing the light at unprotected soft-tissue. This may cause irritation and/or injury.
- Do not stare at the light source and avoid allowing the light to shine directly at eyes. This, and reflected light from shiny surfaces such as the tooth surfaces, may injure the eyes. Always install the eye-protector on the **Celalux 3** in addition to using protective, light filtering safety glasses.
- Confine usage of the light to the area being treated.
- Prolonged usage of any curing light can increase temperature. Extended usage near pulp and soft tissues should be avoided to prevent injury. Curing near or around sensitive tissues should be performed with adequate breaks.

### 3. Operation Instructions

#### 3.1. Owner's Obligations

In addition to complying with the statutory regulations, the owner must ensure that statutory requirements are observed and implemented in the workplace. In particular, the owner must train personnel and comply with industrial safety legislations and all related regulations or laws.

The owner is fully responsible to assure that the **Celalux 3** unit is in a safe working condition at all times. If in doubt, do not use the unit and contact VOCO.

VOCO recommends that you keep this manual and related document(s) together.

#### 3.2. Set-up

- Carefully unpack all **Celalux 3** components and familiarize yourself the individual items. Check for completeness.
  - A. First, check that the voltage indicated on the **Celalux 3** name plate complies with the local power supply (Fig. 8).
  - B. On the underside of the recharging station, plug the phone-jack-type power cable connector into the socket (Fig. 8).
  - C. Adjacent to the recharging station socket, thread the power cable underneath the three clips in one of the three cable exit-channels (Fig. 8). The power cable should fit snugly under the clips and relieve tension on the phone-jack connector.
  - D. Carefully insert and gently screw-on the light guide to the front-end of **Celalux 3** handpiece (threaded socket, Fig. 7, 9). The light guide rotates independently of the handpiece.
  - E. Attach the eye-protector on to the light guide.

1. First, check that the voltage indicated on the **Celalux 3** name plate complies with the local power supply
2. Place the recharging station on a stable, horizontal surface at least 1.5 m from the patient!
3. Attach the eye-protector on to the light guide

#### 3.3. Charging

- A. Fully charge both battery packs prior to first use. Insert the batteries onto the charging connectors in the recharging station (Fig. 10).
- B. When resistance is encountered, push on the battery pack to lock it in place (Fig. 7). A green light will be seen on the recharging station indicating that charging is in progress.
- C. When the green light turns off, battery charging is complete.
- D. When the **Celalux 3** is not in use, battery packs should be kept on the charging mounts to ensure maximum battery operating time.
- E. Depleted battery packs will require approximately an hour to fully recharge. A fully charged battery pack will power the **Celalux 3** for approximately 45 cycles.

1. **Never autoclave the battery packs!**
2. These batteries are memory-free and should be kept charging whenever possible.
3. These battery packs may only be charged by using the provided recharging station.
4. Do not insert metallic parts into open end of battery; short circuit could occur.
5. Keep battery contacts dry for proper operation.

#### 3.4. Operation

- A. Gently insert one battery pack into the rear-end of the **Celalux 3** handpiece (Fig. 7). You will feel the connector slide the battery automatically into the "Off position" (Fig. 7).
- B. When resistance is felt, gently push the battery all the way into the handle (Fig. 7). A click will be heard when the battery pack is completely inserted and correctly seated in the **Celalux 3** handpiece.
- C. Rotate the battery pack by a quarter turn, in either direction, to the "On position". To confirm, that the device is correctly powered, a short flash of blue light is seen.
- D. With the handpiece and battery pack in the "On position", the **Celalux 3** can now be activated via the switch (Fig. 11).
- E. Activate the **Celalux 3** by the switch on the handpiece (Fig. 11). The instrument notifies the start of a cycle by a short vibration.
- F. Once activated, the **Celalux 3** curing light will emit blue light for 20 seconds.
- G. After 10 seconds the **Celalux 3** will vibrate and flash once signaling the time cured as an orientation.
- H. To manually stop curing or interrupt the curing cycle, press a second time the switch on the handpiece (Fig. 13). After 20 seconds the light stops automatically and a long vibration indicates the end of the cycle.
- I. If the battery pack is depleted the **Celalux 3** will indicate that by flashing/vibrating 5 times in quick succession. Replace with 2nd battery pack and resume the procedure. Do not place the depleted battery pack in the recharging station until it is properly cleaned and disinfected.
- J. Adjust the angle of the light if necessary by "rolling" a pen-like grip or rotate the light guide.
- K. For optimal curing results, the light guide tip should be positioned as close as possible to the dental material without actually touching the dental material itself.
- L. The **Celalux 3** handpiece may feel slightly warm during prolonged operation. This is normal. To prevent the device from overheating, a pause of 3 minutes is therefore necessary every time 10 curing cycles of 20 seconds each have been carried out in direct sequence. The device vibrates 4 times to indicate when too many consecutive curing cycles have been carried out. The **Celalux 3** is ready to be used again after the 3-minute break.
- M. To inspect the light output with the built in radiometer: put the cleaned end of the light guide on to the sensor window with the output surface oriented horizontally and

activate a curing cycle. The adjacent LED will light green, if light output is ready to cure within normal curing times. It will indicate red, when light output is reduced, see trouble shooting section. You may stop the cycle as soon as you got the reading. The radiometer works only if the charger is connected to the power supply.

- N. After the procedure or when the instrument is not in use, turn the battery pack to the "Off position" or remove it from the instrument for recharging. This is important for battery life.

1. Prior to using the **Celalux 3**, attached the eye-protector on to the light guide and use protective, light filtering eyewear (ex. glasses).
2. Do not stare at operating curing light. May be harmful to the eyes. Risk Group 2, Caution: possibly hazardous optical radiation emitted from this product
3. Only use the **Celalux 3** after it has completed an adequate hygiene procedure.
4. Never place the **Celalux 3** handpiece, light guide, or depleted battery packs on the recharging stations before they are cleaned & disinfected.

### 4. Dental Material Recommendations

In general, the **Celalux 3** has been optimized to achieve a 10-20 second curing time with dental materials which cure with blue light (450 nm to 480 nm).

These materials include restoratives, bonding agents & adhesives, bases, liners, fissure sealants, some temporary materials and luting materials for brackets as well as dental-lab materials such as ceramic inlays. However, manufacturer's instructions regarding specific light curing dental materials should be followed.

### 5. Hygiene

#### 5.1. Before Patient Treatment

- Check regularly that the light output via the radiometer and inspect the light guide to assure there are no obstructions.
- Check that the eye-protector is installed on the light guide properly.
- Outfit dental personnel and patient with appropriate protective eyewear (light filtering, infection protection).
- Consider patient's health needs, potential light sensitivity and determine the appropriate precautionary actions.
- Place the **Celalux 3** handpiece in a convenient place for use during treatment.

1. Prior to treatment, attach the rubber eye-protector on to the light guide and use protective, light filtering eyewear (ex. glasses)
2. Only use the **Celalux 3** after it has completed an adequate hygiene procedure.

#### 5.2. During Patient Treatment

- Never place the **Celalux 3** handpiece, light guide, or depleted battery packs on the recharging stations before they are cleaned & disinfected.
- Do not use the **Celalux 3** as a tissue retractor to manipulate the tongue and/or cheeks.

1. Only use the **Celalux 3** after it has completed an adequate hygiene procedure.
2. Never place the **Celalux 3** handpiece, light guide, or depleted battery packs on the recharging stations before they are cleaned & disinfected

#### 5.3. After Patient Treatment

- Discard examination gloves according to local regulations and disinfect hands with a bactericidal, virucidal, and fungicidal hand disinfectant solution according to local regulations. Use according to disinfectant solution manufacturer's instructions for use. Use a new, clean pair of examination gloves for cleaning and disinfection of the **Celalux 3**.
- Remove eye-protector. Clean and disinfection this separately.
- Remove bioburden from the **Celalux 3**
- Disinfect with alcohol based fluids by wiping the complete exterior of the **Celalux 3** components. Use a single-use cloth for wiping the unit. Do not submerge and be careful to avoid getting the battery pack contacts wet. Do not allow fluids to penetrate into the handpiece.
- The battery pack can be removed after the **Celalux 3** exterior has been cleaned and disinfected.
- The light guide can be removed after cleaning and disinfection. The light guide and the eye-protector can now be autoclaved using a validated sterilization process at a max. temperature of 134°C and for a time of up to 20 min. Perform sterilization according to EN 17665-1:2006 and EN 556-1:2001 at 134 °C for at least 3 minutes and use steam sterilizers that comply with the requirements of EN 13060:2004+A2:2010, class B or S.
- After cleaning and disinfection, place the **Celalux 3** and battery pack back on the recharging station. Do not leave the battery in the instrument, when not in use.

1. Only use the **Celalux 3** after it has completed an adequate hygiene procedure.
2. Never place the **Celalux 3** handpiece, light guide, or depleted battery packs on the recharging stations before they are cleaned & disinfected
3. Disinfect with alcohol based fluids by wiping the complete exterior of the **Celalux 3** components. Do not submerge in fluids and be careful to avoid getting the battery pack contacts wet. Keep battery contacts dry for proper operation.
4. The battery pack can be removed after the **Celalux 3** exterior has been cleaned and disinfected.
5. Battery packs and handpiece must not be autoclaved!!

## 6. General Care & Maintenance

- Regularly inspect the **Cetalux 3** for damage and if necessary send for repair service.
- Use the built-in radiometer to monitor output.
- The surface of the recharging station can be cleaned and disinfected using a mild cleaner or disinfectant with an alcohol base. To avoid scratches, always use a soft, lint-free cloth to clean plastic surfaces.

### 6.1. Battery Care

- As is the case with all batteries, battery capacity will decline during the course of its product lifetime. With time, batteries are a consumable item. This is due to the internal chemical reaction inside batteries. Never open the batteries. Replacement is at user's discretion.
- When the **Cetalux 3** is not in use, battery packs should be kept on the charging mounts which is connected to an active electrical outlet to ensure maximum battery operating time.
- Battery contacts usually do not require cleaning and must be kept dry.
- This device is provided with 2 batteries. The **Cetalux 3** and battery should not be disposed in normal domestic waste. For environmental reasons, please dispose of the device and battery according to local environmental guidelines or regulations.
- Ensure that only depleted batteries are taken to recycling centres, and take preventive measures against short-circuiting, such as insulation of the battery poles.

- Battery pack contacts usually do not require cleaning and must be kept dry.
- Never open or modify the batteries.
- Keep batteries a safe distance from sources of heat and fire.

### 6.2. Repair Service

- Inform the technical service department of your local dental supplier of a malfunction and the need of repair.
- Pack all components of the **Cetalux 3** in a sturdy carton with appropriate protective inner-packaging, include your name, address, and a brief description of the problem. Important: enclose a copy of proof of purchase (ex. invoice, shipping note) if the **Cetalux 3** is within the warranty period.
- Send your package to your local dental supplier with a reliable postal/shipping service.

## 7. Trouble-Shooting

If there is any doubt about safe operation, then power must be disconnected and the battery removed to avoid accidental operation. This may be the case, for example, if the apparatus is visibly damaged or no longer works correctly.

Symptom	Action
Unit does not turn "On"	<ol style="list-style-type: none"> <li>Check if battery pack is properly inserted and in the "On" position</li> <li>Replace/recharge battery pack</li> </ol>
Radiometer turns "red" or light operates normally but material doesn't cure	<ol style="list-style-type: none"> <li>Check if light guide is correctly inserted, clean and no damages can be found. Clean both end surfaces carefully from any debris with alcohol and a wooden or plastic cleanser, don't use hard or metal tools.</li> <li>Replace light guide if damaged or appears cloudy.</li> <li>Contact your dealer for further technical assistance.</li> </ol>

The instrument communicates with several vibration signals to inform about normal or special operating modes.

Vibration signal	Explanation
1 x	A 20 second curing cycle has started
1 short signal	10 seconds of curing have been completed
1 x	A 20 second curing cycle has ended
2 x	A curing cycle has been interrupted before 20 seconds
3 x	The internal temperature of the handpiece is too high, please allow some minutes for cooling before next use
4 x	Too many consecutive curing cycles have been done, a short pause is required
5 x	Battery charge is low, needs recharging

## 8. Compliance

### 8.1. Declaration of Conformity

VOCO hereby declares that the design, structure and version introduced to the dental market of the unit described in this manual correspond to the relevant safety and health requirements of the EU Directive. This declaration is invalid in the event of an alteration made to the unit without prior VOCO authorization.

VOCO hereby confirms that the **Cetalux 3** unit complies with the directive 93/42 EC and the standards IEC 60601-1 and IEC 60601.

### 8.2. EMC Tables

See Annex 1

### 8.3. Regulatory Certifications

93/42/EEC

Council Directive 93/42/EEC of Date concerning medical devices amended by directive 2007/47/EC, annex 1

## 9. Technical Data

This device is provided with 2 batteries. The **Cetalux 3** and battery should not be disposed in normal domestic waste. For environmental reasons, please dispose of the device and battery according to local environmental guidelines or regulations.

### Cetalux 3 instrument

Classification of product	Class I, according to rule 12, council directive 93/42/EEC
Applied part	Type BF
Light source	High power Light Emitting Diode
Wavelength	450-480 nm with peak at 465 nm
Average light intensity	1300 mW/cm <sup>2</sup>
Operation	intermittent use, 10 cycles @ 20 sec / 3 minutes pause
Protection from liquids	IP X0
Light guide	D 8 mm optical fiber, (autoclavable @ max. 134 °C, max. 20 min)
Battery	Lithium-Polymer, rechargeable battery, 3,7 V; 270 mAh
Battery performance	>45 cycles @ 20 sec
Dimensions	D 12,6 mm x L 210 mm
Weight	70 g (with battery and light guide)

### Charger station

Input	6 VDC, < 1 A
Capacity	simultaneous, automatic charging of 2 batteries with overcharging protection
Radiometer	built in, optical indication for light intensity, green/red - suitable/insufficient for effective treatment
Dimensions	L 160 mm x W 90 mm x H 35 mm
Weight	170 g

### AC Supply

Input	100-240 VAC, 50/60 Hz, 0,5 A
Output	6 VDC/1 A
Classification	Class II, double/reinforced isolated equipment
Plug	exchangeable wall plug adapter

### General

Environment	Not intended for use where flammable gases are present
Operating conditions	10 °C - 30 °C
Storage & transport	-20 °C - 45 °C, 10% - 95% humidity, 50 - 106 kPa barometric pressure

## 10. Warranty

VOCO grants a 24 months warranty on all components of the **Cetalux 3** except the battery and the light guide. The warranty commences on the date of purchase.

Within the warranty period, VOCO will free of charge, eliminate any defects in the **Cetalux 3** and **Cetalux 3** accessories resulting from faults in material or workmanship. VOCO reserves the right to either repair or exchange parts or exchange the complete device. The following is not covered by this warranty: Damage from improper use (ex. operation with incorrect current/voltage, breakage, cleaning by other than the recommended methods); Normal wear and defects that have a minor effect on the value or operation of the device. This warranty becomes void if repairs are undertaken by unauthorized persons.

This warranty extends to every country where this device is supplied by VOCO or by authorized representatives of VOCO products and where no import restrictions or legal regulations hinder or prevent service being given under warranty. Service under this warranty does not affect the expiration date of the warranty. The warranty on parts or entire devices which are exchanged ends when the warranty on this device expires.

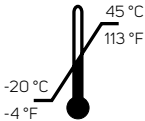
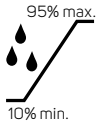
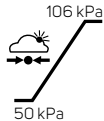
In the event of a complaint claim regarding this device, return the complete **Cetalux 3** device (Recharging station, handpiece, battery packs, and light guides) together with the proof of purchase (ex. Invoice) to your dealer.

All other claims including those for damages resulting from this warranty are excluded unless our liability is legally mandatory. Except where prohibited by law, VOCO will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

## 11. Re-Order information

Accessory	Re-Order Number
<b>Cetalux 3</b> LED curing light.....	9090
8 mm Light Guide.....	9091
Battery Pack.....	9092
Rubber shield (5 pcs.).....	9093

## 12. Storage & Transport Information

				
Keep dry	Fragile, handle with care	Temperature limitation -20 °C / -4 °F to 45 °C / 113 °F	Humidity relative 10% min. to 95% max.	Pressure 50 kPa to 106 kPa

### 13. Explanation of Symbols

	Complies with the regulations of the directive 93/42/EEC.
	Attention: Please see the documentation that accompanies this product!
	This symbol refers to the special disposal of electrical and electronic devices in EU countries. Please do not discard this device in household garbage. Check the proper means of disposal in your country at your community recycling or waste center or at your dealer. Take care to dispose of properly.
	This symbol indicates that a product or instrument (here, <b>Celalux 3</b> Light guides and eye-protector) can be sterilized at the indicated temperature (134 °C = 275 °F).
	Please see the operating manual!
	Warning symbol: high intensity LED radiation

### 14. Contact Information

- In case of problems, please contact or e-mail VOCC.

### Annex 1

#### Guidance and manufacturer's declaration – Electromagnetic emissions

EN 60601-1-2/ Table 201  
Equipment Group 1 and Class B; per CISPR 11

The <b>Celalux 3</b> is intended for use in the electromagnetic environment specified below. The customer or the user of the <b>Celalux 3</b> should ensure that it is used in such an environment		
RF emissions CISPR 11	Group 1	The <b>Celalux 3</b> uses RF energy only for its internal function. Therefore its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The <b>Celalux 3</b> is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Complies	
Voltage fluctuations / flicker emissions IEC 61000-3-3		

#### Guidance and manufacturer's declaration – Electromagnetic immunity

EN 60601-1-2 / Table 202

The <b>Celalux 3</b> is intended for use in the electromagnetic environment specified below. The customer or the user of the <b>Celalux 3</b> should ensure that it is used in such an environment.			
Immunity Test	IEC 60601 Test Level	Compliance	Electromagnetic Environment Guidance
Electrostatic discharge (ESD) IEC 61000-4-2	±6 kV contact ±8 kV air	The device continues to work regularly and in safety.	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Radiated RF IEC 61000-4-3	3 V/m 80 MHz to 2.5 GHz	The device continues to work regularly and in safety.	Field strengths from fixed transmitters cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered.
Electrical fast transient / burst IEC 61000-4-4	±2 kV for power supply lines ±1 kV for input / output lines	The device continues to work regularly and in safety.	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	±1 kV differential mode ±2 kV common mode	The device continues to work regularly and in safety.	

Conducted RF IEC 61000-4-6	3 Veff 150 kHz to 80 MHz	The device continues to work regularly and in safety.	Portable and mobile RF communications equipment should be used no closer to any part of the disposal including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.  Recommended separation distance: $d = 1.2 \sqrt{P}$ $d = 1.2 \sqrt{P}$ 80 MHz to 800 MHz $d = 2.3 \sqrt{P}$ 800 MHz to 2.5 GHz
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3-100 A/m	The device continues to work regularly and in safety.	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	< 5% U/10 ms 70% U/0.5 s 40% U/0.1 s	The device can vary from the required levels of immunity with a duration for as long as the device remains in safety; no malfunctions have been detected and can be restored to pre-test status with the intervention of the operator	Mains power quality should be that of a typical commercial or hospital environment.

#### Note:

- (P) is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and (d) is the recommended separation distance in meters (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, should be less than the compliance level in each frequency range (b). Interference may occur in the vicinity of equipment marked with the following symbol:

- These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people. Field strengths from fixed transmitters, such as base stations for radio (cellular / cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the **Celalux 3** is used exceeds the applicable RF compliance level above, the **Celalux 3** should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the **Celalux 3**.

#### Recommended separation distances between portable and mobile RF communications equipment and the Celalux 3

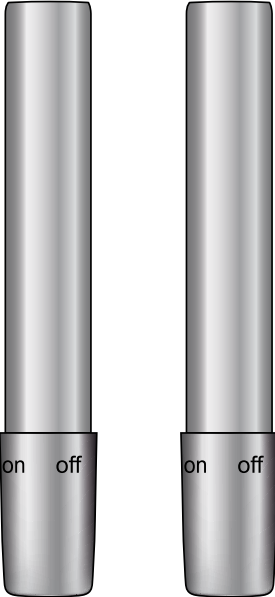
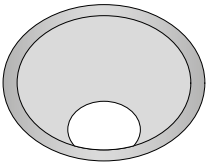
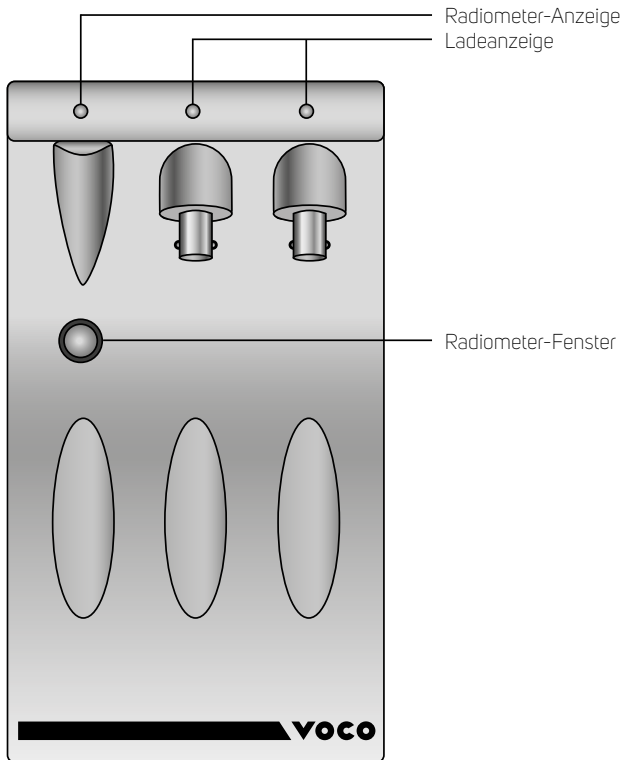
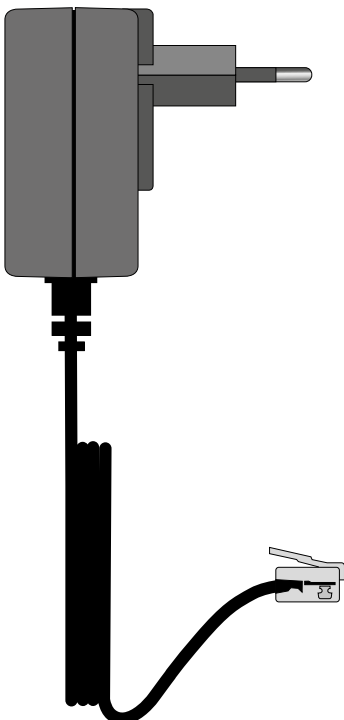
This product is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the **Celalux 3** can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the **Celalux 3** as recommended below, according to the maximum output power of the communications equipment.

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VOCO GmbH