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### 1. Produktbeschreibung

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### 2. Composition

BisEMA, barium glass silanized, fumed silica, ytterbium trifluoride (Filler content: 61 % wt, 40 % by volume, particle size between 0.04 - 3.00 µm, median 0.7 µm) catalysts, inhibitors, pigments

### 3. Indication

Permanent cementation of indirect composite, metal or porcelain/zirconia restorations (inlays/onlays, crowns, bridges, root posts, screws, etc.).

### 4. Contra-indication

Opened pulp, pulpitis, known allergy to the above mentioned ingredients of the product.

### 5. Side effects

In individual cases, contact allergies have been described to products of a similar composition. In case of doubt, the allergy should be clarified and excluded by a specific test before the application of this product. In case of known hypersensitivity (allergies) to these ingredients of els cem, the application should be avoided. To avoid pulp reactions, it is recommended to cover the exposed dentine in the pulp area with suitable underfilling materials (preferably with calcium hydroxide). Note: els cem does not contain BisGMA, TEGDMA or HEMA.

### 6. Interactions

Phenolic compounds (e.g. eugenol or preparations containing thymol), oxidative disinfectants (e.g. hydrogen peroxide) or strong acids, in particular acidic one-component adhesive systems or acid primers, can impair polymerisation of the product. Basic sandblasting abrasives inhibit the effect of the product SAREMCO cmf prime.

### 7. Processing stages

#### 7.1. Preparing the restoration an tooth

Roughen (metal surfaces, zirconium oxide) respectively silanise (porcelain) inner surfaces of the restoration or workpiece to create a physical retention. Especially with zirconium oxide surfaces a good retentive pattern is decisive for the long-life cycle of the restoration. Blast the inner surfaces of zirconium oxide restorations with 110 µ aluminium oxide at 2 - 3 bar (roughen, enlarge surface) and then clean with oil-free air. Afterwards, the use of a zirconium oxide primer is recommended. Observe the manufacturer's instructions for use. Keep dry! Remove temporary restoration and any residues of the temporary restoration, clean and dry the core or cavity.

#### 7.2. Enamel and dentine etching

Apply the etching gel and condition in correspondence to instruction for use of the product. SAREMCO recommends cmf etch. Sufficiently etched enamel appears matt and white.

#### 7.3. Adhesion

Apply the adhesive and condition in correspondence to instruction for use of the product. For patients with a proven allergy to HEMA, TEGDMA or BisGMA SAREMCO recommends cmf adhesive system and els duobond.

**Warning:** To avoid interaction of this product with a strongly acidic medium, do not use any acidic one-component adhesives or cover acidic primers with a properly hardened adhesive film. Adhesives should always be applied in a very thin layer due to the fitting accuracy of the workpiece.

#### 7.4. Cementation, fitting, curing

Remove the cap and place a new mixing tip. Dispense the product directly from the syringe onto the surfaces of the restoration to be bonded. Insert the restoration and fix it with slight pressure in the correct end position. Remove excess immediately by using a suitable instrument/aid. Light cure the joints between tooth and restoration for 20 seconds (per segment).

**Warning:** At a room temperature of 23°C / 73°F and a relative humidity of 50 %, the following times are available for the processing:

Processing time: > 1:30 min.  
Curing time: < 5:00 min.

Lower temperatures extend the stated times, while higher temperatures or stronger action of light shorten them. The stated exposure time is based on light polymerisation devices with an emission maximum ranging between 450 - 470 nm and a luminous power  $\geq 700 \text{ mW/cm}^2$ . The necessary exposure time can vary depending on the light source and its instructions for use. The inserted working piece must not be moved after the end of the processing time.

#### 7.5. Finishing

Finish the margins of the restoration and proximal areas by using suitable instruments (finishing diamonds, flexible polishing discs, strips etc.). It is recommended to fluorideate the treated teeth.

### 8. Storage

Protect light-curing products from strong sources of light and heat! els cem was developed for use at room temperature (20°C - 25°C / 68°F - 77°F). Store in the fridge between 4°C - 8°C / 39°F - 46°F. Do not freeze! Bring to room temperature before use.

### 9. Batch number and expiry date

The batch number should be specified to identify products in case of enquiries. Products should no longer be used once the expiry date has elapsed.

### 10. Precautionary measures

Close containers after each use with the right lid. Keep out of reach of children. Commercially available medical gloves do not provide protection against the sensitisation effect of methacrylates. If the product comes in contact with the glove, remove the glove and dispose of it, wash your hands with water and soap immediately and put on a new glove. In case of an allergic reaction, seek medical advice.

### 11. Emergency measures

In case of direct contact with the oral mucosa, rinse with water. In case of contact with the eyes, rinse thoroughly with water. Consult an eye specialist.

### 12. Hygiene

Use application instruments for one patient only. Dose products away from patients to avoid contamination. Do not disinfect double syringes with oxidative disinfectants, rather with medical alcohol.

### 13. Warranty

Our liability is restricted to the quality of our products. In case of a product being of defective quality, only its value is replaced. For further damages, namely that caused by non-compliance with the instructions for use of other improper handling or inappropriate use of a product, any liability is rejected. It is the responsibility of the user to check, before using the products, whether they are suitable for the intended purpose. He expressly assumes all risks associated with using the product and is solely responsible for any resulting damages. Safety data sheets and technical data sheets are available on the website of SAREMCO Dental.

### 14. Production / distribution

SAREMCO Dental AG  
Gewerbestrasse 4  
CH-9445 Rebstein / Switzerland  
Tel: +41 (0) 71 775 80 90  
Fax: +41 (0) 71 775 80 99  
info@saremco.ch  
www.saremco.ch

Edited: 09/2020 | D600137

Class IIIa medical devices  
Class III medical device (Canada)

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