# VOCO Registrado® Clear

GB

## Instructions for use

### Product description:

**Registrado Clear** is an addition-curing silicone impression material for bite registration characterised by its transparency, high final hardness exceeding 70 Shore A (measurement of hardness specifically for elastomers) and rapid setting.

### Indications:

- Bite registration
- Silicone key for composite restorations
- Front and occlusal record
- Registration key for intraoral bearing device registration
- Fixing of x-ray balls

### Method of Use:

Use **VOCO** Dispenser type 2 or any other suitable extruder gun to dispense **Registrado Clear**. Due to technical reasons there may be slight differences in the filling levels of the two cartridge chambers. Therefore, remove cartridge cap after placing the cartridge into the dispenser type 2 and dispense material until it is delivered uniformly from both chambers.

Then attach mixing tip type 16 and dispense material by pressing the lever. The material is automatically completely mixed in the correct ratio inside the mixing tip. For finer, pinpointed application, an intraoral tip type 2 may be inserted in the open end of the mixing tip type 16.

Place the cap back on the cartridge after use. Before using it again, check the opening of the cartridge each time, and if it is blocked, remove all of the material blocking it and then extract a small amount of material, until both pastes are delivered uniformly.

For occlusal bite registration **Registrado Clear** is applied directly onto the upper or lower dental arch of the patient. Bring bite into position within 20 s. The working time of **Registrado Clear** at room temperature (23 °C) is at least 25 s. After 90 s in the closed mouth the registration can be removed. After curing **Registrado Clear** can be easily trimmed with a scalpel or scissors and remains dimensionally stable.

### Technical Data:

Working time (at 23 °C)	≥ 25 s
Time in mouth (at 37 °C)	90 s
Shore A Hardness	> 70
Elastic recovery capacity	> 99 %

### Information, precautionary measures:

- **Registrado Clear** contains polydimethylsiloxanes and platinum catalyst. **Registrado Clear** should not be used in case of known hypersensitivities (allergies) to any of these ingredients.
- Do not use **Registrado Clear** in combination with condensation-curing silicones.
- The setting process of **Registrado Clear** may be impaired by latex gloves, hand creams, cleaning agents, mouthrinse/retraction solutions or residues from other materials (e. g. impression materials). These should therefore be removed prior to application (if spraying is required).
- Avoid contact of incompletely cured silicone material with clothes.
- Disinfection with an aqueous disinfectant is recommended before further use of the registrations. Follow the instructions of the disinfectant's manufacturer.
- The warranted product characteristics can only be achieved when using the original Sulzer Mixpac mixing tips as approved by **VOCO**. All the mixing tips for use with our respective products are available from **VOCO**.

### Storage:

Store at temperatures of 4 °C to 23 °C. Do not use after expiry date.

Our preparations have been developed for use in dentistry. As far as the application of the products delivered by us is concerned, our verbal and/or written information has been given to the best of our knowledge and without obligation. Our information and/or advice do not relieve you from examining the materials delivered by us as to their suitability for the intended purposes of application. As the application of our preparations is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our preparations in accordance with the existing standards and corresponding to the conditions as stipulated in our general terms of sale and delivery.

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