VOCO

Registrado® X-tra

GB

Instructions for use

**Product description:**

Registrado X-tra is an addition-curing silicone for bite registration with extreme final hardness which sets rapidly. Highly thixotropic properties prevent it dripping from the teeth and allow for application in thicker layers. The special temperature-controlled setting formula eliminates time pressure during application and has an extremely short setting time in the mouth. After only 40 s the registration can be removed without breaking or warping. **Registrado X-tra** can be easily trimmed with a scalpel or rotating instrument and stays dimensionally stable.

Indications:

Registrado X-tra is suitable for occlusal bite registration, fixing of face bow registrations, and it is a location material for intraoral pin tracing templates.

Application:**1. Preparation:**

Use a **VOCO** Dispenser type 2 or any other suitable dispenser.

Due to technical reasons there may be slight differences in the filling levels of the two cartridge chambers. Therefore, remove cartridge cap after placing the cartridge into the dispenser type 2 and extract material until it is delivered uniformly from both chambers.

Residues from other materials (e. g. impression materials) or solutions may interfere with the setting process and should be removed prior to application (by spraying, if required).

Fit a mixing tip type 16 and extract material by pressing the lever. The material is completely and automatically mixed in the correct ratio in the mixing tip.

An intraoral tip type 2 may be inserted in the open end of the mixing tip type 16 to facilitate application.

Place the cap back on the cartridge after use. Before using it again, check the opening of the cartridge each time, and if it is blocked, remove all of the material blocking it and then extract a small amount of material, until both pastes are delivered uniformly.

2. Handling:

In case of occlusal bite registration, apply **Registrado X-tra** directly onto the upper or lower dental arch of the patient.

The working time of **Registrado X-tra** is ca. 30 s (from commencement of mixing at a room temperature of 23 °C). After 40 s in the mouth **Registrado X-tra** can be removed.

Information, precautionary measures:

Avoid contact of the uncured silicone material with clothes. Do not use with condensation-curing silicones. Latex gloves, mouthrinse and retraction solutions can also have a negative influence on the setting process.

Disinfection of the registrations with an aqueous disinfectant is recommended.

The warranted product characteristics can only be achieved when using the original Sulzer Mixpac mixing tips as approved by **VOCO**. All the mixing tips for use with our respective products are available from **VOCO**.

Registrado X-tra contains polydimethylsiloxanes and platinum catalyst. **Registrado X-tra** should not be used in case of known hypersensitivities (allergies) to any of these ingredients.

Storage:

Store at temperatures of 4 °C to 23 °C. Do not use after expiry date.

Our preparations have been developed for use in dentistry. As far as the application of the products delivered by us is concerned, our verbal and/or written information has been given to the best of our knowledge and without obligation. Our information and/or advice do not relieve you from examining the materials delivered by us as to their suitability for the intended purposes of application. As the application of our preparations is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our preparations in accordance with the existing standards and corresponding to the conditions as stipulated in our general terms of sale and delivery.

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VOCO GmbH