VOCO

Structur Premium

EN Instructions for use

MD EU Medical device

Product description:
Structur Premium is a self-curing, highly aesthetic paste-paste system. Structur Premium has excellent mechanical properties and is fluorescent.

Shades:
A1, A2, A3, A3.5, B1, B3, BL

Indications:
Fabrication of temporary
- Crowns
- Bridges
- inlays and onlays

Contraindications:
Structur Premium methacrylates, amines, terpenes, benzoyl peroxide, BHT. Structur Premium should therefore not be used in patients with a known hypersensitivity (allergy) to these ingredients.

Patient target group:
Structur Premium is suitable for use in all patients without any age or gender restrictions.

Performance features:
The product's performance features satisfy the requirements of the intended use and the relevant product standards.


User:
Structur Premium should only be used by a professionally trained dental practitioner.

Use:
Insert Structur Premium cartridge into the VOCO Dispenser (type 2) or a compatible dispenser with a suitable piston. Due to technical reasons, there may be slight differences in the filling levels of the two cartridge chambers. In this case, remove cartridge cap and extract material until both chambers deliver uniformly. Then attach a mixing tip type 6 and lock with a 90° turn. By pressing the lever, the material is extracted and automatically mixed in the correct ratio. The warranted product characteristics can only be achieved when using the (original Sulzer Mixpac) mixing tips as approved by VOCO and available as accessories. The attached mixing tip is intended for single use only. After use, store the cartridge tightly sealed. Check that material flows freely through the openings before reusing.

Advice on application:
First level out any undercuts in the preparation using a non-eugenol cement. Take an impression if prefabricated crowns or similar restorative pieces are not being used. Prepare the impression by levelling undercuts and cutting drainage channels. Enlarge the impression if the wall thickness of the temporary restoration is expected to be too thin, taking care to remove the septums in the interdental embrasures. Clean and air-dry the impression afterwards. Apply Structur Premium directly into the impression. Do not overfill. The impression must be inserted into the mouth within 30 s. Remove the temporary from the mouth during the elastic phase, 1:00 min - 1:30 min after commencement of mixing. The setting process can also be monitored by testing the excess material in the mouth. The setting time of non-used material (in mixing tips) cannot be compared to the setting time of the material in the mouth.

At a room temperature of 23 °C and 50% humidity, working times are as follows:

min:s	min:s	min:s	min:s	min:s
0:00	0:30	1:00	1:30	4:00
insertion in the mouth	setting in the mouth	removal	complete setting	finishing

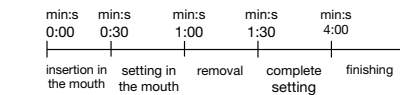Slight differences may result from different room temperatures. After approx. 4 min, take the temporary out of the impression and wipe dry. If necessary, a solvent such as ethyl alcohol may be used. This removes the thin inhibition layer usually caused by oxygen and allows easier finishing. The surface of the temporary restoration should appear silky-matt. Carefully trim any excess material and roughen areas near the gingiva. If required, polish the temporary restoration manually (e.g. with a rubber polisher, fine sandpaper or goat hair brush). Lute the temporary restoration using an eugenol-free temporary cement.

Warnings, precautionary measures:

- Before constructing the temporary, prepared cores should be insulated to promote desensitization (fluoride varnishes, core varnishes, etc.).
- In order to avoid fractures or injuries caused by fractures, the temporary should not be subjected to excessive loads (molar areas).
- Composite-based linings and core build-ups should be sufficiently insulated to prevent them from being accidentally removed at the same time as the temporary (e.g. glycerine).
- Inadequate hygiene or frequent use of certain mouthrinses may lead to discoloration.
- Finishing of the temporary should start only after the material has set completely (approx. 4 min after commencement of mixing).
- Restoration of defective temporaries:
If the temporary breaks despite the extreme stability of Structur Premium, the following procedures are recommended:
- Air bubbles: fill with Structur Premium.
- In case of fracture immediately after construction of the temporary: re-connect the fracture with freshly mixed Structur Premium.
- Repairing fractured temporaries that have already been worn: Roughen the fracture with a bur and place undercuts. Apply bond to the prepared fractures and cure according to the respective instructions for use. Apply a flowable composite on the prepared areas, assemble the fractured pieces and cure according to the instructions for use.
- Our information and/or advice do not relieve you of the obligation of checking that the products supplied by us are suitable for their intended purpose.

Storage:
Storage at 4°C-23°C. Store cartridges protected from light i.e. in closed packages, drawers etc. Do not use after the expiry date.

Disposal:
Dispose of the product in accordance with local regulations.

Reporting obligation:
Serious events such as death, temporary or permanent serious deterioration of a patient's, user's or other person's health condition and a serious risk to public health that arise or could have arisen in association with the use of Structur Premium must be reported to VOCO GmbH and the responsible authority.

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VC 60 BB1712 E1 0721 99 © by VOCO