trayloc® A

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Instructions for use

Vinyl Polysiloxane tray adhesive

Extra low viscosity adhesive for A-silicones, assures the adhesion between metal, plastic as well as individual trays and the impression material. Thin-layer surface modification ensures extraordinarily high adhesion. Can be applied quickly and simply, highly economical due to low-viscosity formulation. With colour indicator for fast, visual application check.

1. Application

Select a suitable impression tray (individual or standard tray made of metal, hard plastic), carefully degrease and clean (e.g. with isopropanol) after try-in in the patient's mouth. Make sure that the impression tray is clean, free of grease and dry. For optimum adhesion apply **trayloc® A** thinly and let it dry for around 3 minutes. Then bring the impression material onto the tray.

2. Tray cleaning

Remove set impression material roughly with a blunt instrument / spatula. The remaining material can be removed from the tray well by brief action of acetone or isopropanol. Then clean and disinfect the tray as customary.

Important hints

- Close the bottle again immediately after use.
- Only for dental use by qualified personnel.
- Observe the safety data sheet.
- Please follow the instructions of the safety data sheet!

Avoid contact with eyes and skin!

In case of inadvertent contact, rinse with plenty of water.

DETAX

GmbH & Co. KG
Carl-Zeiss-Str. 4 · 76275 Ettlingen/Germany
Telefon: 0 72 43/510-0 · Fax: 0 72 43/510-100
www.detax.de · post@detax.de

Made in
Germany

11/2015

Indications for use:

- Adhesive between impression tray and A-silicone impression material

Technical data:

- **Volume:** 17 ml (bottle)
- **Product colour:** blue-transparent
- **Drying time:** approx. 3 min.*
- **Storage:**

- * Higher temperatures accelerate, lower temperatures retard drying.

Ordering information:

trayloc® A **03098**
17 ml bottle with
brush

Inlay Trays
12 x large **02042**
12 x small **02043**
12 x mixed **02044**

Tray adhesive for C-silicones:

sili Adhesive **02051**
250 ml spray can

sili Adhesive fluid
25 ml bottle with
brush **02053**

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