(**GB**) (**DE**) (**FR**) (**ES**) (**IT**) (**PT**) (**NL**) (**SE**) (**DK**) (**NO**) (**RU**)

VOCO

Traypurol® tabs

(**GB**)

(**MD**) EU Medical device

Instructions for use

Product description:

Active-tabs for cleaning trays and instruments.

Traypurol tabs remove residue from alginates, zinc oxide eugenol, carboxylate and phosphate cements from impression trays and instruments without using ultrasound. Traypurol tabs are bio-degradable.

Product performance features:

The product’s performance features satisfy the requirements of the intended use and the relevant product standards.

User:

Traypurol tabs should only be applied by a professionally trained dental practitioner.

Method of use:

2 **Traypurol tabs** are needed for 1 litre of cleaning solution. Simply remove the tablets from the foil, place them in the cleaning container and fill with cold or warm water. Warm water accelerates the cleaning process. Avoid contact with skin.

Place trays and instruments in the **Traypurol tabs** solution after removing any large pieces of material. Standing the trays upright promotes the cleaning process.

Leave impression trays and instruments in the solution for at least 2 hours, but not more than 72 hours. Remove and rinse with clean water.

Traypurol tabs are suitable for all rustproof metals and plastics. Change the cleaning solution every 2 - 3 days.

Traypurol tabs can be used in ultrasonic baths, but not in dishwashers.

Contains complexing agent > 30 %, non-ionic tenside < 5 %, corrosion inhibitors.

Storage:

Store **Traypurol tabs** in a dry area at temperatures of 39 °F to 73 °F / 4 °C to 23 °C. Do not use after expiry date.

Warnings/precautionary measures:

- In case of contact with the eyes, rinse immediately with plenty of water and seek medical advice.

- If swallowed, seek immediate medical attention and present package.

- Our information and/or advice do not relieve you from examining the materials delivered by us as to their suitability for the intended purposes of application.

Disposal:

Dispose of the product according to local regulations.

Reporting obligation:

Serious events such as death, temporary or permanent serious deterioration of a patient’s, user’s or other person’s condition and a serious risk to public health that arise or could have arisen in association with the use of **Traypurol tabs** must be reported to VOCO GmbH and the responsible authority.

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VC 60 002356 E1 0320 99 © by VOCO
VOCO GmbH Phone **+49 (4721) 719-0**