# VOCO

## Vococid®

CE 0482

**EN** Instructions for use  
**MD** EU Medical device

### Product description:

**Vococid** is an etchant which can be used in the enamel etching technique or total-etch technique. The etching gel contains 35 % phosphoric acid, the etching liquid contains 36.1 % phosphoric acid.

### Indications:

**Vococid** is intended to etch only the part of the enamel or the dentine (in total-etch technique), which will subsequently be covered by a composite material:

- for composite fillings
- for adhesive luting of Maryland bridges, inlays, brackets and temporary splinting
- for sealing as caries prophylaxis
- for linings to create a permanent bond (according to manufacturer's instructions)

### Contraindications:

**Vococid** is contraindicated in patients with a known sensitivity or allergy to device chemical materials.

### Patient target group:

**Vococid** is suitable for use in all patients without any age or gender restrictions.

### Performance features:

The product's performance features satisfy the requirements of the intended use and the relevant product standards.

### User:

**Vococid** should only be used by a professionally trained dental practitioner.

### Use:

**Vococid** is both simple to use and easily visible on the enamel due to the blue colouring and consistency of the product:

With **Vococid** gel an exact application is possible (e. g. for etching the enamel margins) and the liquid is suitable for treatment of larger surfaces.

Before using the gel for the first time and also after long storage times, please apply just a little bit of the gel on a mixing pad to get a feeling for the consistency and the application characteristics of the material.

### Without total-etching (only etching of enamel):

Application of the gel: Apply **Vococid** directly from the cannula onto the tooth. Application of the liquid: Coat the surface of the enamel with one or two applications of **Vococid** on a **Pele Tim**.

The enamel surface must be completely moist. If necessary, apply extra etching liquid/gel.

**Vococid should be allowed to act for the recommended period of time as shown below:**

approx. 20 s for regular enamel and for enamel of adolescents; approx. 30 s for devitalized teeth, deeply fluoridated enamel or enamel without prisms (e.g. deciduous teeth).

After application rinse thoroughly with water, spraying each etched tooth for at least 15 - 30 s. Afterwards the patient should not close his/her mouth. Dry the etched surfaces thoroughly using an air blower.

Absolutely avoid contamination with saliva. In case of contamination, repeat the etching process (10 s). Check the etched areas (chalky-white).

### Total-etch-technique with dentine and enamel bonding materials (etching of enamel and dentine):

Etch entire cavity (enamel approx. 20 - 30 s, dentine max. 15 s). Remove **Vococid** etching gel/liquid by suction, rinse and remove excess moisture with an air-blower.

### Instructions for filling an empty 2 ml syringe:

- Wear protective gloves and safety glasses!
- Remove the protection caps from the 60 ml supply syringe and the empty 2 ml syringe.
- Ensure that the plunger of the empty 2 ml syringe is pressed down completely.
- Attach the 2 ml syringe to the 60 ml syringe and join them firmly by turning the 2 ml syringe clockwise (Caution: Do not overtighten the thread!).
- Decant the etching gel from the 60 ml syringe into the 2 ml syringe by slowly pulling the plunger of the 2 ml syringe (do not decant etching gel from the 2 ml syringe back into the 60 ml syringe!).
- Detach the syringes from each other by slowly turning the 2 ml syringe anticlockwise and close both syringes with their respective protection caps.
- To ensure batch traceability write the batch number and expiry date of the 60 ml syringe in use on one of the labels provided and attach it to the filled 2 ml syringe.
- To use the 2 ml syringe remove the protection cap, attach the application cannula and proceed in accordance with the above mentioned instructions for use.

**Please note:** 2 ml syringes may only be filled once, do not refill a second time! The application cannulae, too, are disposable devices. Syringes and application cannulae cannot be sterilised.

### Warnings, precautionary measures:

- **Caution!** **Vococid** contains phosphoric acid!
- Wear safety glasses and protective gloves!
- Avoid contact with eyes, skin and mucous membranes. In the event of accidental contact, rinse with plenty of water. Do not swallow!
- In case of contact with eyes, rinse the eyes with water for 15 minutes and seek medical advice.
- Avoid contact with gingiva, lining materials containing calcium hydroxide and neighbouring teeth.
- If total etching is not used, cover the dentine before etching, if necessary. It is recommended to cover adjacent teeth and etched enamel surfaces, which are not covered with composite material, once or twice with a fluoride varnish for remineralisation.
- Our information and/or advice do not relieve you of the obligation of checking that the products supplied by us are suitable for their intended purpose.

### Storage instructions and application method:

Storage at 4°C-23°C. Tightly close after use. Only give completely emptied packages to recycling centers. Do not use after the expiry date.

### Disposal:

Dispose of the product in accordance with local regulations.

**Vococid** residues can be disposed of with waste water, when highly diluted with water or neutralised.

### Reporting obligation:

Serious events such as death, temporary or permanent serious deterioration of a patient's, user's or other person's health condition and a serious risk to public health that arise or could have arisen in association with the use of **Vococid** must be reported to VOCO GmbH and the responsible authority.

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VC 60 AA1063 E1 0621 99 © by VOCO