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01.03.2015

Oraliner information:

self-curing resin especially for direct, permanently hard relining of acrylic dentures.

KKD® Oraliner is applied directly on the denture. There are no complicated laboratory procedures. A relining with KKD® Oraliner is as simple as taking a functional impression. In general the total time spent on a relining is less than 12 minutes for the dentist. KKD® Oraliner, free of methyl methacrylate, is based on the copolymers of ethyl methacrylate. This guarantees the best possible biological compatibility during and after the period of treatment. Due to a special formula of the activator/ accelerator system a perfect polymerization within 12-15 minutes is obtained which guarantees highest color stability and minimal residual monomer. The polymerization temperature is less than 40° C. KKD® Oraliner combines with every common denture resin.

Security instructions

KKD® Oraliner liquid is irritant. It has to be stored at a cool and shadowy place. Immediately close bottle after every use.

Presentation:
100 ml powder, 50 ml liquid,
1 mixing bowl, 1 spatula
REF 232005

Directions:

1. Prepare denture. Roughen the parts which have to be relined with a carbide bur or with sandpaper. These parts can be wettened with methyl methacrylate (liquid of any common self curing resin) to obtain an additional bonding between denture and relining.
2. The parts which do not have to be relined must be coated with a separator (e.g. vaseline). Pay attention that the insulation is not applied on the denture margin. It is recommended to leave a distance of 3-4 mm. Also acrylic teeth should not come in touch with KKD® Oraliner.
3. Mixing KKD® Oraliner. Mixing ratio 2 parts powder to 1 part liquid. Pour the liquid into a mixing bowl, add the powder and mix thoroughly. Mixing time approx. 30 seconds. Allow the paste to well up (approx. 3 minutes) to a viscous but still fluid consistency.
4. Apply mixture evenly onto the surfaces of the denture where it has to be relined.
5. Insert the denture furnished with KKD® Oraliner into the mouth of the patient as if taking an impression. Instruct the patient to close lightly into occlusion. After approx. 3 min. support the movement of the mimic muscles.
6. After initial setting and when the mixture has reached its elastic phase (after approx. 4 minutes), remove the denture from the mouth for control and if necessary trim off excess material. Reseat denture immediately and leave it for 3-4 minutes. Eventually support the movement of the mimic muscles again.
7. For final setting immerse the denture for about 5 minutes in warm water of approx. 40° C. The total polymerization time is maximal 15 minutes. At normal layers of approx. 1 1/2 mm the polymerization temperature comes up to maximal 40° C. For optimum results (practically free of residual monomer) and faster curing we recommend to use a pressure pot filled with warm water of approx. 50° C and cure at pressure of 2 bar. We recommend TOPDENT Polypot, the practical and safe polymerization unit (independent of compressed air, no electrical parts).
8. Finish and polish as usual.

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KENTZLER-KASCHNER DENTAL GmbH