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OraSoft information:

self-curing resin especially for direct and soft relining of acrylic dentures

KKD® OraSoft is applied directly on the denture. There are no complicated laboratory procedures. A relining with KKD® OraSoft is as simple as taking a functional impression. In general the total time spent on a relining is less than 12 minutes for the dentist. KKD® OraSoft, free of methyl methacrylate, is based on the copolymers of ethyl methacrylate. This guarantees the best possible biological compatibility during and after the treatment. KKD® OraSoft reliner is intended for a wearing time of at least 3 months. KKD® OraSoft combines with every common denture resin

Indication of KKD® OraSoft

- if the patient has a thin, non-elastic, delicate and irritated mucosa
- if a hard palate makes wearing a denture complicated
- after jaw surgeries or anatomic alterations
- if the basal area is narrow, small or irregular
- if the patient cannot wear a denture with a hard basal area because of physical or psychical reasons
- if free-end saddle or partial dentures cause pain or discomfort
- for better border molding
- with immediate dentures

Presentation:
80 ml powder, 50 ml liquid,
1 mixing bowl, 1 spatula
REF 230335

Directions for cleaning: If treated duly, the KKD® OraSoft relining will stay soft and elastic for at least 3 months. Periodical cleaning under running cold water is recommended. Only very soft brushes should be used. Do not use any aggressive cleanser.

Security instructions: KKD® OraSoft liquid is irritable. It has to be stored at a cool and shadowy place. Immediately close bottle after every use.

Directions:

1. Prepare denture. Roughen the parts which have to be relined with a carbide bur or with sandpaper. Eventually denture base material has to be taken off to get enough space for the relining. The parts which have to be relined can be wetted with methyl methacrylate (liquid of any common self curing resin) to obtain an additional bonding between denture and relining.
2. The parts which do not have to be relined must be coated with a separator (e.g. vaseline). Pay attention that the insulation is not applied on the denture margin. It is recommended to leave a distance of 3-4 mm. Also acrylic teeth should not come in touch with KKD® OraSoft.
3. Mixing KKD® OraSoft. Mixing ratio 2 parts powder to 1 part liquid. Pour the liquid into a mixing bowl, add the powder and mix thoroughly. Mixing time approx. 30 seconds. Allow the paste to well up (approx. 3 minutes) to a high viscous consistency.
4. If the patient is allergic or has an irritation, it is recommended to rub the gingiva lightly with vaseline to avoid direct contact with the unset reliner.
5. Apply the mixture evenly onto the surfaces of the denture where it has to be relined.
6. Insert the denture furnished with KKD® OraSoft into the mouth of the patient as if taking an impression. Instruct the patient to close lightly into occlusion. After approx. 3 minutes support the movement of the mimic muscles.
7. After initial setting and when the mixture has reached its elastic phase (after approx. 3 minutes), remove the denture from the mouth for control and if necessary trim off excess material. Reseat denture immediately and leave it for 3-4 minutes. Eventually support the movement of the mimic muscles again.
8. For final setting immerse the denture for about 5-10 minutes in hot water of 50°-60° C. The total polymerization time is maximal 12 minutes. For optimum results (practically free of residual monomer) and faster curing, we recommend to use a pressure pot filled with warm water of approx. 50° C and cure at pressure of 2 bar. We recommend TOPDENT Polypot, the practical and safe polymerization unit (independent of compressed air, no electrical parts).
9. Finish and polish as usual. For better workability of the soft relining, lay the denture at a cool place or in ice cold water for a short time. The relining becomes hard and can be finished without clogging. At mouth temperature the relining will get soft again.

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KENTZLER-KASCHNER DENTAL GmbH