# MUPPY

Standard  
starr & elastisch

D
GB
F
I
E

**Patienteninformation**

## MUPPY zur Prophylaxe

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**Patient information**

## MUPPY for prophylaxis

Even if there is still no well-developed malposition, the damaging sucking habit can be ended with the help of **MUPPY**. **MUPPY** is offered to children as substitute, which they voluntarily accept and wear because it is not forced on them.

The easy **MUPPY** standard also helps with faulty (infantile) swallowing. Tongue habits such as lip biting and sucking serve to switch from mouth to nasal breathing.

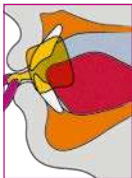
## MUPPY - rules

Studies have shown that more than 90 % of children give up damaging sucking habits with the help of **MUPPY** . Your support and a few rules help your child:

- 1 **MUPPY** is a substitute for for a thumb or a dummy  
The substitution must be voluntary.
- 2 **MUPPY** must always be worn at night.
- 3 To get used to it easier children should also wear **MUPPY** a few hours during the day (when playing or watching television).
- 4 As a rule, weaning is successful within a week. However, **MUPPY** should also be worn for 6–8 weeks after successful weaning of the dummy ("recidivism").
- 5 **MUPPY** can be stored in the hygiene box.
- 6 Do not forget to regularly brush the teeth and **MUPPY!**

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**MUPPY** is worn similarly to a common weaning dummy, lying loosely behind the lips and in front of the teeth, in the so-called oral vestibule. It is held automatically in place by the lip muscles.

### **If your child is having speech therapy treatment, you should take note of the following:**

- Follow your speech therapist's individual wearing and exercise instructions exactly.
- Support your child during this stage, as he/she has to feel the importance of mouth closure in order to take the exercises seriously.
- Make your child aware of the "rules of play" and supervise the necessary exercises.

## Instructions for use

Before using **MUPPY** for the first time, clean it thoroughly under warm running water. Check **MUPPY** regularly for places that have been bitten through! Replace **MUPPY** when necessary.

## Cleaning instructions

Clean thoroughly with a toothbrush under warm running water. Alternatively, you can also use the OROFAN® dental cleaning tablets. **MUPPY** is neither dishwasher-safe nor microwave-safe. Only clean the hygienic **MUPPY**-Box with a damp cloth.

## MUPPY - designs

**MUPPY standard** is available in two different sizes:

- with violet ring for deciduous dentition (size I)
- with blue ring for mixed dentition (size II)

There are also different hardness grades (elastic, rigid) available.

**MUPPY standard** rigid  
(transparent)

**MUPPY standard** elastic  
(transparent)

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