

KaVo PiezoLED™ Ultrasonic Scaler Adjustment Recommendation

Product Code	Therapeutic Indication	Power Setting	Spray Water Volume	Mode Adjustment*
PIEZO Scaler Tips	Scaler tip 201			P3 P2 P1 (E)
	Scaler tip 202			P3 P2 P1 (E)
	Scaler tip 203			P3 P2 P1 (E)
PIEZO Paro Tips	Paro tip 210 + 211			P3 P2 P1 (E)
	Paro tip 212 + 213			P3 P2 P1 (E)
	Paro tip 214			P3 P2 P1 (E)

recommended
 possible
 not recommended

* Only applicable for ESTETICA treatment units and Primus 1058 Life.
PiezoLED for Primus 1058 is bound to P3.

KaVo PiezoLED™ Ultrasonic Scaler

PIEZO Prep Tips	Product Code	Therapeutic Indication	Power Setting	Spray Water Volume	Mode Adjustment*
	CEM tip 225				(P3) (P2) (P1) E
	Prep tip 226				P3 P2 P1 (E)
	Prep tip 227 + 228				P3 P2 P1 (E)
	Prep tip 229				P3 P2 P1 (E)

PIEZO Endo Tips	Product Code	Therapeutic Indication	Power Setting	Spray Water Volume	Mode Adjustment*
	Endo tip 220				P3 P2 P1 (E)
	Endo tip 221	Tip not applicable for Primus 1058 			P3 P2 P1 E
	Endo tip 222 with files	Tip not applicable for Primus 1058 			P3 P2 P1 E

PIEZO Implant Tips	Product Code	Therapeutic Indication	Power Setting	Spray Water Volume	Mode Adjustment*
	Implant tip set				P3 P2 P1 (E)

■ recommended
 ■ possible
 ■ not recommended

* Only applicable for ESTETICA treatment units and Primus 1058 Life. PiezoLED for Primus 1058 is bound to P3.

Dieses Dokument wird Ihnen zur Verfügung gestellt von:

Med-Dent24 Handelsgesellschaft mbH
Gotenweg 7
63128 Dietzenbach

Telefon: +49 (0)6074 803 3615
Telefax: +49 (0)6074 803 4092
E-Mail: info@med-dent24.com

Registergericht: Offenbach am Main
Registernummer: HRB 56518
Umsatzsteuer-Identifikation-No.: DE307869540

Geschäftsführerin: Ana Ban