## Zyklonabscheider ZK1

## Cyclone separator ZK1

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---

## Dear Customer

Thank you for choosing this high-quality product from BDT.

In order to achieve the best results with this product, it is important to carefully study and follow these instructions for use.

These written instructions cannot cover all possible situations that can show up during use. For this reason, our hotline is available to answer all questions or concerns that may arise.

Due to constant developments, we recommend, even when using the same products regularly, to study the enclosed current instructions for use or to refer to our website at [www.bdt-dental.de](http://www.bdt-dental.de).

## Manufacturer

BDT GmbH • Industriestr. 27 • 77656 Offenburg • Germany

Telephone +49 781/58 688

Fax +49 781/58 664

E-Mail [info@bdt-dental.de](mailto:info@bdt-dental.de)

## With reference to our quality standards

BDT ensures a faultless quality of the products manufactured by us. The content of these instructions is based upon our own experiences. The user is responsible for the correct use of the products.

Responsibility for failures cannot be taken, as we have no influence on the processing on site.

---

# Instruction for use

## Cyclone separator ZK1

### 1. Safety instruction

#### 1. Intended use

The cyclone separator is intended exclusively for collecting non-combustible solids in the dental/orthodontic laboratory. Any other use or use that goes beyond this is deemed to be improper. Proper use also includes observing all the information in the instruction's manual. When finally decommissioning the BDT device, the relevant country-specific regulations must be observed. Questions about the proper disposal of the device will be answered by BDT or the dental specialist trade.

#### 2. Symbols and identification of sources of danger

The following symbols for hazards are used in the instructions for use:

**Warning** This symbol indicates direct danger to the life or health of the user. Disregarding this warning may involve a serious risk to health.

**Caution** This symbol indicates important safety instructions. Failure to follow these instructions can endanger the safety of your device and/or the property of others.

#### 3. Disposal

This symbol indicates that products marked with it should not be disposed of as normal household rubbish. The legislature prohibits commercial customers from returning old devices via municipal collection points. Further information is available through BDT or the dental trade.

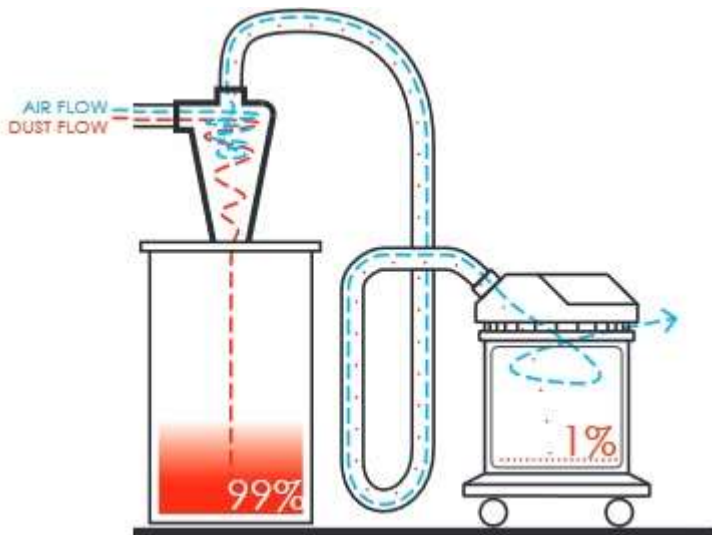
---

## 2. Area of application and description

The cyclone separator is used to pre-separate milling dust such as zirconium. The cyclone separator is installed upstream of the extraction system and enables wet or dry dust particles to be separated, reducing the contamination of your dust filters and increasing their life-span. 99% of the dust particles are collected in the cyclone separator, which leads to a significant increase in its life-span. This reduces ongoing operating costs of the dust filters. The cyclone separator is made of PETG and was manufactured using a 3D printing process. Bucket and lid are made of PP.

## 3. Principle of operation

The suction flow, consisting of a mixture of air and dust, enters the filter via the tangential inlet. The shape of the filter is designed to create a vortex to allow the heavier-than-air particles to fall out through the lower outlet under the action of gravity and the air to escape through the upper outlet.

---

## 4. Installation steps for commissioning

1. Mount the cyclone filter on the lid. To do this, slide the seal over the thread. Insert the cyclone through the opening in the lid and fasten the cyclone with the nut M 72. Screw the safety valve on the 1 ½ inch thread.
2. Place the foil insert in the middle of the bucket. The foil insert prevents the foil from being pulled and the cyclone from clogging.
3. Mount the lid on the bucket using the clamping ring.
4. Mount the existing hose on the appropriate adapter. Adapter 1 has a diameter of 60 and 50mm. Adapter 2 has a diameter of 40, 38, 36 and 34 mm.
5. Insert the adapter from the milling machine into the vertical opening.
6. Insert the adapter from the suction into the horizontal opening.
7. Set the suction power of the suction as low as possible, but strong that the dust produced is aspirated.
8. Set the preload on the safety valve so that it is completely closed during normal operation. In the event of a blockage in the suction hose, the safety valve is opened by the negative pressure.

## 5. Cleaning

Empty the bucket when it is 75% full. You are able see the level through the transparent bucket.

1. Open the clamping ring and remove the lid.
2. Pull the foil insert out of the bucket.
3. Remove the insert foil from the bucket and dispose it appropriately.
4. Insert a new foil and place the foil insert in the middle.
5. Put the lid on the bucket and close the lid with the clamping ring. The flat side of the clamping ring must point upwards.

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